making work easy

- FAQ - Deutsch
- FAQ - English
- FAQ - Français
- FAQ - Italiano
- FAQ - Español
- FAQ - Português
- FAQ - Türkçe
- FAQ - Русский
- FAQ - Polski
- FAQ - Čeština
- FAQ - 中文
- FAQ - 日本語
- FAQ - 한국어
- FAQ - اللغة العربية
- FAQ - Українська

MILLO

FAQ - frequently asked questions

Made in Germany

21-6543 30032016

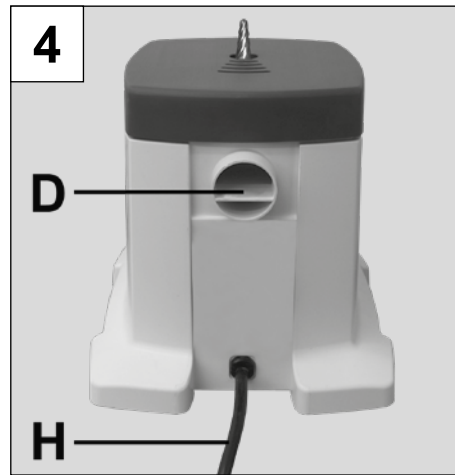
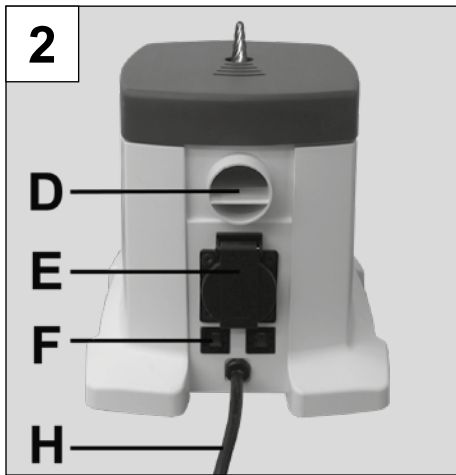
Fig. 1

FAQ - Deutsch

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FAQ - English

i For information on operating elements, see [Fig. 1](#) on page 2 of this document.

i For chapter indications, refer to the complete instruction manual on the internet.

Problem	Cause	Remedy
Unit does not operate after switching it on.	<ul style="list-style-type: none"> • Mains cable not plugged in. • No electricity supply to the mains socket. • Unit connected to an extractor that is switched off. 	<ul style="list-style-type: none"> • Plug in the mains cable. • Check the mains socket. • Check the extractor, refer to the extractor instructions if necessary.
Unit switches off during operation.	<ul style="list-style-type: none"> • Motor fuse triggered, as the motor temperature is too high. • Fuse triggered. 	<ul style="list-style-type: none"> • Unplug the unit and allow it to cool for approx. 1.5 hrs. • Use an extractor (see Section 1.3). • Switch on the safety cut-out / replace fuse (see Section 3.3). • If the fuse triggers again, return the unit for repair.
Heavy dust accumulation on the work table.	<ul style="list-style-type: none"> • Extractor not used. • Extraction power too low. 	<ul style="list-style-type: none"> • Use an extractor (see sect. 1.3). • Check the filter of the extractor. • Check that the suction hose is not bent. • Check that the suction hose is fitted securely to the connector (D, Figure 2.4). • Use a higher extraction setting.
Fuse triggers.	<ul style="list-style-type: none"> • Load of the connected extractor too high, adhere to the max. connected load (see technical data). 	<ul style="list-style-type: none"> • Connect the extractor separately to the electricity supply.
Reduced removal or application of greater pressure required.	<ul style="list-style-type: none"> • Cutter blunt 	<ul style="list-style-type: none"> • Change the cutter (see Section 2.5).

Problem	Cause	Remedy
Cutter slips out during operation.	<ul style="list-style-type: none"> • Cutter not inserted far enough into the clamping ring. • Clamping ring not tightened enough. 	<ul style="list-style-type: none"> • Insert the cutter fully into the clamping ring. • Tighten the clamping ring further.
Cutter cannot be inserted.	<ul style="list-style-type: none"> • Original Renfert cutter not used. 	<ul style="list-style-type: none"> • Always use original Renfert cutters.
Cutter cannot be removed.	<ul style="list-style-type: none"> • Clamping screw not loosened enough. • Corrosion on the cutter shank. 	<ul style="list-style-type: none"> • Loosen the clamping screw further. • Return the unit for repair.
Clamping screw cannot be loosened.	<ul style="list-style-type: none"> • Screw overtightened. 	<ul style="list-style-type: none"> • Return the unit for repair.
Connected extractor does not operate.	<ul style="list-style-type: none"> • Extractor not switched on. • Extractor is in automatic operation mode. • No electricity supply to the plug-in socket. 	<ul style="list-style-type: none"> • Switch on the extractor. • Set the extractor to continuous operation. • Return the unit for repair.

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