# Clinical Guide

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Confidence  
Vertrauen

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Esthetics  
Ästhetik

# IPS Empress® Direct

Direct Esthetic Composite

# IPS Empress® Direct

## Clinical Guide

### Dear Doctor

Thank you for purchasing IPS Empress Direct – a restorative system which allows you to meet the highest restorative requirements.

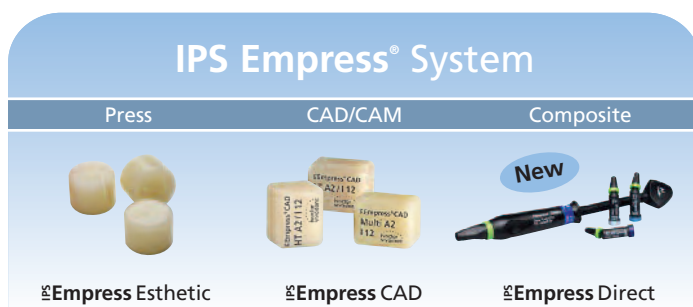
### IPS Empress® System

For nearly 20 years, the name IPS Empress has been associated with the highest level of esthetics in the reproduction of natural tooth characteristics. With more than 37 million restorations fabricated with IPS Empress to date, the system has gained a reputation for producing long-lasting, natural-looking results and maintaining high quality standards.

Over the years, the IPS Empress System has been further developed keeping pace with the latest trends in cosmetic dentistry. As a result enhancements have been made to the material's esthetics, function and processing properties.

In 2004, the system was further expanded and optimized with the introduction of IPS Empress Esthetic pressable ceramic. This was followed by the release of IPS Empress CAD in 2006, creating an opportunity for esthetic restorations to be milled chairside using CAD/CAM technology, and placed immediately.

IPS Empress Direct is the latest addition to the IPS Empress System. It provides dentists with the esthetics of a ceramic and the convenience of a composite.

### IPS Empress Direct

IPS Empress Direct is a universal nano-hybrid filling material for direct esthetic restorative procedures. IPS Empress Direct offers a high-gloss polish and the lifelike opacity, fluorescence and opalescence necessary to create true-to-nature, esthetic restorations.

### Material

The new composite material for anterior and posterior restorations features excellent polishing properties, imparting a long-lasting natural-looking lustre to restorations.

Given the material's low sensitivity to ambient and dental operatory light, dentists have ample time to restore the natural anatomy and the optical appearance of teeth, under ideal lighting and stress-free working conditions.

The high radiopacity allows teeth to be checked for secondary caries with ease. Due to the smooth consistency of IPS Empress Direct, the material can be precisely applied in natural-looking layers.

A large number of researchers and colour specialists contributed their knowledge and experience to the development of the coordinated shade system of IPS Empress Direct. Their efforts have produced a selection of 32 different shades in 5 levels of translucency.

### Application

Natural teeth are made of different types of hard tissue, which exhibit different degrees of opacity: Dentin, for example, is clearly more opaque than enamel.

The 32 different IPS Empress Direct materials allow you to cover a wide range of anatomical conditions and variations. They are divided into 13 Dentin shades, 16 Enamel shades and 3 Translucent shades, which are characterized by different optical properties. The opaque **Dentin shades** are primarily designed for the reconstruction of the dentin core and the mammelons. They are identified by **dark blue** markings. The semi-translucent **Enamel shaded** materials are especially designed for reconstructing the enamel layer. The Enamel shade range also includes very light Bleach shades, which can be used for e.g. the reconstruction of bleached teeth. The Enamel materials have **light blue** markings. Additionally, **Translucent shades** are available. They are mainly used for the incisal edges of restorations placed in younger patients or for characterizing restorations on the surface. The numbers in the names "Trans 20" and "Trans 30" stand for the respective degree of translucency in %. Trans 20, for example, features a degree of translucency of 20%. Moreover, Trans-Opal material is available, which appears blue or orange, depending on the incidence of light. The opalescent effect primarily found in adolescent patients can be reliably mimicked with this material. The Translucent shades have **white** markings.

As the IPS Empress Direct Enamel, Dentin and Translucent materials can be applied using various layering methods, they provide users with a level of creative freedom that matches their working techniques, requirements and experience.

Enjoy creating with IPS Empress Direct.

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# Case presentation 2 – Elderly patient

## Fall 2 – älterer Patient

1  
Preoperative situation: Class IV, facial view (please note the central enamel crack)  
Ausgangssituation Klasse IV, Ansicht von fazial (man beachte den zentralen Schmelzriss).

2  
Preparation line marked  
Geplante Präparationslinie

3  
Wave-line preparation completed  
Fertige Wellenschliffpräparation

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# Class IV restoration with enamel crack

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15  
Completed restoration after polishing  
Fertige Restauration nach Politur

# Case presentation 3

## Fall 3

1  
Preoperative situation: Class V cavity; facial view  
Ausgangssituation Klasse V-Kavität (Ansicht von fazial)

2  
Disto-facial view  
Ansicht von disto-fazial

3  
Preparation line marked  
Geplante Präparationslinie

4  
Completed preparation  
Fertige Präparation

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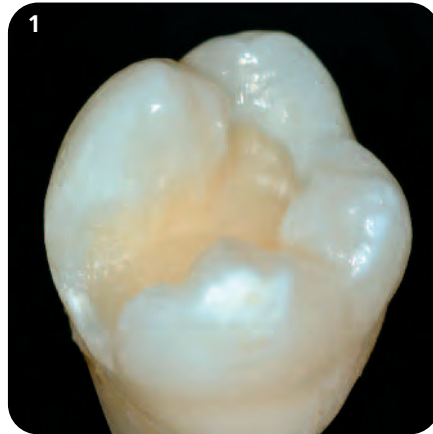
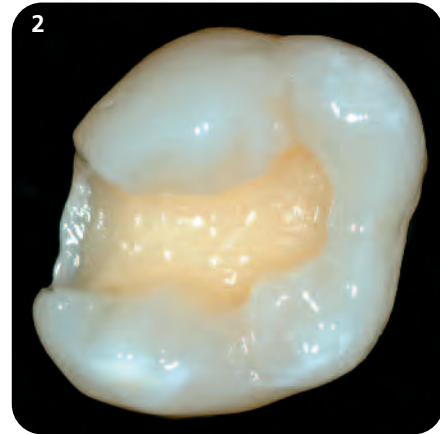
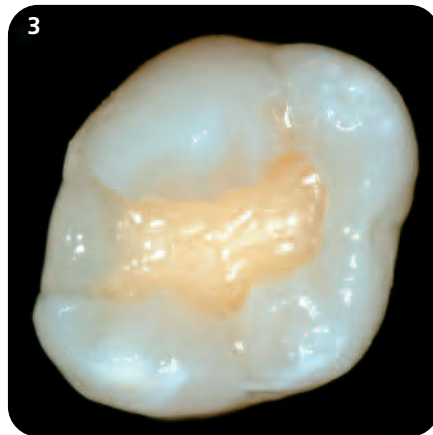
8  
Mesial view  
Ansicht von mesial

9  
The dentin core is rebuilt with Dentin material; third layer.  
Restauration des Dentinkerns in Dentinmasse, dritte Schicht

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9  
The restoration was completed with a layer of Trans 30 and subsequently polished. Tooth 12 was restored according to the same principle. View immediately after the restoration had been completed.

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10  
The restoration was completed with a layer of Trans 30 and subsequently polished. Final result after 24 hours and rehydration of the dental hard tissue.

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Application of the bonding agent  
Bondingapplikation

# Restoration with opalescent effect Dr Markus Lenhart, Switzerland

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7  
Completed build-up of the palatal wall  
Fertige Dimension der palatinalen Wand

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