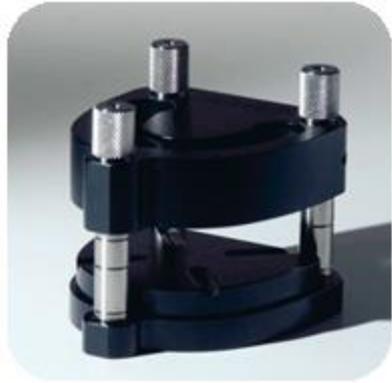
FINALIZATION OF DENTURE

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anaxFORM Metal Flask

anaxFORM Base Former

The complete denture modeled

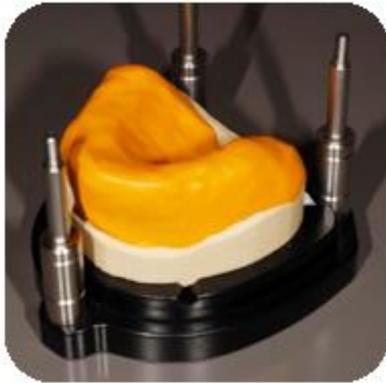
Position in the **anaxFORM** Large Metal Flask

Adjust the height of the lid for clearance

High definition 1:1 putty silicone, Matrix form 60.

Apply a thin layer over the entire wax version

Check for clearance of the lid again

For more precise detail, place in pressure pot for 5 minutes at 2 bar

Mix 4 – 5 level scoops of Matrix Form 70 putty, and use half of the material to cover the matrix completely. Use the other half of the mixed material to...

... fill the opening in the lid of the flask...

...then press the lid of the flask down until it reaches the vertical stops.

Remove excess putty and allow to set for 4 – 5 minutes

Remove the prosthesis and generously apply Separator to the stone model

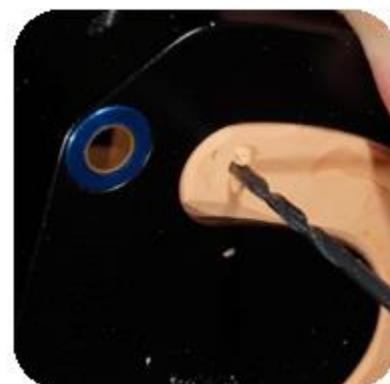
For easier reinsertion,
trim the edges at a 45
degree angle

Use the large drill to
create the injection
channel

Use the small drill to
create the exit
channel

Remove any debris
from channels

Create a reservoir at each channel site

Acryline Starterkit

Use a digital scale to weigh powder

Powder:Liquid ratio = 10:7
For full upper denture, use
approximately 18-26g
Powder und 9-13g Liquid

Add liquid to achieve full weight at 10:7 ratio

Mix thoroughly for 20 seconds

Pour mixture into a syringe in a thin stream

Insert syringe plunger

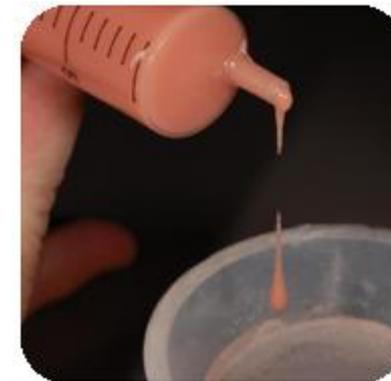
Turn syringe upward and extrude air

Allow bubbles to rise to top, and express additional air

Allow to polymerize in syringe for 1 – 2 minutes (exact time depends on the temperature of the material)


Ideal polymerization time produces a honey-like consistency


Position the flask at a 90 degree angle with injection channel on bottom

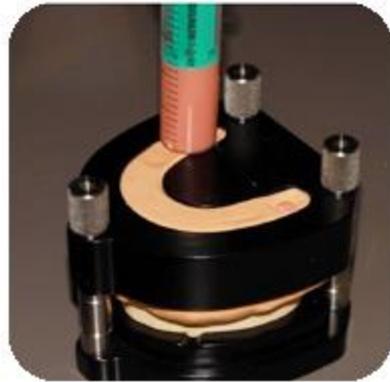Inject acrylic slowly and with consistent pressure

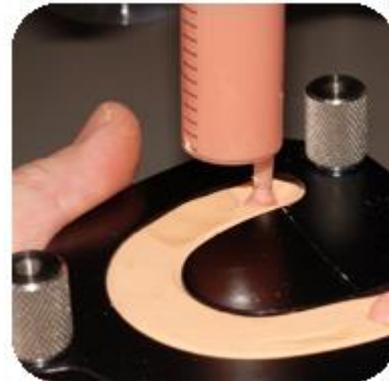Allow the exit channel reservoir to fill with material

Important! When removing the syringe, nachinjizieren

Polymerize in the pressure pot for 25 minutes at approx. 115°F (45°C)

Remove the funnel with a wax knife

Open the flask (shown without funnels removed)

This process can now be repeated to make a spare denture, a clear stint (Acryline Clear) and/or a scanning appliance (Acryline X-ray)

Final Prosthesis

Large Metal Flask includes large and small silicone drills

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