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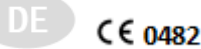
NOVODent (ISO 20795-1 Type 1, Cl.1)

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INSTRUCTION

NOVODON (ISO 20795-1 Type 1, Cl.1)

The product that you have purchased qualifies as a medical Device under European Directive 93/42/EEC. After receipt of the product please ensure that you file the lot number and product name in your administrative system lay down. For each work that you produce, please make a note of all the materials used along with all the used

lot numbers. Please follow the important guideline in the following instructions for your own protection as well as of your patients and the environment. May be used only by qualified personnel

Indication:

Pack and press technique and injection technique:
partial dentures and full dentures

Contraindication:

Avoid direct contact within the oral cavity with unpolymerised material if a patient has hypersensitivities against or towards a component of NOVODON we recommend not to use it

Advice

The product contains methyle methacrylate compounds, which may cause sensitization by skin contact. Discontinue using the product if symptoms of irritation occur. Do not use NOVODON in case of known allergies to resins based on methyle methacrylates.

Connecting to other medical devices:

NOVODON can be used with metal alloys and metal frameworks (partial dentures), metal scaffolding, Silanisierungssystemen, opaque, artificial teeth and metal braces. Can be combined with cold polymerization acrylic

Material Composition:

Powder: Polymethylmethacrylate, Benzoylperoxide, Barbiturat - no phtalathalt
Liquid: Methylmethacrylat-Monomer, Dimethacrylat, Catalystr

Shades:

C Clear, 11 yellowish pink, 22 pink - also available veined (V).

Benefits

Very easy handling, best mechanical properties. absolutely colour-stable, cadmium free.

Processing:

Mixing ratio: 2, 8 - 3 weight parts of powder: 1 weight part of liquid. Too much monomer liquids will modify consistency, handling and shades characteristics of this material. The polymerization is also affected and can lead to mucous membrane irritation and porosities. First pour liquid and then the powder into mixing cup and mix with spatula thoroughly for 30 seconds. Rest mixture for 7 – 9 minutes (depending on room temperature) in covered cup. If left covered, the mixture can be processed for up to 1 hour. The matured Denture Base Material must be well kneaded and packed in a cold flask with Novodent Separator Universal isolated plaster model. Close flask slowly. With adequate surplus of material you may omit trial packing. The closed flask may be immediately placed into boiling water. Polymerisation must be done for 30 minutes in boiling water. Polymerisation must be done for 30 minutes in boiling water; this must absolutely be observed. The 30 minutes must be considered from the moment the water starts boiling again. After ploymerisation, the flask may 20 minutes stand in air after be cooled in water. By using the recommended instructions can be achieved a residual monomer content of < 2%.

Durability and storage

The shelf life is 3 years. The expiry date is clearly marked on the product. Do not use NOVODON after expiry date. Keep away from children. Store NOVODON tightly closed in a cool, dark and well ventilated place. Storage temperature: +5°C – +25°C (41°F-77°F).

Safety information:

The liquid contains methyl methacrylate (MMA) and is irritant as well as easily flammable. (Flash point +10°C, 48°F). The liquid is harmful to the eyes, the respiratory system and the skin. Possible skin reactions by direct contact. Keep in a safe, well ventilated place. Do not inhale vapors. Keep away from fire – do not smoke. This product must be disposed of safely, not through the drains. Protect from electrostatic sources. Wear a mask when grinding and use air suction. Always use spatula when handling. If a patient is known to be allergic to any of the ingredients in NOVODON, the material must not be used. During storage of the finished work in warm water, or virgin olive oil 3 days or more, the residual monomer content decreased to values similar to hot curing dentures.

Protection hands:

Disposable gloves butyl rubber, layer thickness 0,7mm, permeation (min) >480

Disposable gloves nitrile rubber, layer thickness 0,4mm, permeation (min) >120

The exact break trough time has to be requested from the protective hemp shoe manufacturer and must be observed.

For damage that will result from failure to observe the instructions and the application area no liability. This also applies when the product is mixed with products of competitors or processed together.

Quantities

Single packing

Powder: 200 g, 500 g, 1 kg, 5 kg, 10 kg

Liquid: 100 ml, 250 ml, 500 ml, 1 l, 2,5 l, 5 l

GHS: Danger

Expiry date

Storage temperature
Values min. max

NOVODENT ETS.

Industriestrasse 32
9487 Benden

Principality of Liechtenstein

Tel.: +423 377 1500
Fax.: +423 377 1509

info@novodent.com
www.novodent.com