SILENT TS2

2930 0050 / 2930 1050

Quick Start Guide

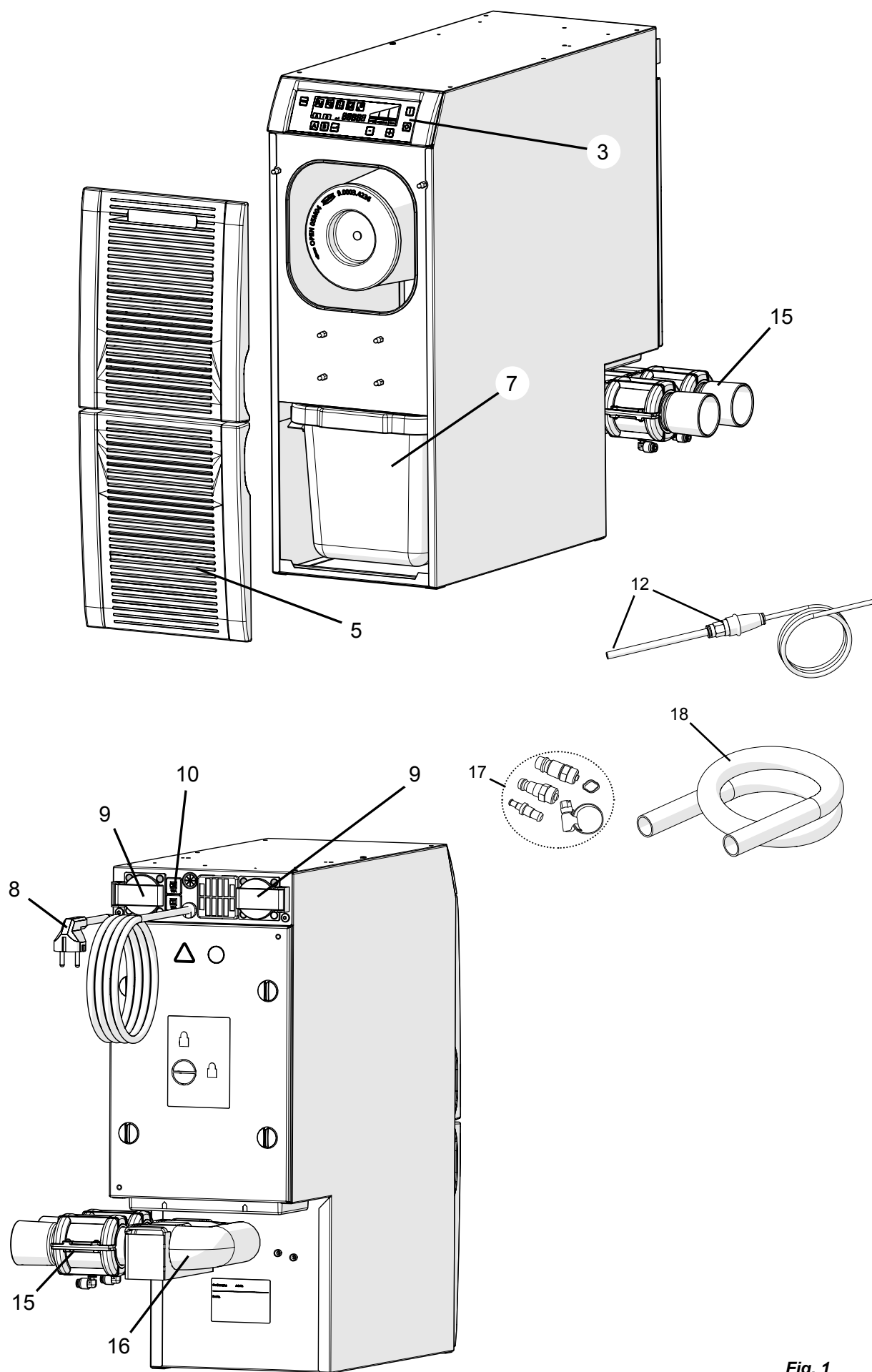
Fig. 1

Einleitung

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Introduction

This Quick Start Guide contains instructions on the operation of the product and relevant safety information.

Detailed information on the product can be found in the operating instructions which are available for download on our website at www.renfert.com.“

Symbols

In the instructions for use and on the unit itself you will find these symbols with the following meanings:

Danger

This indicates a direct risk of injury. Consult accompanying documents!

Electrical current

This indicates a risk of hazard due to an electrical current

Attention

Disregarding this warning may result in damage to equipment..

Note

This provides the operator with useful information to improve and ease use.

The device complies with the requirements of the applicable EU directives.

The device is subject to the EU directive 2002/96/EG (WEEE directive).

Other symbols are explained as they occur.

Safety

Intended Use

The product is intended for indoor use for the extraction of dry, non-explosive dust particles. The ambient temperature must be between 5 - 40 °C.

This device is intended solely for commercial use in a dental laboratory.

The intended use also includes compliance with the instructions specified by the manufacturer concerning operation, servicing and maintenance.

Hazard and Warning Information

General Information

- ▶ If the device is not used in compliance with the supplied instructions, the safety of the device can no longer be guaranteed.
- ▶ Only operate the device under supervision.
- ▶ The mains plug must be easily accessible.
- ▶ Disconnect the device from the mains before carrying out work on the electrical parts.
- ▶ The product does not contain any parts inside which require servicing.
- ▶ Defective devices must be put out of service immediately. Remove the mains plug and ensure the device is not used. Send the device for repair!
- ▶ The product may not be opened, except as instructed below.

Specific Information

- ▶ Please observe the national regulations and permitted exposure to dust in a working environment. Please ask the “National Institute for Occupational Safety and Health” or other responsible authority
- ▶ It is necessary to wear suitable personal protective equipment when emptying the dust drawer or cleaning, depending on the type of extracted material.
- ▶ Do not extract flammable or explosive gasses, fumes or dust.
- ▶ Do not extract hot materials.
- ▶ Do not extract liquids.
- ▶ Dispose of extracted material according to local statutory regulations.

Authorized Persons

Operation and maintenance of the device may only be performed by qualified personnel.

Product Description

See Fig. 1 (on inside cover)

- 3 On / off switch
- 5 Front panel/ Dust drawer
- 7 Dust drawer
- 8 Mains cable
- 9 Device coupler socket (A, B)
- 10 Device protection switch
- 15 Pinch valve with extractor connection fitting (rotatable)
- 16 Suction pipe
- 17 Connection set
- 18 Suction hose

The other units are covered in the full set of operating instructions available on the internet.

Scope of Delivery

- 1 Silent TS2
- 1 Operating instructions
- 1 Quick reference card
- 1 Dust bag (ready for use in the dust drawer)
- 1 Compressed air tube (sealed)
- 2 Pinch valves with suction port
- 1 Connection set
- 2 Shockproof plug (only for 2930-0050)
- 1 4 m Suction Hose
- 4 PVC end bushings

Setting Up

Unpacking

⇒ Check the delivery for completeness (refer to the “Scope of Delivery” section).

Setup


If the device is to be installed in a cabinet, an opening for the exhaust air with the following dimensions must be provided, in addition to the openings for the suction hoses:

- Circular opening: Min. 120 mm diameters.
- Rectangular opening: Min. 170 x 65 mm.

Installing the Pinch Valves

- ⇒ Push the suction pipe (16) onto the suction support on the extractor and hang the holder (20) onto the holder screws.
- ⇒ Connect the compressed air tubes to the pinch valves:
 - Push the compressed air tube onto the pipe connection in the pinch valve as far as it will go. There will be a slight resistance to overcome when attaching.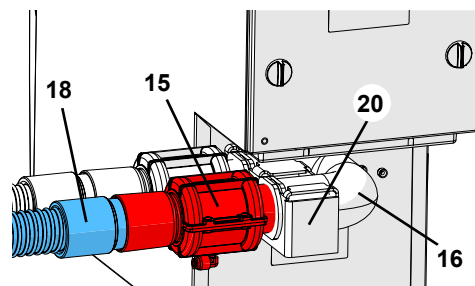

Connection

⇒ Connect to the compressed air supply system, using a connection kit if necessary (17, Fig. 1).

Observe the minimal / maximum connection pressure, see technical data!

Compressed air should be clean and dry (without condensed water). Wet compressed air can damage the unit.

⇒ Shorten the suction pipes for both extraction points to the required length with a knife.

⇒ Screw the PVC end bushings onto the suction pipes (left handed thread).

Caution: risk of injury!

When shortening the suction pipe please ensure to cut at a right angle.

⇒ Fit suction hose (18) in intake socket (15) and connect to suction point.

Long suction hoses, tight bends and kinks will considerably reduce the extraction force at the extraction point.

⇒ Electrical appliances can be connected to the coupler sockets (9, Fig. 1) at the back of the unit. Check the connections are correct - coupler socket (A, B) - suction hose / pinch valve (A, B).


When connecting electrical appliances to the extraction unit, please ensure that the power consumption of all devices connected to the unit power outlet does not exceed the total maximum power for connected equipment (see “tech. Data”).

⇒ Insert the mains cable into the plug socket.

Operation

Switching the Unit On

The extractor is put into operation at the on / off switch (3).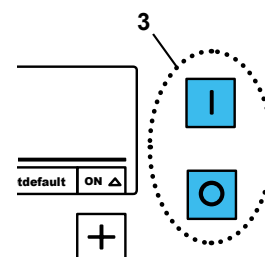

Stand-By

In the stand-by mode the display is dark.

Leave stand-by mode:


- Press any button
- Switch on any electrical appliance connected to the extractor.

Once switched on, the suction hose will open immediately and the extractor turbine will activate.

Select vacuum channel

Start / stop suction turbine and open / close corresponding pinch valve valve:


- Automatically:
The electrical appliance connected to the coupler socket (9, Fig. 1) is activated / deactivated.
- Manually:
By pressing a suction hose button (39).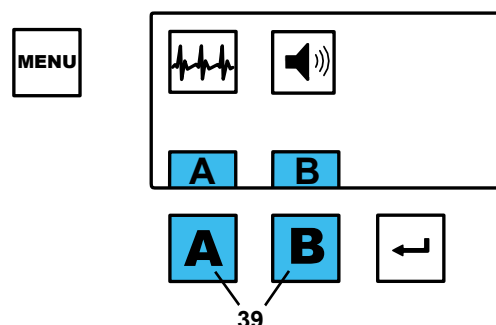

Adjust suction power

Set the extraction force:

- ⊕ Button (43), increases extraction force.
- ⊖ Button (41), reduces extraction force.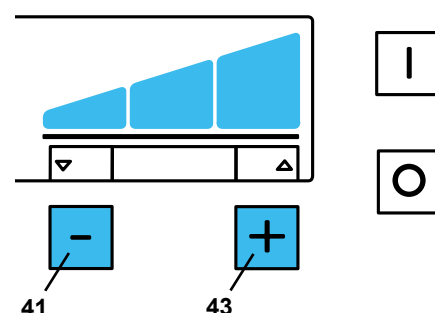

Programming

For the extractor programming instructions, please see enclosed Quick Reference Card and full set of operating instructions on the internet.

Cleaning / Maintenance

Wipe the product down with a slightly damp cloth.

Replacing the Dust Bag

It is essential to change the dust bag when indicated as follows on the product display:

The set dust bag fill level has been reached:

- ◆ The dust drawer symbol blinks.
- ◆ An audible signal will be emitted 3 x.
- ◆ The display blinks "Cod01".




Emptying the dust drawer:

- ⇒ Switch the extractor off.
- ⇒ Pull the front panel forward and off (5, Fig.1).
- ⇒ Pull the dust drawer (7) out towards the front.
- ⇒ Remove the protective film from the glue flap and use it to cover the dust bag opening!
- ⇒ Dispose of the dust bag properly.
- ⇒ Insert a new dust bag into the dust drawer. Ensure that the new dust bag is properly aligned in the guide grooves (21) and the glue flap faces upwards.

Use only original Renfert dust bags (see spare parts list).

- ⇒ Inspect the dust drawer seal for damage and replace it, as required (24).
- ⇒ Insert the dust drawer, making sure it is straight, and push it in as far as it will go. Make sure that the dust drawer is seated correctly on the guide (23).
- ⇒ Install the front panel on the bottom hooks then push it into the upper lock bolts, pressing firmly until it locks in place.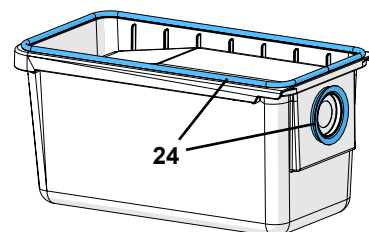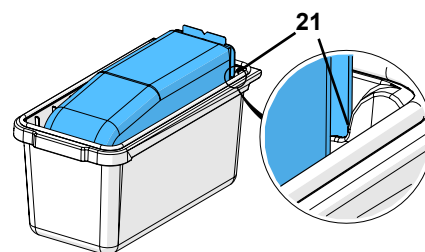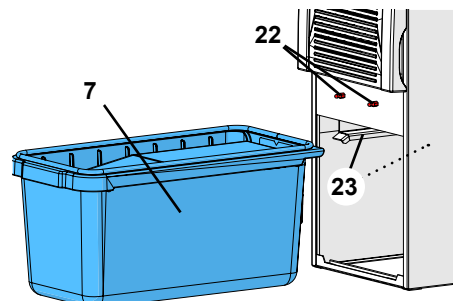

Errorcodes

If an error code is displayed during operation or in diagnostic mode, please consult the full set of operating instructions on the internet for the relevant information on the required course of action.

Spare parts

You can find components subject to wear and the spare parts on the spare part list in the internet at www.renfert.com.

The components excluded from the warranty (such as consumables or parts subject to wear and tear) are marked on the spare part list.

Disposal Information

Within the European Union, units with this symbol should not therefore be disposed of in unsorted domestic waste.

Please contact your local authorities for more information on proper disposal.

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Technical Data

Working Voltage	230 V	120 V
Permissible mains voltage:	220 - 240 V	120 V
Mains frequency:	50 / 60 Hz	60 Hz
Suction turbine power: *)	1400 W	1400 W
max. connecting power: *)	2000 W	360 W
Total connected power: *)	3400 W	1800 W
Mains input fuse:	2 x 15 A (T)	
max. connection pressure:	8 bar [116 psi]	
min. connection pressure:	4.5 bar [65 psi]	
LpA **) (at max. volume flow):	59 dB(A)	
Number of suction hoses:	2	
Volume flow, max per suction hose:	3300 l/min	
Vacuum pressure, max.:	200 hPa [2.9 psi]	
Fine particle filter:		
Filter surface area, approx.:	0.8 m ² [1240 sq inch]	
Filter quality:	Class M according to EN60335-2-69	
Fill volume, dust bag, approx.:	7.5 l [2 US gal]	
Dimensions (Width x Height x Depth):	224 x 591 x 584 mm [8.8 x 23.3 x 23 inch]	
Weight (empty), approx.:	27 kg [59.5 lbs]	
Ø Suction fittings: internal external	45 mm [1.77 inch] 50 mm [1.97 inch]	

*) Working voltage power

**) Sound pressure levels according to EN ISO 11202

**Hochaktuell und ausführlich auf ...
Up to date and in detail at ...
Actualisé et détaillé sous ...
Aggiornato e dettagliato su ...
La máxima actualidad y detalle en ...
Актуально и подробно на ...**

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c/o Renfert GmbH
Renfert GmbH • Industriegebiet • 78247 Hilzingen/Germany