FINOGEL PLUS

Dubliermasse

14002

Deutsch

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English

Instructions for use

**Please read carefully before use
Keep for referral purposes**

1. Fields of application

Agar-agar-based reversible duplicating gel for model casting technique. Also suitable for embedding of wax set-ups in resin cast technique. Can be reused.

2. Safety precautions

⚠ Be careful when handling heated gel, risk of scalding!

3. Processing instructions

3.1 Preparation

Place the duplicating material in a duplicating device and allow this to stir until the duplicating material flows freely and is free of lumps. Observe the instructions for use of the duplicating device's manufacturer.

⚠ Caution!
Do not exceed a temperature of 92 °C – risk of fire.
Water the model to be duplicated briefly with cold water and then fixate in a duplicating flask.

3.2 Duplicating of model

Allow the duplicating material in the duplicating device to cool to the processing temperature of 45-46 °C under constant stirring. Then pour the material quickly and evenly into the prepared duplicating form.

⚠ Gel temperature should not exceed 48 °C to avoid damaging the wax components.

Allow the duplicating material to cool for 45 mins. at room temperature, then carefully remove the model.

After demoulding the master model, immediately cast the negative form with plaster-free embedding material or PMMA resin. Due to their high water content, agar-agar-based materials are subjected to increased shrinkage, interruptions in processing can lead to incorrect fit.

3.3 Reuse of old material

Only use completely cleaned duplicating material cut into small sections.

Compensate any water loss through storage by adding water when melting again.

The quality, melting times and temperatures may vary after frequent repeated melting.

Replace the duplicating material after six reuses to avoid a loss in quality.

4. Storage

P402 Store in a dry place.

P234 Keep only in original container.

P233 Keep container tightly closed.

Protect against frost.

5. Physical data

Melting point 92 °C; casting temperature 45-46 °C; setting time approx. 45 mins. at room temperature.

6. Delivery forms

FINOGEL PLUS duplicating material, green
Bucket 6000 g 14002

7. Guarantee

Our technical recommendations of application are based on our own experiences and tests and should only be regarded as guidelines. It rests with the skills and experience of the user to verify that the products supplied by us are suitable for the intended procedures. Our products are undergoing a continuous further development. We reserve the right of changes in construction and composition. It is understood that we guarantee the impeccable quality of our products.

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