FINO Schiebelehre

59112

Deutsch

Gebrauchsinformation

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User Information

1. Fields of application

For the quick, precise and universal measurement of tooth width, tooth length, arch length and positions. Digital.

2. Safety precautions

No particular hazards worth mentioning.

3. Processing instructions

Before first operation check the item for transport damages. Claim any transport damages immediately with the supplier.

Check whether the slider on the guiding rail can be moved easily. Submit claims immediately in the event of faults. Warranty rights are invalidated if the claim is made at a later date.

3.1 Commissioning

Open the battery compartment located on the bottom right beside the digital display, and insert a battery by hand with the plus side facing upwards.

Press the red on/off switch. Close the tips of the calliper gently without applying pressure. If this position is not shown as the zero position on the display, press the yellow button (ZERO) to save the zero position.

Enclose the object to be measured using the measuring jaws of the calliper and read the measured value shown on the display. Switch between millimetres and inches with the blue button.

After use, switch off the calliper using the red on/off switch.

If the device is not used for long periods or is stored, remove the battery as standby current can cause it to be used up even if the display is switched off.

3.2 Changing the battery

When changing the battery (or when reinserting the battery), insert the battery by hand with the plus side facing upwards. The zero position must be set again, see 3.1 Commissioning.

4. Troubleshooting

1. Display flashes:

Battery voltage is too low. Change the battery.

2. No display:

a) Battery voltage is too low. Change the battery.

b) Battery does not have any contact. Remove the battery and bend the lower contact slightly upwards. Do not touch the contacts with metal!

3. Display blocked/does not continue to count:

Electrical circuit fault. Remove battery and reinsert after approx. 30 s.

4. Measurement error is too large (> 0.1 mm): Dirt on the scanning surface. Unscrew the cover and the circuit board and clean the scanning surface.

5. Service and maintenance

Keep the scale body clean. In order to prevent damage to the electronics, do not allow liquids to enter the display unit. Clean non-metal surfaces using the usual care products. Grease metal parts using a little machine oil. Do not use any acetone! Do not touch the contacts in the battery compartment or on the circuit board using metal.

6. Storage

Avoid any impact.

Store the device in a dry location.

Remove the battery if the device is not used for long periods.

7. Technical data

Measuring range 0-100 mm; accuracy of reading 0.01 mm; tolerance ± 0.02 mm; measuring speed 1.5 m/s; battery 1.5 V SR 44.

8. Delivery forms

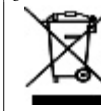
FINO caliper in a case complete with battery

1 pc. 59112

9. Guarantee

Our technical recommendations of application are based on our own experiences and tests and should only be regarded as guidelines. It rests with the skills and experience of the user to verify that the products supplied by us are suitable for the intended procedures. Our products are undergoing a continuous further development. We reserve the right of changes in construction and composition. It is understood that we guarantee the impeccable quality of our products.

10. Disposal in accordance with EU guideline 2002/96/EU

This equipment may not be disposed in domestic waste! Please contact the manufacturer or your dealer!

10.1 Registration Number

WEEE-Reg.-No. DE 75542610

Important:

FINO GmbH has prepared these instructions to help users operate the unit and to make it easier to take the unit into operation for the first time and then use it. FINO GmbH cannot be held liable for damages due to improper use or due to modifications made to the device itself. The warranty will be voided if the unit is used for any purpose other than its proper intended purpose. FINO GmbH has a policy of continuously developing and improving its products. FINO GmbH reserves the right to make changes or improvements to the products described in this document or to the document itself without prior notice. FINO GmbH offers no guarantees for the completeness or correctness of this document.

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