PERMABOND

Gebrauchsinformation

Stand: 31.05.17

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- Use of the etch gel is necessary on both the enamel as well as the dentine surfaces. The material may not be applied without dentine etching.
- The oxygen inhibition layer which is generated during application provides bonding to the above applied materials and may therefore not be removed.
- Contamination of the contents of the bottle through contaminated disposable brushes and premature polymerisation through ambient light should be avoided.
- A matrix should, if possible, only be used after application of the bond to avoid the accumulation of excess material between tooth and matrix (pooling effect).
- Do not use chemical and dual-curing composites with M+W Permabond.
- Avoid skin contact of the liquid. Wash with water and soap in case of inadvertent contact with skin. In case of contact with eyes, immediately rinse with copious amounts of water and consult a doctor.

Interactions

M+W Permabond may not be used in conjunction with materials containing eugenol as these can inhibit polymerisation.

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Contraindications

The application of M+W Permabond is contraindicated if it cannot be assured that the work area can be kept dry during application, if the pulp is exposed, or in case of an existing allergy to one of the material's ingredients.

Side effects

No side effects are known to-date. Hypersensitivity or contact allergy to products with a similar composition have been described for individual cases.

Composition

Bis-GMA-based resin matrix, water, additives, catalysts.

Storage

Store dry at 2- 25°C. Do not use after the expiry date.

Keep out of reach of children! For dental use only!

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GB Instructions for use

Intended purpose

M+W Permabond is a light-curing, single-component bonding agent for dentine and enamel for application with light-curing composites. Etching of dentine and enamel surfaces is required prior to the application of M+W Permabond.

Indication

Light-curing bonding agent for enamel and dentine in conjunction with the total etch technique for light-curing composites.

Recommended application

Note: when using M+W Permabond, it is essential for both the enamel as well as dentine surfaces to be etched beforehand!

1. Prepare cavity according to the rules of general adhesive technique. Bevel the enamel edges and protect the areas near the pulp in deep cavities with a suitable pulp lining (for example, calcium hydroxide liner). The use of a rubber dam is recommended.

2. Etch dentine and enamel with 37 % phosphoric acid gel (for example, M+W etch gel). Please observe the respective manufacturers' instructions for the etch gel. The exposure time of the etch gel on dentine must not exceed 15 seconds under any cir-

cumstances. After the etching process, the enamel should appear chalky white.

3. Rinse the etched surface for at least 15 s with water and then dry the surface with oil and water-free air. Ensure that the dentine is not dried excessively.

4. Apply M+W Permabond to the etched surface and process for 20 seconds. Blast briefly with oil and water-free air for 5 seconds. The treated surface must display a uniformly glossy layer. If this is not the case, add a little material in addition.

5. Light-cure M+W Permabond with a suitable lamp unit for 10 seconds. A further layer of M+W Permabond is not required!

6. Apply the restoration material according to the respective manufacturers' instructions.

Please note

- Lamp units should have a light intensity of 400 mW/cm², emit at 450 nm and have their performance checked regularly. The light should be positioned as close as possible to the material.

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Müller & Weygandt GmbH • Reichardsweide 40 • D-63654 Büdingen • Tel.: +49 (0)6042 – 88 00 88