Indications:

Celtra Universal Low Fusing Stains and Overglaze are indicated for staining and glazing Celtra Duo restorations. They also can be used for staining and glazing lithium silicate based restorations, and for restorations made from Ceramco® II, Ceramco® II Silver™, Ceramco® 3, Ceramco® IC porcelain and Cercon® zirconia.

Contraindications:

Only the indications listed above are suitable.

Warnings/Adverse Reactions:

Consult Celtra Stains and Overglaze Safety Data Sheets (SDS) for safe handling and usage. Use only according to the indications listed above.

Precautions:

Refer to the special notes throughout the instructions below for best practices for using Celtra Low Fusing Stains and Overglaze.

Instructions:

1. Before the Stains and Overglaze application, the restoration must be free of dirt and grease. Use a steam cleaner to clean the surface of the restoration or put the restoration in distilled water and place in an ultrasonic cleaner for 10 minutes. Any contamination after cleaning must be prevented. (See Figure 1).

Figure 1
Clean crown is ready for staining.

2. Place the clean crown on the prepared DENTSPLY Prosthetics All-Ceramic Die Material stump (See Figure 2). To ensure the accurate shade representation is accomplished, prepare the stump, using the stump shade recommendation provided in Table 1 below.

Figure 2
Crown on a stump ready for staining.

Table 1.
Restoration Shade vs. Recommended Die Material for Stump

Shade	A1	A2	A3	A3.5	A4	B1, B2
Die Material	F1	F12	F10	F9	F7	F11

Shade	B3, B4	C1, C4	C2	C3	C4	D2, D3
Die Material	F8	F3	F4	F5	F6	F2

3. The following steps must be observed:

- If a more intensive shade effect is desired, it is achieved by several staining applications and firings. The application of too many stain layers results in an unnatural shade effect.
- The cusps and fissures can be individualized using stains.
- The basic chromatic shade is supported with the corresponding Shade material (see Table 2 and Table 3).

Table 2. Shade Stain Recommendations

Desired Shades	Stain
Bleached	0
A1, A2, A3, A3.5	1
B1, B2, B3, B4, D4	2
C1, C2, C3, C4	3
A4, D2, D3	4

Table 3. Incisal Stain Recommendations

Stain	Shade to Apply
Incisal Stain i1	A1, A2, A3, B1, B2, B3, B4, Bleached
Incisal Stain i2	A3.5, A4, C1, C2, C3, C4, D2, D3, D4

4. If the Overglaze inside the jar has separated, mix thoroughly with spatula. Remove desired amount of Celtra Overglaze paste from jar to palette. If a different consistency is desired, adjust the consistency by diluting the material with Stain and Glaze Liquid. (See Figure 3)

Figure 3
Overglaze consistency after mixing with Stain and Glaze Liquid.

5. Apply a thin layer of the Overglaze material all over the crown surface in the usual manner using a brush. Make sure not to apply the Overglaze material either in too thick or too thin layers. Too thin of a layer will result in a more matte finish. Too thick of a layer can result in puddling and/or pitting of the material. (See Figure 4)

Figure 4
Application of the Overglaze.

6. If the stain inside the jar has separated, mix thoroughly with spatula. Remove the desired quantity of Celtra Shade Stain and Celtra Incisal Stain and place on palette. Dilute and mix with Stain and Glaze Liquid to the desired consistency. The material must not be applied too thickly to avoid pooling.

7. Paint a thin layer of the Shade Stain on the gingival and thin towards to the incisal edge or wherever the desired effect is needed. Check the shade using the shade guide. Use your brush to increase or decrease the amount of stain. (See Figure 5)

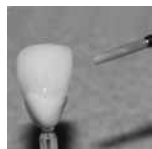
Figure 5
Application of Shade Stain.

8. Paint a thin layer of the Incisal Stain on the incisal edge. (See Figure 6)

Figure 6
Application of incisal stain.

9. Keep the crown on a stump and check the shade using a shade guide. Use your brush to increase or decrease the amount of stain. (See Figure 7)

Figure 7
Before firing, keep the crown on a stump to check shade match of restoration.

10. If a more intensive shade effect is desired, it is achieved by several staining applications and firings. However, the application of too many stain layers results in an unnatural shade effect.
11. Fill the restoration with refractory putty (such as Super Pegfil™ Refractory Material - Reorder #215812). Place the restoration on a DENTSPLY Prosthetics All-Ceramic firing pin. To avoid crown cracking, be sure to not let any part of the crown come into contact with the metal pin. Then, position the restoration on top of a honeycomb sagger tray and conduct Stain and Overglaze Firing according to the Firing Schedule. (See Figure 8)

Figure 8
Fill restoration with refractory putty, and place on a metal pin. Fire on a sagger tray.

Firing Schedule:

Stain and Overglaze	Firing Program in °C	Firing Program in °F
Dry closing time (min)	3	3
Pre Heat (closed min)	3	3
Low Temp	500	932
Heat Rate	55°C/min	99°F/min
High Temp	820	1508
Vac Start Temp	0	0
Vac Stop Temp	0	0
Hold Time Vac	0	0
Hold Time Air	1.5 min	1.5 min
Cool Time	0	0

When working with a furnace from another manufacturer, firing parameters may have to be adjusted accordingly.

Finally, the shade of the completed restoration is checked on the stump. (See Figure 9)

Figure 9
Check final shade match of the restoration using the stump.

Note: After the first firing the restoration may come out underglazed. If that's the case, apply a thin layer of the Celtra Overglaze material all over the crown surface in the usual manner using a brush. Once the Overglaze is applied, conduct the second Stain and Overglaze Firing, according to the Firing Schedule.

Additional Stain/Overglaze and Characterization firing cycles can be conducted with the same firing parameters.

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