# **MTplus**

**No.1803-x000 /-x500**

***Bedienungsanleitung***  
***Instruction manual • Mode d'emploi***  
***Istruzioni d'uso • Instrucciones para el servicio***  
***Instrukcja obsługi • Návod k obsluze***  
***Használati utasítás • Инструкция по эксплуатации***  
***Instruções de operação***  
**操作说明书 • 取扱説明書 • 사용 설명서**

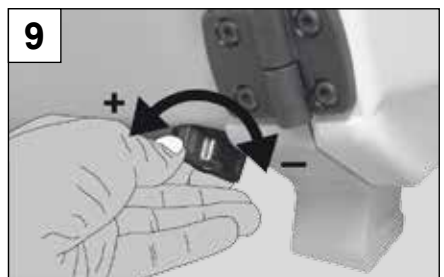
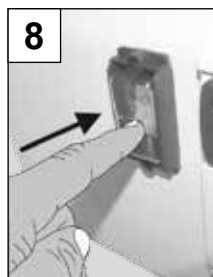
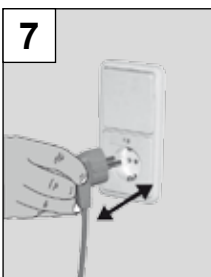
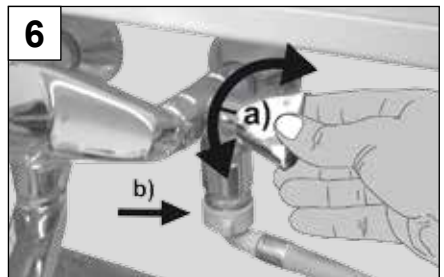
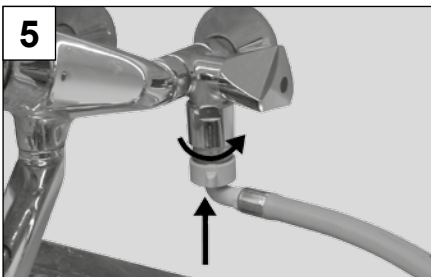
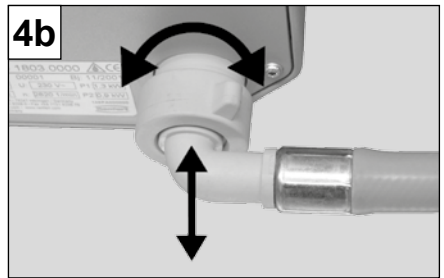
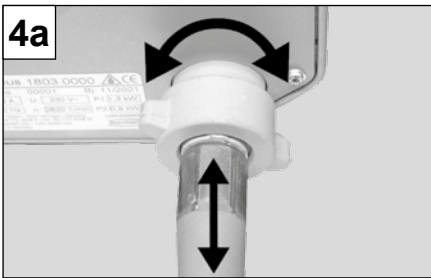
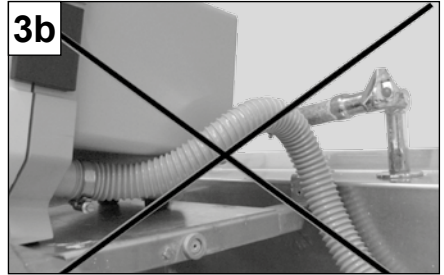
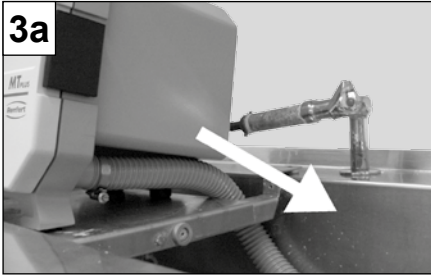
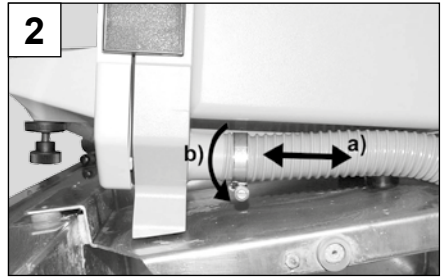
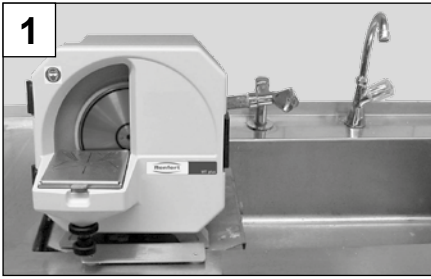
21-9873 13032018

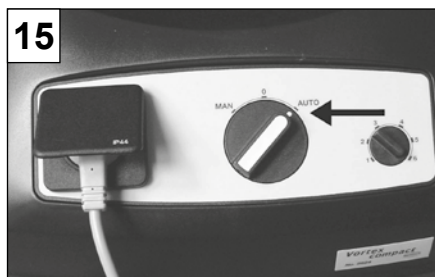
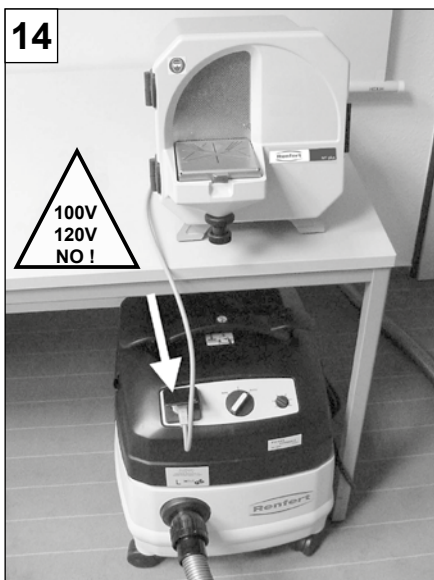
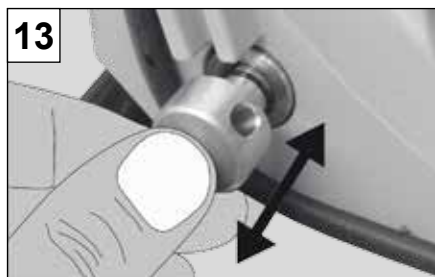
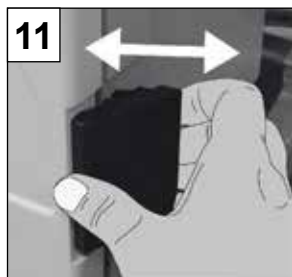
Made in Germany

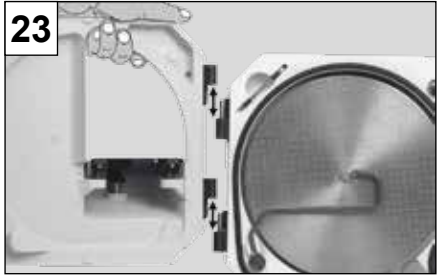
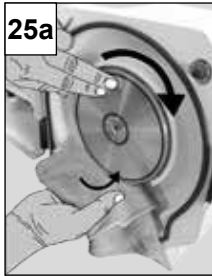
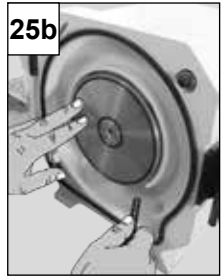
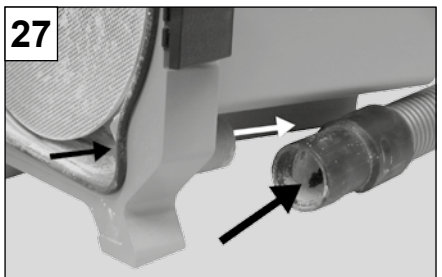
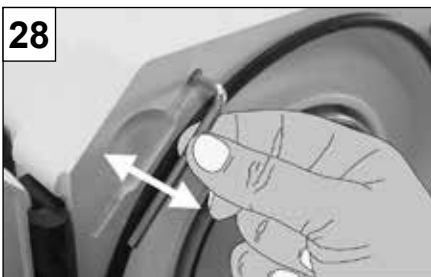
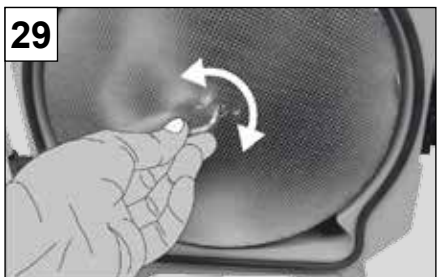
**Renfert**

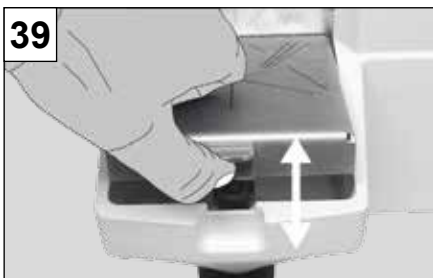
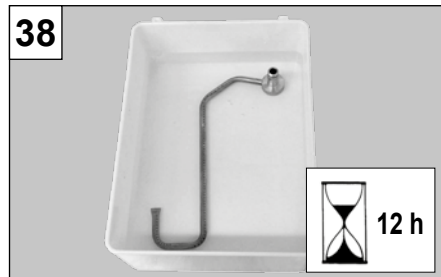
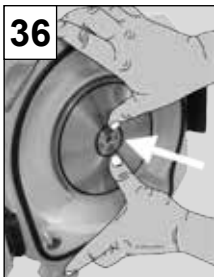
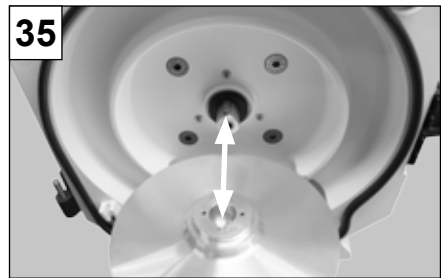
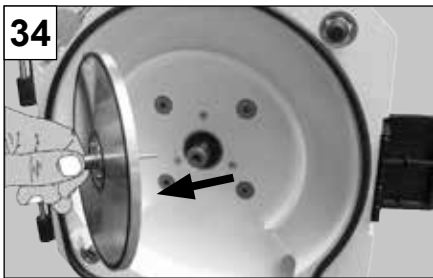
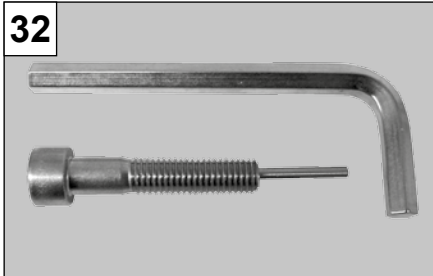
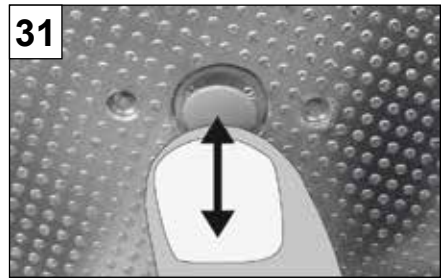
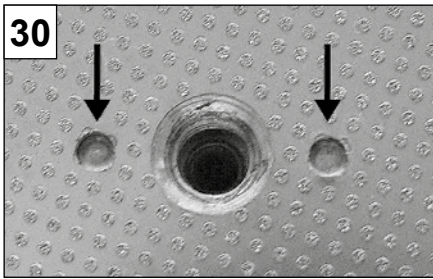
Ideas for dental technology

**19****20****21****22****23****24****25a****25b****26****27****28****29**

# MTplus Gipstrimmer

No.1803-x000 /-x500

DEUTSCH

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## 1. Einleitung

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ENGLISH

## 1. Introduction

We are pleased with your decision to purchase an *MTplus*. The trimmer sets new standards with regard to function, performance, safety, and design.

Please note the following information.

## 2. Application Area

The *MTplus* is a model trimmer designed for use in dental technology operations to grind plaster models. It can be employed as either a wet or dry trimmer.

### 2.1 Ambient Conditions (in accordance with DIN EN 61010-1)

The *MTplus* may only be operated:

- Indoors;
- Up to an altitude of 2,000 m [6,500 ft] above sea level;
- At an ambient temperature of 5 - 40 °C [41 - 104 °F] \*;
- At a maximum relative humidity of 80 % at 31 °C [87.8 °F], decreasing linearly to 50 % relative humidity at 40 °C [104 °F] \*;
- With a mains power supply where current fluctuations do not exceed 10 % of the nominal value;
- Under level 2 contamination conditions;
- Under overvoltage category II conditions.

\*) Between 5 – 30 °C [41 - 86 °F], the unit can be operated at a relative humidity of up to 80 %. At temperatures between 31 - 40 °C [87.8 - 104 °F], the humidity must decrease proportionally in order to ensure operational readiness (e.g., at 35 °C [95 °F] = 65 % humidity; at 40 °C [104 °F] = 50 % humidity). The unit may not be operated at temperatures above 40 °C [104 °F].

## 3. Safety Information

Please observe all applicable accident prevention regulations together with the following safety notes when trimming:

*MTplus* is an electrical appliance with a danger potential. This appliance may be connected / used only by authorized and qualified persons after checking its conformity with respect to the national norms.

Never reach into the grinding wheel while the unit is connected to the power supply. Inadvertently starting the unit can result in injury!

Always keep long hair, loose or wide sleeves or other clothing away from the trimmer. Hair or clothing can be pulled into and wrapped up on the grinding wheel, resulting in injury!

The unit is designed solely for grinding plaster models. Cancer causing dusts are released when trimming investment material.

Never open the front cover while the motor is running. The motor safety shutoff **will not** stop the grinding wheel immediately.

Only open the front cover after the grinding wheel has come to a complete stop.

Always wear protective eyewear when grinding.

Do not reach into the grinding wheel while it is turning.

Do not work without the model table installed and locked („click“). Small parts or fingers can be drawn into the trimmer, resulting in serious injury.

Remove the trimmer table only after the grinding wheel has come to a complete stop and the front cover is opened.

Operate the equipment only under supervision

It is the responsibility of the operator that national regulations during operation and regarding a repeated safety inspection of electrical equipment are complied with. For Germany these are the regulation 3 by DGUV (German Statutory Accident Insurance) in relation with VDE (Association for Electrical & Electronic Technology) 0701-0702.

#### Additional notes for dry trimming:

The extraction must be selected to match the dust being generated (if necessary, consult the responsible professional organization).

Operating the unit can result in a very high noise level >>> always wear ear protection.

### **3.1 Liability Exclusion**

Renfert GmbH shall be absolved from all claims for damages or warranty if:

- The product is employed for any purposes other than those cited in the operating instructions;
- The product is altered in any way other than those alterations described in the operating instructions;
- The product is repaired by other than an authorized facility or if any but Renfert OEM parts are employed;
- The product continues to be employed, despite obvious safety faults.

## **4. Commissioning / Operation**

First, check to make sure that your mains power corresponds with the power requirements given on the trimmer nameplate.

The type of usage wet- (see pt. 4.1 and 4.2) or dry-trimming (see pt. 4.1 and 4.3) must be determined prior to use.

### **4.1 Mounting the Trimmer Disc**

Please refer to the trimmer disc instructions (resp. to pt. 5.1 in this instructions) for the mounting procedure.

Range of application of the discs:

- *Kletifix* for wet trimming
- *Marathon* for wet or dry trimming
- *Infinity* for wet trimming

### **4.2 Wet Trimming**

1. Set the unit up near the water source / drain, with a plaster trap (Fig. 1).
2. Connect the drain hose to the trimmer (Fig. 2). **Turn the drain hose on the connecting branch counter clockwise and fix with a hose clamp (Fig. 2b)!**
3. Position the trimmer hose in the drain (Fig. 3a / 3b).
4. Connect the water hose to the trimmer (Fig. 4a / 4b). **Observe the necessary water pressure (see pt. 8 / technical specifications)!**
5. Connect the water hose to a water supply with a shutoff tap (Fig. 5).
6. Open the water supply (Fig. 6a) and check for leaks (Fig. 6b).
7. Plug the power cord into the wall outlet (Fig. 7).
8. Switch the unit on (Fig. 8).
9. Adjust the water flowrate (Fig. 9).  
Attention: the water inlet is closed on delivery.

The *MTplus* is now ready for use as a **wet trimmer**.

### General information concerning wet trimming:

- An adequate water flow is important, particularly when trimming with diamond-coated discs. Overheating will damage the disc.
- The water supply shutoff tap should be turned off during longer work interruptions or overnight.

## 4.3 Dry Trimming

- 1a. Connect the trimmer to an **external extractor** (Fig. 10 illustrates an example using a Renfert extraction hose and coupling).

**Attention!**  
**100 V / 120 V units in combination with dust extractors should be separately connected to the mains in order to avoid an overcharging of the extractor and of the electric network (do not use the automatic start-up!). If the electric network is protected by fuse of only 15 A, it is necessary to connect the trimmer and the extractor to separately protected electric circuits.**

### Recommended dust extractors:

see: Accessories, Extraction units etc.

- 1b. Connect the trimmer to the **central extractor** (Fig. 10 illustrates an example using a Renfert extraction hose and coupling).

### Recommended accessories:

see: Accessories, Extraction units etc.

2. Open the front cover (Fig. 11).
  3. Remove the rinse tube (Fig. 12).
  4. Close the opening with the blind plug (**completely impress it**) from the hardware set (Fig. 13).
  5. Close the front cover (Fig. 11).
- 6a. For extraction **with** automatic start-up:
- Connect the trimmer power cord to the extractor (Fig. 14).
  - Set the extractor up for automatic operation (Fig. 15).

- 6b. For extraction **without** automatic start-up:

- Plug the trimmer power cord into the wall outlet (Fig. 7).
- Switch on the *MTplus* (Fig. 8), switch the extractor on automatic use.

The *MTplus* is now ready for use as a **dry trimmer**.

## 4.4 Conversion from Wet to Dry Operation

1. Disconnect the unit from the power supply (Fig. 7).
2. Shut off the water supply (Fig. 6a).
3. Remove the water hose (Fig. 4a / 4b).
4. Remove the drain hose (Fig. 2).
5. If necessary, remove the unit from the wet area **allow to dry completely. All remaining moisture must be blown off with compressed air!**
6. Remove the trimmer disc (see pt. 5.1, steps 4 - 7).
7. Clean the area behind the flange first with a wet and then with a dry cloth (Fig. 25a). Moreover blow off with compressed air (Fig. 25b).
8. Once the unit is dry, proceed as described in Section 4.3.

**Partially diamond-coated** trimmer discs should be used for dry operation.

### Recommended trimming disc:

see: Accessories, Trimming discs - for wet and dry trimming

## 4.5 Conversion from Dry to Wet Operation

1. Disconnect the unit from the power supply (Fig. 7 or 14).
2. Remove the extractor hose (Fig. 10).
3. Remove the trimmer disc (see pt. 5.1, steps 4 - 7).
4. Using a brush, thoroughly clean the front cover and the inside of the housing with an extractor and a brush (Fig. 26).

5. Remove the blind plug. Should it be difficult, you can use the Allen key (Fig. 16).
6. Mount the trimmer disc (see instructions for trimmer discs).
7. Install the rinse tube (Fig. 12).
8. Then, proceed as described in Section 4.2.

#### 4.6 Recommendation for dry or wet trimming

- Dry trimming is not allowed when the *MTplus* is equipped for wet trimming.
- Wet trimming is not allowed when the *MTplus* is equipped for dry trimming.
- Not observing the prior instructions will result in plaster residue mixing with water and hardening to a difficult to remove crust. Such residues particularly on the back of the trimmer disc can lead to imbalance.

#### 4.7 Grinding Angle Adjustment (89° - 99°)

1. Loosen the knurled nut (Fig. 17).
- 2a. Adjust the grinding angle roughly with the adjusting screw (Fig. 18).  
If the adjusting screw is turned to the **top**, the angle between model table and trimmer disc is of 89°. If the adjusting screw is completely turned **out**, the angle between model table and trimmer disc is of 99°.
- 2b. Adjustment of the model table roughly with the adjusting screw (Fig. 18). The lower border of the model table should coincide with the upper border of the model-table support (Fig. 19 + 20).
- 2c. Fine adjustment of the model table at 90° with the adjusting screw (Fig. 18) and set square out of the hardware set (Fig. 21).
3. Retighten the knurled nut (Fig. 17).

#### 4.8 Operation / Trimming

Using both hands, securely set the model on the trimmer table and press it against the disc (Fig. 22).

#### 5. Cleaning / Maintenance

**A raising of plaster dust in aerodynamic dead areas when dry trimming is normal and has no negative influence on the function or grinding power of the *MTplus*.**

1. Disconnect the unit from the power supply (Fig. 7 or 14).
2. If necessary, shut off the water supply (Fig. 6).
3. Open and unhook the front cover (Fig. 11 + 23).
- 4a. **For wet trimmer:** If necessary, remove the table (Fig. 39 + 40) and clean with a brush under running water (Fig. 24).

**Do not remove the grinding table if the front cover is closed otherwise the trimmer disc will be damaged!**

- 4b. **For dry trimmer:** Clean using a vacuum and brush (Fig. 26). Clean the drain tube and tube connection (Fig. 27).
5. Reinstall and close the front cover (Fig. 23 + 11).

#### 5.1 Trimmer Disc Replacement

1. Switch the unit off (Fig. 8).
2. Disconnect the unit from the power supply (Fig. 7 or 14).
3. If necessary, shut off the water supply (Fig. 6a).
4. Open the front cover (Fig. 11).
5. If necessary, remove the rinse tube (Fig. 12).
6. Remove the SW4 Allen key from its holder (Fig. 28).
7. Remove the thread cover, loosen the centring screw, and remove the disc (Fig. 31 + 29).

8. If necessary, clean the flange (refer to Section 5.2).
9. Place a new disc on the lock pin (Fig. 30).
10. Retighten the centring screw with the Allen key (Fig. 29).
11. Place the thread cover back on the centring screw (Fig. 31).
12. Return the Allen key to its holder (Fig. 28).
13. If necessary, replace the rinse tube (Fig. 12).
14. Close the front cover (Fig. 11).
15. If necessary, open the water supply (Fig. 6a).
16. Plug the power cord into the wall outlet (Fig. 7 or 14).

## 5.2 Flange Replacement / Cleaning

1. Remove the disc (refer to Section 5.1, steps 1 - 7).
2. Have the pulling screw and Allen key at hand >>> see tool kit enclosed (Fig. 32).
3. Screw the pulling screw into the flange (Fig. 33) >>> **The flange may not be removed by hand. Always use the pulling screw to remove the flange!**
4. Remove the flange (Fig. 34).
5. Remove the pulling screw from the flange.
6. Clean the flange and the inside of the housing  
>>> **Do not damage the seals!**  
Recommended for cleaning:  
see: Accessories
7. Lightly coat the motor shaft and seal with Vaseline.
8. Replace the flange  
>>> **Note the engagement groove** (Fig. 35).
9. Replace the flange by hand on the shaft (Fig. 36).
10. Reinstall the grinding disc (refer to Section 5.1, steps 9 - 16).

## 5.3 Spray Tube Replacement / Cleaning (Wet Trimmer)

1. Remove the rinse tube (refer to Section 5.1, steps 1 - 5).
2. Remove the gaskets (Fig. 37).
3. Allow the rinse tube to soak in decalcifying agent overnight (Fig. 38).
4. Reinstall the gaskets and coat them lightly with Vaseline (Fig. 37).
5. Replace the rinse tube (Fig. 12).
6. Close the front cover (Fig. 11).
7. If necessary, open the water supply (Fig. 6a).
8. Plug the power cord into the wall outlet (Fig. 7 or 14).

## 5.4 Grinding Table Replacement / Cleaning

1. Switch the unit off (Fig. 8).
  2. Disconnect the unit from the power supply (Fig. 7 or 14).
  3. Open the front cover (Fig. 11).
- !** **Do not remove the grinding table if the front cover is closed otherwise the trimmer disc will be damaged!**
4. Unlatch and remove the grinding table (Fig. 39 + 40).
  5. Remove the front cover (Fig. 23).
  6. Clean the grinding table and front cover with a brush under running water (Fig. 24).
  7. Replace and close the front cover (Fig. 23 + 11).
  8. Reinstall the grinding table (Fig. 40).
- i** **Make sure the retainers are completely free of all plaster residue! Make sure the work table is properly seated on the front cover!**
9. Plug the power cord into the wall outlet (Fig. 7 or 14).

## 5.5 Switch Cover Replacement

1. Loosen the fastening screws (Fig. 41).
2. Remove the switch cover and replace it with a new one (Art. No. 90003-5788).
3. Retighten the fastening screws.

**The *MTplus* may not be operated with a defective switch cover!**

## 6. Spare Parts

You can find components subject to wear and the spare parts on the spare part list in the internet at [www.renfert.com/p918](http://www.renfert.com/p918).

The components excluded from the warranty (such as consumables or parts subject to wear and tear) are marked on the spare part list.

Serial number and date of manufacturing are shown on the type plate of the unit.

## 7. Warranty

Provided the unit is properly used, Renfert **warrants** all parts of the *MTplus* for a **period of 3 years**. In case of any claims for warranty original dealers' invoice is required. Components subject to natural wear (e.g., trimmer discs) are excluded from this warranty.

The warranty is voided in case of improper use, failure to observe the operating, cleaning, maintenance, and connection instructions, in case of independent repairs or repairs by unauthorized personnel, if spare parts from other manufacturers are employed, or in case of unusual influences or influences not in compliance with the utilization instructions. Warranty service shall not extend the original warranty.

## 8. Technical Specifications

Mains voltage: 220 - 230 V, 50 / 60 Hz  
230 - 240 V, 50 Hz  
120 V, 60 Hz  
100 V, 50 / 60 Hz

Power consumption:  
220 - 230 V: 1300 W (1,743 hp)  
230 - 240 V: 1300 W (1,743 hp)  
120 V: 1500 W (2,012 hp)  
100 V: 1200 W (1,609 hp)

Water consumption,  
max.: 9 l/min. [0.32 cfm]

Necessary water pressure:  
- min.: 1 bar [14.5 psi]  
- max.: 5 bar [72.5 psi]

Speed: 50 Hz: 2850 1/min.  
60 Hz: 3400 1/min.

Connection piece  
diameters for external  
extraction: 36 mm [1.42 inch]

Dimensions  
(W x H x D): 300 x 320 x 410 mm  
[11.8 x 12.6 x 16.2 inch]

Ø Trimmer disc  
dimensions: 234 mm [9.2 inch]

Weight: 13.5 kg [30 lbs]

Sound pressure level  
accord. to DIN 45635-01-KL3,  
measured under full load:  
- Lp(A) : 74 dB(A)  
- K: 4 dB

## 9. Standard Delivery

- 1 *MTplus*
- 1 Inlet tube
- 1 Drain tube
- 1 Tool set
- 1 Hardware set
- 1 *Marathon* trimmer disc, only on No. 1803-x500

## 10. Delivery Versions

- No. 1803-0000 *MTplus*, 230 V / 50 Hz  
No. 1803-0500 *MTplus*, 230 V / 50 Hz,  
incl. *Marathon* trimmer  
disc  
No. 1803-4000 *MTplus*, 120 V / 60 Hz  
No. 1803-4500 *MTplus*, 120 V / 60 Hz,  
incl. *Marathon* trimmer  
disc  
No. 1803-6000 *MTplus* 230 - 240 V /  
50 Hz, AU/NZ  
No. 1803-6500 *MTplus* 230 - 240 V /  
50 Hz, AU/NZ  
incl. *Marathon* trimmer  
disc

## 11. Disposal instructions for countries in the EU

To conserve and protect the environment, prevent environmental pollution and improve the recycling of raw materials, the European Commission adopted a directive that requires the manufacturer to accept the return of electrical and electronic units for proper disposal or recycling. Within the European Union units with this symbol should not therefore be disposed of in unsorted domestic waste:

For more information regarding proper disposal please apply at your local authorities.

## 12. Accessories

- No. 2011-0000 GO-2011, plaster solvent  
**Trimming discs - for wet trimming only:**  
No. 1803-1000 Basic kit:  
5 *Klettfix* silicon carbide  
trimmer discs (3 x corn  
80, 2 x corn 120)  
+ support disc  
No. 1803-1100 Refill kit:  
5 *Klettfix* silicon carbide  
trimmer discs, corn 80  
No. 1803-1200 Refill kit:  
5 *Klettfix* silicon carbide  
trimmer discs, corn 120  
No. 1803-3000 1 *Infinity* trimmer disc,  
diamond-coated  
**Trimming discs - for wet and dry-  
trimming:**  
No. 1803-2000 1 *Marathon* trimmer disc,  
partially diamond-coated  
Extraction units etc.:  
**Silent TS:**  
No. 2921-0050 Silent TS, 230 V,  
50 / 60 Hz  
No. 2921-1050 Silent TS, 100 - 120 V,  
50 / 60 Hz  
No. 2921-0002 Dust bag set (5 bags)  
No. 2921-0003 Muffler set, 2 mufflers  
No. 15-0823 Suction hose (by the  
metre)  
No. 900034305 Hose fitting adapter  
No. 933334240 Suction hose  
No. 900034314 Y-junction  
**Vortex**  
No. 2924-5000 *Vortex compact 3L*, 230 V  
No. 2924-6000 *Vortex compact 3L*, 120 V  
No. 2924-5003 5 filter bags  
No. 2924-5005 1 cloth filter  
No. 92923-0001 3 m suction hose with  
elastic coupling  
No. 92923-0010 1 elastic coupling only,  
Ø 36 mm  
No. 2-0973 1 90° plastic elbow for  
drain tube

## 13. Troubleshooting Guide

Fault	Possible cause	Corrective action
<b>Device fails to start.</b>	<ul style="list-style-type: none"> <li>• Power supply not established.</li> <li>• Mains fuse faulty.</li> <li>• Front cover not properly closed.</li> <li>• Motor overheated.</li> </ul>	<ul style="list-style-type: none"> <li>• Check the mains power supply.</li> <li>• Check the mains fuse.</li> <li>• Close the front cover (Fig. 11).</li> <li>• Allow the motor to cool down, then restart at the main power switch.</li> </ul>
<b>Trimmer disc wobbles.</b>	<ul style="list-style-type: none"> <li>• Disc is not lying flush on the flange.</li> <li>• Disc not tightened sufficiently.</li> </ul>	<ul style="list-style-type: none"> <li>• Reinstall the disc according to the directions. Pay attention that the disc is mounted exactly on the lock pin (Fig. 30).</li> <li>• Remove all deposits from the back of the disc and from the stopping face of the flange.</li> <li>• Hand-tighten the disc (Fig. 29).</li> </ul>
<b>Excessive noise while the trimmer disc is running.</b>	<ul style="list-style-type: none"> <li>• Disc not adequately tightened.</li> </ul>	<ul style="list-style-type: none"> <li>• Hand-tighten the disc (Fig. 29), or reinstall, as required (Sec. 5.1).</li> </ul>
<b>Trimmer table wobbling.</b>	<ul style="list-style-type: none"> <li>• Trimmer table incorrectly installed.</li> </ul>	<ul style="list-style-type: none"> <li>• Reinstall the table according to the directions (Fig. 40).</li> </ul>

## Wet Trimming:

Fault	Possible cause	Corrective action
<b>Disc buildup.</b>	<ul style="list-style-type: none"> <li>• Water volume incorrectly adjusted at the device.</li> <li>• Water supply not fully open.</li> <li>• Spray tube plugged.</li> <li>• Inlet strainer plugged.</li> <li>• Inlet water pressure too low.</li> </ul>	<ul style="list-style-type: none"> <li>• Increase the water volume at the trimmer (Fig. 9).</li> <li>• Open the water supply (Fig. 6).</li> <li>• Clean / decalcify the spray tube (Sec. 5.3).</li> <li>• Remove the strainer with pliers and clean / decalcify it.</li> <li>• The diameter of the inlet pipe is too small resp. too many users are connected to a small pipe &gt;&gt;&gt; Check if water pressure is at least 1 bar (ask your plumber). If necessary the water pressure can artificially increased by a domestic water work (electric pump and pressure tank).</li> <li>• Ask your plumber.</li> </ul>
<b>Water sprays out of the grinding opening.</b>	<ul style="list-style-type: none"> <li>• Too much water.</li> <li>• Water drainage impeded.</li> </ul>	<ul style="list-style-type: none"> <li>• Reduce the water volume at the trimmer (Fig. 9).</li> <li>• Check/properly lay out drain tube &gt;&gt;&gt; slope (Fig. 3a + 3b).</li> </ul>
<b>No rinse water.</b>	<ul style="list-style-type: none"> <li>• Water connection not properly established.</li> <li>• Spray tube soiled / calcified.</li> </ul>	<ul style="list-style-type: none"> <li>• Check the water connection (Sec. 4.2 steps 4 - 6).</li> <li>• Clean/decalcify the spray tube (Sec. 5.3).</li> </ul>
<b>Water is dripping out between the front door and the housing.</b>	<ul style="list-style-type: none"> <li>• Gypsum powder order plaster grains on the sealing surface.</li> </ul>	<ul style="list-style-type: none"> <li>• Clean the sealing surface and the sealing ring.</li> </ul>

### **Dry Trimming:**

<b>Fault</b>	<b>Possible cause</b>	<b>Corrective action</b>
<b>Disc buildup.</b>	<ul style="list-style-type: none"><li>• Models are too wet.</li></ul>	<ul style="list-style-type: none"><li>• Clean the disc with a brush under running water or with plaster solvent (Renfert GO-2011 Art. No. 2011-0000).</li></ul>
<b>Excessive dust being blown out of the grinding opening.</b>	<ul style="list-style-type: none"><li>• Extraction not switched on.</li><li>• Insufficient suction.</li><li>• Extractor or dust bag full.</li><li>• Extractor hose blocked.</li></ul>	<ul style="list-style-type: none"><li>• Switch the extraction on.</li><li>• Increase the suction.</li><li>• Replace the dust bag.</li><li>• Check/clean the hose.</li></ul>

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c/o Renfert GmbH
Renfert GmbH • Industriegebiet • 78247 Hilzingen/Germany