& ACCESSORIES

50 pcs

50 pcs

COM	PONENT	S & ACCE	SSOF
PRECI-SAGIX MALE P Castable spherical plastic male. For lab use only. Cast in a hard dental alloy. Mini size Ø 1.7:			
# 1321	6 pcs size ø 2.2:	# 1321B	50 pcs
# 1371	6 pcs	# 1371B	50 pcs

PRECI-SAGIX MALE M

NOPRAX** male for cast-on technique with non-precious alloys. Contains Ni.

Mini size ø 1.7:

1322 #1322B 2 pcs 20 pcs Standard size ø 2.2: # 1372 2 pcs #1372B 20 pcs

PRECI-SAGIX MALE V

NOPRAX** Standard male ø 2.2 mm with M2 threading and 0.9 mm internal hex. Contains Ni. Use the IMP-XS-042 screw-driver.

1373 1 pc

PRECI-SAGIX BASE RING

For cast-on technique in IRAX* (with precious alloys) or in NOPRAX** (with non-precious alloys, contains Ni). M2 threading for the # 1373 Standard male

RA 0063 IRAX* ALLOY 1 pc # RA 0064 NOPRAX** ALLOY 1 pc

PRECI-SAGIX 2.2 MALE VCC

TITANAX*** male ø 2.2 for CAD/CAM applications. M2 x 0.35 threading and 0.9 mm internal hex. Contains Ni. Use the IMP-XS-042 screw-driver.

1374 1 pc

PRECI-SAGIX HOUSING

Cr-Co female housing (Co 65 - Cr 28 - Mo 6) for incorporation into acrylic resin, for soldering with CEKA SOL or for the laser technique.

Mini ø 1.7: H 4.4 mm - W 4.0 mm - L 5.2 mm

1325 # 1325B 20 pcs 2 pcs Standard ø 2.2: H 5.4 mm - W 4.9 mm - L 5.7 mm 2 pcs

1375

1375B

20 pcs

PRECI-SAGIX PARALLELING MANDREL FOR MALE

Universal paralleling mandrel for PRECI-SAGIX males. For lab use only.

1300 P for ø 1.7 and 2.2 1 pc

PRECI-SAGIX SCREW-DRIVER FOR THREADED MALE

Total length: 26 mm - External hex: 0.9 mm

IMP-XS-042

1 pc

PRECI-SAGIX INSERTION TOOL

Universal insertion tool for PRECI-SAGIX females 1 pc

1323 for ø 1.7 and 2.2

PRECI-SAGIX ANALOGUE

Male model analogue. For lab use only.

1331 for ø 1.7 2 pcs # 1381 for ø 2.2 2 pcs

* IRAX 1400-1460 °C (2552-2660 °F) - For cast-on technique with precious alloys only. Au 60 - Pt 24 - Pd 15 - Ir 1

** NOPRAX 1355-1450 °C (2471-2642 °F) - For cast-on technique with non-precious alloys only. Ni 72 - Cr 17 - Fe 10 - Mn 1.

*** TITANAX - Only for bonding and acrylic fixation. Ti 90 - Al 6 - V 4 Do not heat!

CEKA AXIAL PRECI-CLIX AXIAL PRECI-CLIX RADICULAR PRECI-BALL

CEKA EXTRACORONAL PRECI-VERTIX PRECI-CLIX EC

PRECI-SAGIX

PRECI-HORIX

PRECI-52

PRECI-PROFILE PRECI-POST **CEKA SOL CEKA SITE CEKA BOND** 3C-BOND PERMA-RET PRECI-SEP **PLASTICWAX EXPANDO** CEKA Multi (O) MEASURING GAUGE

CKPL DIGITAL LIBRARY





WWW.CKPL.EU

SIDE EFFECTS. WARNINGS AND PRECAUTIONS

The attachments are intended for single use.

The products are non-sterile.

of the attachments.

There is a risk of poor fit when patient conditions change.

Bacterial adhesion can be avoided by applying hygiene measures. Inappropriate use or bad manufacturing can lead to premature wear


The functionality of the attachments will be adversely affected by traumas such as grinding and bruxism.


For the purpose of traceability we advise you to record the lot number of the applied products in the patient file.

Do not heat products containing titanium.

Do not use products containing nickel in case of nickel allergy.

The accessories RE H 79 and H 35 must be used outside the mouth.

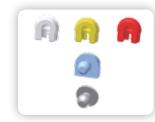

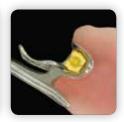













PRECI-SAGIX

SAGITTAL BALL ATTACHMENT WITH SEGMENTED FEMALE FOR PARTIAL **DENTURES & IMPLANT APPLICATIONS**





Patented snap mechanism



Inexpensive



Wide range of applications



Easy replacement of parts



Choice of working procedures



Long-lasting retention








1a. Castable plastic male. Determine the path of insertion. Incorporate the castable plastic male (# 1321 or # 1371) with the paralleling mandrel (# 1300 P) into the wax-up. A lingual bracing arm is recommended for removable partial dentures. Cast in a hard dental alloy.



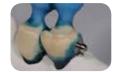



1b. Cast-on metal male. Determine the path of insertion. Incorporate the NOPRAX** male (# 1322 or # 1372) with the paralleling mandrel (# 1300 P) into the wax-up. A lingual bracing arm is recommended for removable partial dentures. Cast in a non-precious dental alloy.









1c. Threaded male. Determine the path of insertion. Assemble the threadable male (# 1373, Standard ø 2.2 mm) with the IRAX* base ring (# RA 0063 precious alloys) or the NOPRAX** base ring (# RA 0064 - non-precious alloys). Use the screw-driver (# IMP-XS-042). Incorporate the assembly with the paralleling mandrel (# 1300 P) into the wax-up. Unthread the male part. Use a two-stage investment technique and cast. PLASTICWAX bar profiles (# IMP-CK-014) are available.







2. Processing the female. Always use the pink duplicating dummy to make the refractory model! Place the pink duplicating dummies over the males and align them all parallel. Block out the undercuts and duplicate.







Cover also the replica with wax and cast the frame. Finish the cast frame without damaging the retentive ledge in the cavity for the female. Polish the access opening to a high shine. Press the pink processing female (# 1316 or # 1366) with the insertion tool (# 1323) into the frame to process the acrylic





Use of the chrome-cobalt housing (# 1325 or # 1375)

Use the duplicating dummy (# 1316 or # 1366) to make the refractory model. Wax up the structure and leave sufficient free space around the replica of the duplicating dummy (# 1316 or # 1366).



Replacing the processing female To replace a female, use a pointed instrument to dislodge and remove. Use the insertion tool (# 1323) to insert a new female. 3 retention levels are available: vellow: normal retention white: reduced retention

red: increased retention

INTRO KITS







PRECI-SAGIX INTRO KIT P

Introduction kit with castable plastic males. the necessary tools and females in 3 retention levels

for Mini size 1.7 and Standard size 2.2. Cast the males in any hard dental alloy.

22-piece kit:

2 x 1321 + 2 x 1371 castable plastic males

2 x 1311 + 2 x 1361 yellow females (normal retention)

2 x 1312 + 2 x 1362 white females (reduced retention)

2 x 1313 + 2 x 1363 red females (increased retention) 2 x 1316 + 2 x 1366 duplicating dummies/processing

females

1 x 1300 P universal paralleling mandrel

1 x 1323 universal insertion tool

Intro kit castable male: # 1301







PRECI-SAGIX INTRO KIT M

Introduction kit with cast-on NOPRAX males.

the necessary tools and females in 3 retention levels for Mini size 1.7 and Standard size 2.2.

Cast-on technique for non-precious alloys.

22-piece kit:

2 x 1322 + 2 x 1372 cast-on NOPRAX males

2 x 1311 + 2 x 1361 yellow females (normal retention)

2 x 1312 + 2 x 1362 white females (reduced retention)

2 x 1313 + 2 x 1363 red females (increased retention)

2 x 1316 + 2 x 1366 duplicating dummies/processing females

1 x 1300 P universal paralleling mandrel

1 x 1323 universal insertion tool

Intro kit cast-on male:

1303

#1313B 50 pcs

PRECI-SAGIX FEMALES

PRECI-SAGIX FEMALE - polyacetal



Mini size ø 1.7: height: 3.1 mm - ø 3.1 mm Yellow - normal retention: #1311 6 pcs # 1311B 50 pcs

White - reduced retention: #1312 6 pcs #1312B 50 pcs Red - increased retention:

#1313 6 pcs

Combination package:

#1314 2 + 2 + 2 pcs Standard size ø 2.2: height: 4.1 mm - ø 4.2 mm



Yellow - normal retention: #1361 6 pcs #1361B 50 pcs White - reduced retention:



#1362 6 pcs #1362B 50 pcs Red - increased retention:

#1363 6 pcs #1363B 50 pcs Combination package:



#1364 2 + 2 + 2 pcs



PRECI-SAGIX DUPLICATING DUMMY / PROCESSING FEMALE

For lab use only Mini size ø 1.7: #1316 6 pcs

Standard size ø 2.2:

1366 6 pcs

#1316B 50 pcs #1366B 50 pcs