BASIC eco

2949 XXXX

Quick Start Guide

DE

EN

FR

IT

ES

PT

TR

RU

ZH

JA

KO

AR

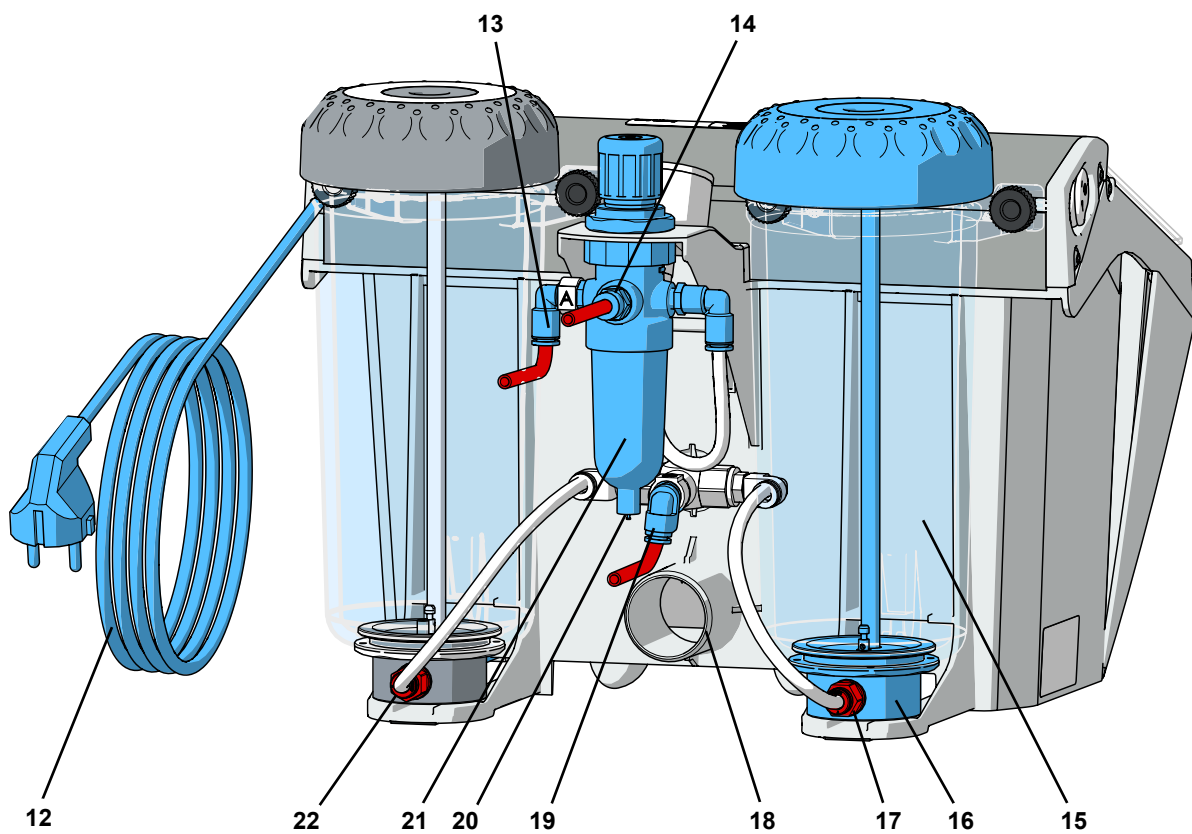
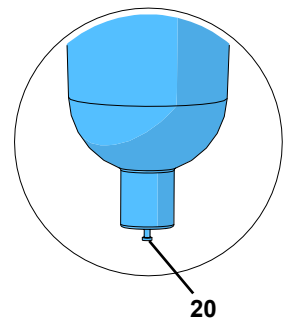
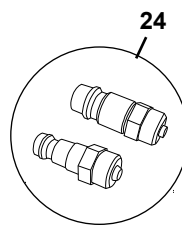
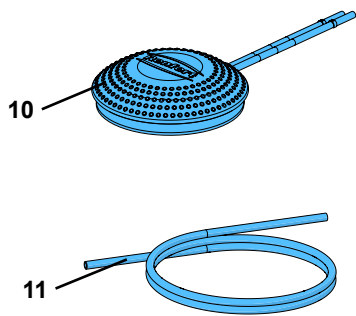
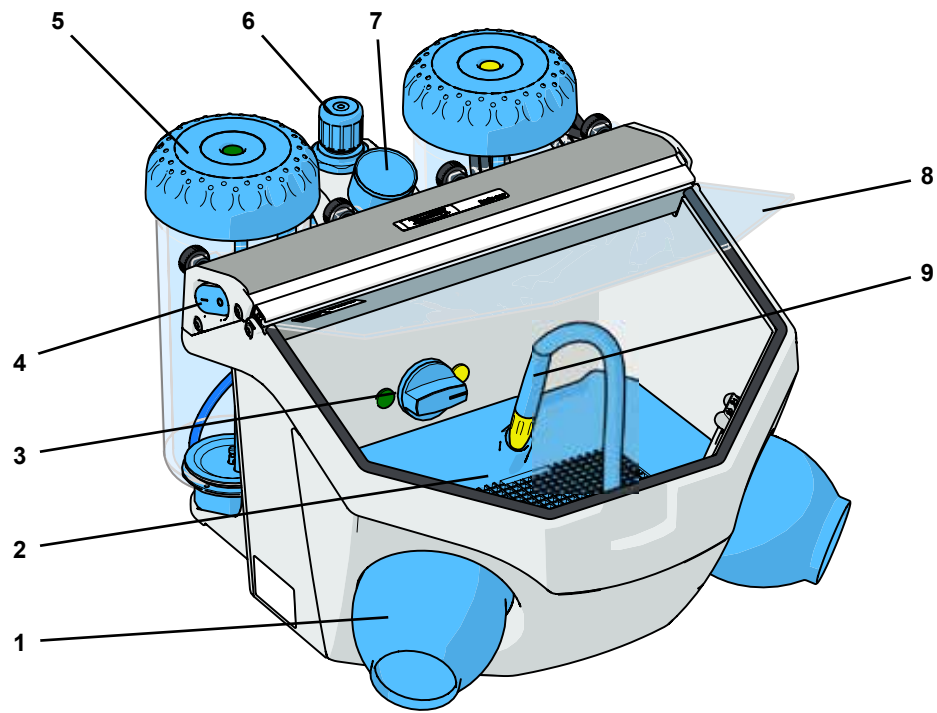
Fig. 1

Einleitung

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Introduction

This Quick Start Guide contains instructions on the operation of the product and relevant safety information.

Detailed information on the product can be found in the operating instructions which are available for download on our website at www.renfert.com.

Employed symbols

In these instructions or on the unit itself you will find symbols which have the following meaning:

Danger

This indicates an imminent risk of injury. Please observe the accompanying documents!

Electrical hazard

There is a risk of electrical shock.

Attention

Failure to observe the associated information can result in damage to the unit.

Note

The operating instructions contain useful information to make handling easier.

Attention!

Wear eye protection.

The unit complies with the relevant EC guidelines.

The unit is subject to the EC guidelines 2002/96/EG (WEEE Directive).

Other symbols are explained as they occur.

Safety

Intended use

The Basic line of sandblasting units are designed for use in dental laboratories to remove investment residues, oxides on cast objects, and for surface treatment.

Other areas of use also include divesting press ceramic or sandblasting porcelain occlusal surfaces.

Improper use

Only spare parts and accessories supplied or authorized by Renfert GmbH may be used with this product. If other spare parts or accessories are used, this could have a detrimental effect on the safety of the device, increase the risk of serious injury and lead to damage to the environment or the device itself.

Hazard and warning information

General information

- ▶ If the device is not used in compliance with the supplied instructions, the safety of the device can no longer be guaranteed.
- ▶ The device may only be operated using a mains cable with the country-specific plug system. Any necessary alterations must be carried out by a qualified electrician.
- ▶ The device may only be operated if the information on the identification plate conforms to the specifications of your local mains power supply.
- ▶ The device may only be plugged into outlets which are connected to the protective conductor system.
- ▶ The mains plug must be easily accessible.
- ▶ Disconnect the device from the mains before carrying out work on the electrical parts.
- ▶ It is the responsibility of the operator that national regulations during operation and regarding a repeated safety inspection of electrical equipment are complied with. For Germany these are the regulation 3 by DGUV (German Statutory Accident Insurance) in relation with VDE (Association for Electrical & Electronic Technology) 0701-0702.
- ▶ Check connection cables (such as power supply cords), tubes and housing (i.e. the key-pad) regularly for damage (i.e. kinks, cracks and porosity) or signs of ageing. Devices with damaged connection cables, tubes or housing parts or other defects must not be operated!
- ▶ Defective devices must be put out of service immediately. Remove the mains plug and ensure the device is not used. Send the device for repair!

- ▶ Only operate the device under supervision.
- ▶ Please observe the national accident prevention regulations!

Specific instructions

- ▶ Always disconnect the unit from the compressed air and power sources prior to beginning any maintenance work.
- ▶ When working on the tank (filling up, cleaning, maintenance) wear protective glasses for protecting your eyes.
- ▶ Any residue of abrasive material on the gasket may lead to leakage and early wear of the gasket. After filling up clean the thread and the gasket and close the lid properly.
- ▶ **DO NOT** use solvent-containing cleaning agents or disinfectants.
- ▶ Solvents and tensides can create micro-cracking in the plastic (danger of explosion!). Clean the tank and the lid only with a dry cloth. Do not write or stick something on the tanks.
- ▶ Do not unscrew the tank lid as long as the pressure had not been released.
- ▶ Check the tanks and the lids regularly on damages and replace them in case of doubt.
- ▶ Check the tank cover for correct seating prior to beginning operation. Covers that are not securely closed can suddenly be blown off as the tank pressure increases. The resulting flying parts and abrasive represent a serious hazard.
- ▶ Never operate the blasting units without suitable dust extraction and appropriate protective gear, as this could result in health hazards. The type of dust extraction should be selected on the basis of the dust being generated. Follow here absolutely EN 60335-2-69 appendix AA or ask the appropriate authorities.
- ▶ Improper use can result in the risk of eye or skin injuries.
- ▶ Never direct the blasting material towards eyes or unprotected areas of skin!
- ▶ Never work with the view screen open!
- ▶ **Caution:** Failure to wear proper eye protection can result in eye injury due to particles suspended in the air. Always wear proper eye protection to protect your vision!
- ▶ Do not employ the foot switch when only the compressed air supply hose is connected. The loose hose could wrap around itself and cause serious injury.

Authorised persons

This product may only be used by persons over the age of 14.
Operation and service may only be carried out by trained personnel.

Liability exclusion

Renfert GmbH shall be absolved from all claims for damages or warranty if:

- ▶ **The product is employed for any purposes other than those cited in the operating instructions.**
- ▶ **The product is altered in any way other than those alterations described in the operating instructions.**
- ▶ **The product is repaired by other than an authorized facility or if any but Renfert OEM parts are employed.**
- ▶ **The product continues to be employed, despite obvious safety faults.**
- ▶ **In the case of mechanical impact or if the unit is dropped.**

Product description

Components and functional elements

See Fig. 1 (on inside cover)

- | | |
|----------------------------------------------------|-------------------------------------------------------------------------------------|
| 1 Hand sleeves | 13 Foot switch connection (A) (push-in connection) |
| 2 Base grid | 14 Air connection (push-in connection) |
| 3 Selector switch
(only on the 2 tanks version) | 15 Sandblasting tank |
| 4 ON / OFF switch (lighting) | 16 Mixing chamber |
| 5 Tank lid | 17 Filter cartridge (push-in connection) |
| 6 Pressure regulator | 18 Suction tube |
| 7 Pressure gauge | 19 Foot switch connection (B) (push-in connection) |
| 8 Protection screen | 20 Water trap valve |
| 9 Hand piece | 21 Water trap |
| 10 Foot switch | 22 Foot switch connection (B) (push-in connection)
(only on the 1 tanks version) |
| 11 Compressed air tube | 23 Dosing nipple tool |
| 12 Power cord | 24 Connection set |

Standard delivery

- 1 Sandblasting unit *Basic eco*
- 1 Foot switch
- 1 Connection set
- 1 Dosing nipple tool
- 1 Quick Start Guide

Delivery format

- 2949 1XXX, 2949 3XXX 1 tank version
- 2949 2XXX, 2949 4XXX 2 tank version

Installation

Unpacking

⇒ Check to ensure all delivery parts are complete (compare “Included in Delivery”).

Setting up

⇒ Place the unit on a level and stable base.

Connect foot switch

When inserting the tubes you will notice a slight resistance which must be overcome.

2 Tank version:

- ⇒ Push the tube marked „A“ as far as it will go into the tube connection point „A“ (13).
- ⇒ Push the tube marked „B“ as far as it will go into the tube connection point „B“ (19).

1 Tank version:

- ⇒ Push the tube marked „A“ as far as it will go into the tube connection point „A“ (13).
- ⇒ Push the tube marked „B“ as far as it will go into the tube connection point „B“ (22)
(on sandblasting tank).

Compressed air connection

- ⇒ Push the compressed air tube (11) onto the tube connection point (14) as far as it will go. In doing so there will be a slight resistance to overcome.
- ⇒ Select the relevant hose connection piece from the enclosed attachment set and fit it onto the end of the tube.
- ⇒ Attach to the compressed air network.

Electrical connection

⇒ Insert the power plug into the building installed wall socket.

Connect to extraction unit

- ⇒ Connect the suction tube from the extraction unit to the suction pipe (18).
- For suction devices with automatic activation:
- ⇒ connect the mains plug from the sandblasting unit into the user socket on the suction device.

Filling the sandblasting tanks

EN

Attention: Never press the foot switch during filling.

- ⇒ Add abrasive up to the maximum fill level (the point where the tank jacket thickens just below the thread).
- ⇒ Reinstall the tank lid (5).

Authorised sandblasting material

- Cobra (Renfert GmbH)
- Rolloblast (Renfert GmbH)
- ROCATEC (3M ESPE)

Sandblasting materials from other manufacturers may be used providing the grain size, grain shape and degree of purity comply with the above mentioned materials.

Operation

- ⇒ Switch the illumination on (4).
- ⇒ Select the blasting tank at the selection switch (3) (only on the 2 tanks version).
- ⇒ Take the handpiece out of the holder of the relevant sandblasting tank (see color markings).
- ⇒ Activate the foot switch (10).

Select blasting pressure

- ⇒ Take the handpiece out of the holder of the relevant sandblasting tank (see color markings).
- ⇒ Activate the foot switch.
- ⇒ Set the sandblasting pressure at the pressure regulator (6) (see „Technical Data“).
 - ◆ The amount of pressure will be shown on the pressure gauge (7).

Cleaning / maintenance

Inside the extractor there are small parts which require maintenance.

Opening the device, other than for the processes described below, is not permissible!

Clean housing / tanks

- ⇒ **DO NOT** use solvent-containing cleaning agents or disinfectants.
- ⇒ **DO NOT** use aggressive or abrasive cleaning agents.
- ⇒ Wipe the housing and aluminium frame gently with a slightly damp cloth, do not apply pressure.
- ⇒ Wipe out the tanks and lids only with a dry cloth!

Cleaning the Interior

- ⇒ Remove bottom grate and the blast protection mat.
- ⇒ Vacuum the blasting chamber.
- ⇒ Check the unit regularly for damage underneath the bottom grate / blast protection mat.
- ⇒ Exchange the damaged bottom grate / blast protection mat.

Condensation

- ⇒ Inspect the water separator (21).
- ⇒ Drain any water off through the valve (20).

Spare parts

You can find components subject to wear and the spare parts on the spare part list in the internet at www.renfert.com.

The components excluded from the warranty (such as consumables or parts subject to wear and tear) are marked on the spare part list.

Disposal instructions

Within the European Union units with this symbol should not therefore be disposed of in unsorted domestic waste.

For more information regarding proper disposal please apply at your local authorities.

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Hilzingen, 20.04.2016

Technical Data

Nominal Voltage	230 V	120 V	100 V
Permissible mains voltage:	220 - 240 V	120 V	100 V
Mains frequency:	50 / 60 Hz		
Power consumption:	9 W		
Maximum connection pressure:	6 - 8 bar [87- 116 psi]		
Working pressure:	1 - 6 bar [14.5 - 87 psi]		
Dimensions (Width x Height x Depth):	350 x 275 x 400 mm [13.8 x 10.8 x 15.7 inch]		
Weight, approx.: 1 tank version: 2 tank version:	4.4 kg [9.7 lbs] 5.5 kg [12.1 lbs]		
Ø Compressed air hose: interior: exterior:	4 mm [157 mil] 6 mm [236 mil]		
Ø Size of pipe union for external extraction unit: interior: exterior:	35 mm [1.38 inch] 40 mm [1.57 inch]		
Blasting chamber volume:	10 l [2.64 US gal]		
Tank capacity, each:	1000 ml [34 fl oz]		

**Hochaktuell und ausführlich auf ...
Up to date and in detail at ...
Actualisé et détaillé sous ...
Aggiornato e dettagliato su ...
La máxima actualidad y detalle en ...
Актуально и подробно на ...**

www.renfert.com

Ideas for dental technology

Renfert GmbH • Industriegebiet • 78247 Hilzingen/Germany
Tel.: +49 7731 82 08-0 • Fax: +49 7731 82 08-70
www.renfert.com • info@renfert.com

Renfert USA • 3718 Illinois Avenue • St. Charles IL 60174/USA
Tel.: +1 6307 62 18 03 • Fax: +1 6307 62 97 87
www.renfert.com • info@renfertusa.com
USA: Free call 800 336 7422

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aşağıda belirtilen AB Yönetmelikleri ile uyumlu olduğunu
yetkili kişi: c/o Renfert GmbH