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**GUARANTEED TOP QUALITY
THROUGHOUT**

English

User Information

FINO GASEX Decontamination Agent
23150

By purchasing this product you have made an excellent choice.

FINO quality products from DT&SHOP convince through consistently high quality for economical prices.

1. FIELDS OF APPLICATION

Decontamination agent for ceramic firing. FINO GASEX prevents that gases, which can develop if non-precious or low-precious alloys are used, affect the firing of porcelain.

2. ADVANTAGES

- ◆ Improves the quality of ceramic
- ◆ Cleans heating element of the ceramic furnace
- ◆ Absorbs developing gases

3. PHYSICAL DATA

No information.

4. PROCESSING / INSTRUCTIONS

Simply add FINO GASEX Sticks to the fired objects for every porcelain firing. FINO GASEX can be placed on the firing tray or directly on firing mould. Firing has to be done in accordance with the temperature directions of the manufacturer of the porcelain.

4.1 Notes

FINO GAS-EX must not come in direct contact with fired objects.
FINO GAS-EX degrades by itself after a couple of applications. Hence exchange the sticks in time to ensure a safe decontamination.

5. STORAGE

No information.

6. SAFETY PRECAUTIONS

No information.

7. AVAILABILITY

FINO GASEX Decontamination Agent
12 pieces 23150

8. GUARANTEE

Our technical recommendations of application are based on our own experiences and tests and should only be regarded as guidelines. It rests with the skills and experience of the user to verify that the products supplied by us are suitable for the intended procedures. Our products are undergoing a continuous further development. We reserve the right of changes in construction and composition. It is understood that we guarantee the impeccable quality of our products.

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