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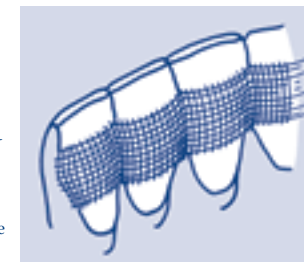
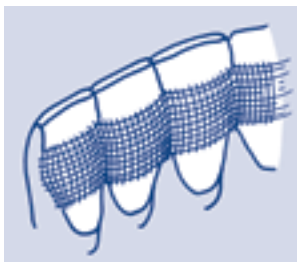
System for the Splinting and Stabilization of Teeth

Instructions for Use

Affixation

1. Cut the Perfect-Splint net to the desired length. Cut V-shapes along the gingival margin side to match the interproximal area.
2. Clean the teeth to be splinted with cleaning paste and etch the palatal or lingual glued surface with a phosphoric acid medium, such as Micro-Etch.
3. Place the orthodontical wire (cut into approx. 5 cm pieces) interproximally along the prepared teeth.
4. Apply a thin layer of composite to the entire etched tooth surface.
5. Place a small amount of a flowable composite in the center of the palatal or lingual surface of the tooth. Start with the first tooth to be splinted.
6. Push the prepared Perfect-Splint net to the tooth with an instrument, e.g. an amalgam condenser. The material should pour through the mesh. If this does not happen, apply more of a flowable composite to the mesh.
7. Allow the composite to cure.
8. Lead the orthodontical wire over the net to the labial side and pull it with both ends of the wire into the approximal area. Repeat procedure from point 5.
9. Remove the wires and completely cover the Perfect-Splint net with a composite material.

- Areas of application:**
- traumatically or periodontally loosened teeth
 - transplanted, implanted und replanted teeth
 - before and after surgical procedures
 - diastema closures
 - orthodontical procedures

- Recommendation:**
- intraoral use not longer than 30 days

Removal

1. Place the Desmoclean® instrument in the green, slowly rotating contra angle hand piece and loosen the mesh interdentially between the teeth that were splinted last.
2. When the net is ripped, it can be removed palatally or lingually with Desmoclean – with light pressure and water spray.
3. Once the mesh is removed and the etched enamel smoothed, a sealing of the area is recommended in order to prevent the development of caries.

The gingiva calms down quickly by applying the Myzotect-Tincture or Miradont Gel.

Perfect-Splint-System is a disposable product. It was not designed to be re-used. Sterilizing it could have a negative impact on the product features. Re-use of the article causes a high risk of infection.

* Other instruments, such as finishers, may also be employed. The Desmoclean provides for a fast and easy removal while preventing damage to the enamel. Tartar that could not be removed during the wear of a splint can be eliminated at the same time and without instrument change.

Warning

Perfect Splint C is a disposable product. It was not designed to be re-used. Sterilizing it could have a negative impact on the product features. Re-use of the article causes a high risk of infection.

Combi set	10 PET nets/10 stainless steel nets/attachment aid	REF 152 280
Set stainless steel	20 stainless steel nets/attachment aid	REF 152 281
Set PET	20 PET nets/attachment aid	REF 152 282

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