

Info Sheet INFO 138.E-ed.B

Date of issue: 01/03/17 Replaces version from: 01/09/09

Page 1/1

PROCESSING THE MALES OF PRECI-SAGIX P

- 1. Determine the path of insertion of the prosthesis.
- 2. Select the appropriate **1321** (Mini Ø 1.7 mm) or **1371** (Standard Ø 2,2 mm) plastic male and place it on the **1300 P** paralleling mandrel. The spherical part of the male fits into the bent part of the paralleling mandrel.
- 3. Fix the plastic male to the abutment crowns with wax. Make sure that there is sufficient space for interdental cleaning.
- 4. Remove the **1300 P** paralleling mandrel from the male by sliding up the rubber ring.
- 5. Sprue the crowns and prepare for investing.
- 6. Invest, preheat carefully, and cast in a hard alloy. Do not use fast heating investments.
- 7. Sandblast the casting without damaging the male.

Side effects, warnings and precautions

- The attachments are intended for single use.
- The products are non-sterile.
- There is a risk of poor fit when patient conditions change.
- Bacterial adhesion can be avoided by applying hygiene measures.
- Inappropriate use or bad manufacturing can lead to premature wear of the attachments.
- The functionality of the attachments will be adversely affected by traumas such as grinding and bruxism.
- For the purpose of traceability we advise you to record the lot number of the applied products in the patient file.
- Do not heat items containing titanium.
- Do not use items containing nickel in case of nickel allergy.
- The accessories RE H 79 and H 35 must be used outside the mouth.