Cercon®

# TCT Stains

Directions for Use – Supplement  
Ergänzung zur Gebrauchsanweisung  
Complément au mode d'emploi  
Integrazione delle istruzioni per l'uso  
Ampliación de las instrucciones de uso

# Cercon® TCT Stains

<b>en</b>	Directions for Use - Supplement.....	04 - 06
<b>de</b>	Ergänzung zur Gebrauchsanweisung.....	07 - 09
<b>fr</b>	Complément au mode d'emploi.....	10 - 12
<b>it</b>	Integrazione delle istruzioni per l'uso.....	13 - 15
<b>es</b>	Ampliación de las instrucciones de uso.....	16 - 18

# Cercon® TCT Stains

## Directions for Use – Supplement

### **Indications for use:**

Cercon® stains are designed for use on zirconia (zirconium oxide) in single tooth or bridge type restorations. Applications include both anterior and posterior locations.

### **Product Information:**

Cercon® stains are suitable for applying individual shades and characterizations to zirconia (Y-TZP) crowns and bridges with a coefficient of thermal expansion of 10.5  $\mu\text{m}/\text{m}\cdot\text{K}$  (25–500°C), preferably made of Cercon®.

Follow the Directions for Use for Cercon® ceram Kiss/ Cercon® ceram love. For the most current versions of the Directions for Use and firing tables, visit [www.dentsplysirona.com](http://www.dentsplysirona.com).

### **Contraindications:**

Only the indications listed above are suitable.

### **Warnings/Adverse Reactions:**

Consult Cercon® TCT Stains and Glaze Safety Data Sheets (SDS) for safe handling and usage. Use only according to the indications listed above.

### **Precautions:**

Refer to the special notes throughout the instructions below for best practices for using Cercon® TCT Stains and Glaze.

- This product is intended to be used only as specifically outlined in these Directions for Use. Any use of this product inconsistent with the Directions for Use is at the discretion and is the sole responsibility of the practitioner.

### **Storage conditions:**

Store materials in packages with the lid tightly sealed. Do not leave open bottles where dust and debris may contaminate the porcelain. Replace cap when not in use. Avoid vibration and heat. Keep dry. Shake well before use. The lot number is printed on each bottle; please refer to the lot number in all correspondence. Do not use after expiration date.

## Firing chart

Please observe the instructions of our zirconium oxide discs Cercon® ht. There you will find information on the intended use, contraindications, warnings, cautions and technical data.

Make sure to apply the stains in very thin layers. For darker shades, an extra firing step may be added if necessary in order to obtain even better shade fidelity.

°C	Dry	Preheat	Low Temp	Heat Rate	High Temp	Vac Start Temp	Vac Stop Temp	Hold Time	Hold Time Air	Cool Time
	(min)	(min)	°C	°C	°C	(min)	(min)	(min)	(min)	(min)
Stains/Glaze ⇨ PFZ	3	3	450	55	820	0	0	0	1.5	6
Stains/Glaze ⇨ Cercon® ht	3	3	450	55	820	0	0	0	1.5	6

°F	Dry	Preheat	Low Temp	Heat Rate	High Temp	Vac Start Temp	Vac Stop Temp	Hold Time	Hold Time Air	Cool Time
	(min)	(min)	°F	°F	°F	(min)	(min)	(min)	(min)	(min)
Stains/Glaze ⇨ PFZ	3	3	842	99	1,508	0	0	0	1.5	6
Stains/Glaze ⇨ Cercon® ht	3	3	842	99	1,508	0	0	0	1.5	6

### Cercon® TCT Stain & Glaze

REF

Cercon® TCT Stain Kit 1x Universal Stain & Glaze Liquid 1x Cercon® TCT Glaze 15 x Cercon® TCT Stains		D651590
Dentsply Prosthetics Universal Stain & Glaze Liquid	15 ml	D601315
Dentsply Prosthetics Universal Stain & Glaze Liquid	50 ml	D601350
Cercon® TCT Glaze	5g	D651322
Cercon® TCT Stain 0	5g	D651500
Cercon® TCT Stain 1	5g	D651501
Cercon® TCT Stain 2	5g	D651502

REF

Cercon® TCT Stain 4	5g	D651504
Cercon® TCT Stain i1	5g	D651511
Cercon® TCT Stain i2	5g	D651512
Cercon® TCT Stain white	5g	D651520
Cercon® TCT Stain creme	5g	D651521
Cercon® TCT Stain sunset	5g	D651522
Cercon® TCT Stain copper	5g	D651523
Cercon® TCT Stain khaki	5g	D651524
Cercon® TCT Stain olive	5g	D651525
Cercon® TCT Stain mahogany	5g	D651526
Cercon® TCT Stain purple	5g	D651505

## The sleek color scheme

V- Shade	Disk- Shade	1 <sup>st</sup> Stain firing Stain/glaze	2 <sup>nd</sup> Stain firing Stain/glaze	Incisal
A1	A1*	Stain 3, some olive		Stain i1 + i2
A2	A1	Stain 2 + 3, some olive		Stain i1 + i2, white
A3	A3*	Stain 1 + 3		Stain i1 + i2
A3,5	A3	Olive	Stain 3	Stain i1 + i2
A4	A3	Stain 1 + 3	Stain 3	Stain i1 + i2
B1	B1*	Stain 3		Stain i1 + i2
B2	B1	Stain 3 + 4		Stain i1 + white
B3	B3	Stain 2 + 3		Stain i1 + i2
B4	B3*	Stain 3 + 4, some olive + sunset		Stain i1 + i2
C1	C1*	Stain 3		Stain i1 + i2
C2	C1	Stain 3		Stain i1 + i2
C3	C1	Stain 3		Stain i1 + i2
C4	C1	Stain 3, ein wenig olive		Stain i1 + i2
D2	D2*	Stain 4		Stain i1 + i2
D3	D2	Stain 4		Stain i1 + i2
D4	D2	Stain 3, some olive olive	Stain 3, some sunset	Stain i1 + i2

\* Cercon® TCT Stains are necessary for individualisation not for coloring.

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DeguDent GmbH