BOHRERBAD PLUS

REF 200 720 Inhalt/Content/Contenu/Contenuto/Obsah/Tartalom: e 2 L

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DRILL BATH PLUS

Type of application:

For manual cleaning and prophylactic disinfection of drills, milling cutters, diamond grinders and other rotating instruments. Not suitable for the terminal disinfection of invasive (semi-critical and critical) medical devices.

Instructions for use:

Always use the M+W Bohrerbad Plus undiluted! After use, put the instruments straight into the ready-for-use solution, all the parts to be disinfected have to be fully covered. After disinfection the instruments should be rinsed, ideally with demineralised water, and dried thoroughly. Do not mix with other disinfectants or cleaners, not suitable for aluminium. Depending on the degree of contamination, the working solution may need to be replaced (RKI recommendation: at least once per working day). The working solution can also be disposed of in the sewage water system. It is recommended to wear safety gloves and eye protection.

Exposure times:

Bactericidal (incl. MRSA) as per EN 13727, EN 14561, VAH, (dirty condition): Conc. 5 min

Levurocidal (Candida Albicans (EN 13624, EN 14562, VAH, dirty condition): Conc. 5 min

Tuberculoicidal (M. terrae) as per EN 14348, EN 14563, clean condition: Conc. 5 min

Limited virucidal (enveloped viruses (e.g. HBV, HIV, HCV)

as per DVV/RKI (dirty condition): Conc. 5 min

Safety notes:

H412 Harmful to aquatic life with long lasting effects. Avoid release to the environment. Disposal of contents / container in accordance with local / regional / national / international regulations.

Ingredients:

REG 648/2004/EC:
Contains <5% non-ionic surfactants, <5% cationic surfactants, <5% EDTA salts, fragrances.

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