EASY
EXCELLENCE

3D-SCANNERS
MILLING
PROSTHETIC PARTS
ORTHODONTICS

Instruction manual

CS.Neo2 / CS.Neo2 Pro / CS.Neo2 Max

Table of contents

1. Safety and general information.....	2
1.1 Symbols and means of representation.....	2
1.2 Labeling	3
1.3 Customer service.....	3
1.4 Safety instructions for individual operating phases.....	3
1.5 Intended use.....	4
1.6 Potential misuse.....	4
1.7 Intended environment.....	4
1.8 Duties of the operator	4
1.9 Qualifications of personnel	5
1.10 Duties of personnel.....	5
1.11 Personal protective equipment.....	5
2. Transport and installation.....	5
2.1 Unpacking and setting up the device	6
2.2 Select a setup area.....	6
2.3 Carrying points.....	7
2.4 Installation example.....	7
3. Device overview and connections	8
3.1 Components of the scanner.....	8
3.2 Scanner connector strip.....	8
3.3 Power supply unit	9
3.4 USB cables	9
3.5 Accessories.....	9
4. Function and installation.....	9
4.1 Installing the scanner.....	9
4.2 Installing the computer	10
4.3 Installing the software	10
5. Working process	10
5.1 ID recognition.....	11
5.2 LED indication bar	11
5.3 Switching off the scanner.....	11
6. FAQ Troubleshooting	11
7. Cleaning and maintenance work for the operator	11
7.1 Cleaning.....	12
7.2 Recalibration	12
7.3 Repair / Troubleshooting.....	12
8. Disposal	12
9. Technical data.....	13
9.1 Product diagrams & clearances.....	14

1. Safety and general information

This operation manual is structured in accordance with the applicable EU regulations and contains safety instructions. Individuals are responsible for compliance with safety instructions. This chapter contains a general introduction to the safety instructions. It also contains important information on accident prevention.

This optical 3D scanner (hereinafter referred to as the scanner), is a state-of-the-art product in terms of safety for operating personnel and operational reliability. The scanner may nevertheless present hazards if it is used by inadequately trained personnel, or if it is used improperly or not as intended. We refer to possible dangers explicitly in the chapter "Safety and general information," as well as elsewhere in the operation manual.

This operation manual contains the relevant information for installation, commissioning, operation, maintenance and disposal of the scanner.

This operation manual is intended to enable the user to work safely with the scanner.

Observing the instructions in this document will help to avoid hazards and prevent damage to the scanner.

All persons working on and with the scanner must have read the operation manual and observe the information and notes relevant to them. This operation manual must always be complete and fully legible.

CADstar Technology is not liable for any typographical errors in this manual. Likewise, no liability is assumed for damages that are directly or indirectly attributable to the delivery. Even after taking the utmost care in the design and construction of the scanner, and even after taking all issues relevant to safety into account, residual hazards may still exist. These have been evaluated by means of a risk assessment.

This chapter lists all residual risks and warnings from the risk assessment.

The information in this document is subject to change without notice. Every effort has been made in the preparation of this document to ensure the accuracy of its contents. Any statement, information or recommendation in this document does not constitute a promise of characteristics of any kind, either expressed or implied.

1.1 Symbols and means of representation

1.1.1 Warnings and additional information

Warnings

The sections marked "NOTE", "CAUTION" and "DANGER" in this document do not include all safety instructions to be followed. They are only supplements to the general safety instructions. CADstar assumes no liability for any consequences caused by the violation of general safety requirements or design, production and usage safety standards. This document distinguishes between the following warning levels, depending on the severity and probability of the danger:

Warning symbol	Description
	DANGER Indicates a significant hazard that will result in death or serious injury if not avoided.
	CAUTION Indicates a minor hazard that may result in minor injury if not avoided.
	NOTE A NOTE is used to explain practices that are not related to personal injury or to improve operations with the product.

1.1.2 Safety sticker on the scanner

Safety instructions are attached to the scanner, which draw attention to possible dangers/residual dangers.

The instructions on the safety sticker on the scanner must be followed under all circumstances. If the safety labels become faded or damaged during the life of the scanner, they must be replaced immediately with new stickers. They must be checked at regular intervals to ensure they are

complete and legible. As soon as the signs are not immediately recognizable or comprehensible at first glance, the optical 3D scanner must be taken out of operation until the new signs are installed.

The pictograms for warnings, prohibitions and requirements on the scanner and their meaning:

Symbol	Meaning
	Crushing hazard This pictogram is located in the area of the swivel arm and warns of a possible crushing hazard between the scanner and the swivel arm.

1.2 Labeling

The scanner is clearly identified by the contents of its nameplate. CE marking according to:

- EMC Directive 2014/30/EU
- Low Voltage Directive 2014/35/EU
- RoHS Directive 2011/65/EU

1.3 Customer service

In case of technical problems, contact your customer service. Or contact CADstar Technology directly. Link: www.cadstar.dental
CADstar Technology Headquarters, Grasslau 30, 5500 Bischofshofen, Austria

1.4 Safety instructions for individual operating phases

1.4.1 Transport and installation

- The scanner may only be transported if it has been properly packed and secured.
- When handling the scanner, ensure that the scanner does not bend, especially when parking after transport.
- The scanner is designed to be set up at an ergonomic height. (Observe the employee protection regulations in your country).
- When laying the cables, ensure that they do not pose a tripping hazard. Always lay the cables along walls. Under no circumstances may the cables be routed so that they are under tension or sagging.
- The location of the scanner must be chosen so that there are no vibrations during scanning operation, i.e. the load capacity of the chosen base (workbench, table, etc.) must be at least twice the value listed in the Technical Specifications and must absorb all vibrations.
- The ambient light must be taken into account when setting up the device. There must not be any change in the ambient light during the scan operation, such as might occur due to shadows from trees or blinds. Neon lights or direct sunlight should also be avoided.
- The scanner must not be operated in damp rooms.
- No liquids should come into contact with the scanner. Do not place any vessels with liquid on the scanner.

1.4.2 Installation and operation

- The operating and maintenance personnel must be instructed regularly on the operation of the system and on potential hazards. They are required to read the operation manual.
- Before each start-up, make sure that there are no parts in the swivel range of the arm, as it performs a reference run when switched on.
- Keep children away from the scanner. Only trained personnel may work with the scanner.
- The optical 3D scanner may only be operated when it is in perfect and clean condition.
- Power supply cables and feeder cables must not be damaged or crushed by other objects.
- Malfunctions or damage must be reported to the operator immediately.
- Removing, modifying, bypassing or circumventing any protective, safety or monitoring devices is prohibited.
- Be aware of possible crushing hazards when operating the scanner inserts.
- If the scanner is not in use for a longer period of time, disconnect it from the power supply.
- Do not place any objects or tools on the scanner.
- Ensure that the area around the scanner remains clear and organized. All objects and containers not required for the workflow must be removed from the area of the scanner.
- Climbing on the scanner, as well as sitting or standing on the scanner are prohibited.

	The swivel arm will move when the scanner is switched on. Do not reach into the danger area as finger injuries may occur.
	The swivel arm moves during scanning. Do not reach into the danger area as finger injuries may occur.

1.4.3 Improper maintenance, modifications and repairs

The subsequent installation of additional devices from other manufacturers is not permitted, nor is the modification of protective devices without consulting CADstar Technology or the manufacturer of the respective additional device.

- Repairs to the scanner may only be performed by CADstar Technology customer service.

1.5 Intended use

The intended use of the scanner is the three-dimensional, optical measurement of duplicated human structures in combination with the supplied software.

Only impressions and plaster models of the jaws, as well as articulators with plaster models may be inserted (max. 1500 g) dimensions W x L x H: 200 mm x 200 mm x 150 mm.

Only accessories recommended by CADstar Technology may be used. The scanner is intended to be used in dental laboratories. Any other use requires the written consent of CADstar Technology. Improper use may endanger persons and damage the scanner. The product is not an automatically functioning device; it must be loaded manually. Furthermore, any liability and warranty claims are excluded in the event of non-compliance with the intended use.

1.6 Potential misuse

The information in this documentation describes the properties of the product, but does not guarantee them. No liability and warranty claims will be accepted for damages caused by:

- Improper use of the scanner.
- Disregarding the operating instructions.
- Unauthorized changes to the scanner.
- Improper work on and with the scanner.
- Improperly performed repairs.
- Unauthorized, improper modification of operating parameters.
- Catastrophic events, impacts due to foreign objects and force majeure.

1.7 Intended environment

The limits of use for the scanner are restricted as follows:

- To be operated indoors only
- Ambient temperature: 18 °C to 35 °C
- Relative humidity: up to 31°C 20 % - 80 % at temperatures up to 35°C maximum 20-50% linear decreasing - not suitable for humid rooms, non-condensing
- No flickering light (e.g. fluorescent lighting; moving blinds; shadows from trees in the wind).
- Maximum ambient brightness 500lux. No direct sunlight in the scanning area
- Clean and dust free environment
- Horizontal installation site on a stable, vibration-absorbing workstation
- Maximum installation altitude <2000m above sea level

1.8 Duties of the operator

The operator is responsible for the safe operation of the scanner.

- Make sure that only trained, educated or qualified persons work with the scanner.
- Make sure that every person working on the scanner for the first time has read and understood this operating manual.
- Draw special attention to the safety instructions in this manual and on the scanner.
- Keep the operation manual in the area near the scanner and pass it on to new personnel.
- Make sure that no one works on the scanner without having the professional expertise necessary to do so.
- Operating personnel must be instructed at regular intervals on how to work safely with the scanner.
- Please observe national regulations for the protection of female employees.
- When servicing, maintaining or cleaning the scanner, it must be disconnected from the power supply.
- Repairs may only be carried out by a certified service agent
- Make sure that the notices attached to the scanner when it is delivered remain legible. Missing and damaged warning signs must be replaced immediately.

1.9 Qualifications of personnel

This manual is addressed to the operating and maintenance personnel of the scanner. The operating and maintenance personnel are to be named by the operator and must have the following qualifications:

Maintenance personnel:

- Basic technical knowledge
- Reading and understanding these operating and maintenance instructions is a prerequisite.
- The person must be strong enough to lift the unit and place it in its final location.
- Inspection and cleaning may only be carried out by technical specialists with product-specific training or under the guidance of the customer service department

Operators:

- Product training
- Regular safety instruction

1.10 Duties of personnel

- The safety instructions and operating instructions contained in this operating manual must be observed at all times.
- This scanner may only be operated by appropriately qualified personnel. These personnel must be familiar with all safety instructions and with the corresponding measures contained in this operating manual and on the scanner for operation, maintenance and cleaning.
- The optical scanner must be disconnected from the power supply before cleaning, maintenance and repair work.

1.11 Personal protective equipment

The 3D scanner is designed in such a way that the persons working on the scanner do not require any additional protective equipment beyond the required standard of the operator (e.g. work clothing, work shoes, etc.).

- Persons performing cleaning and maintenance work must observe the prescribed measures for the corresponding cleaning agents (e.g. gloves for cleaners; splash protection, etc.).

2. Transport and installation

First of all, check to ensure the packaging is undamaged. If this is not the case, document the damage and contact the seller. Pay attention to the warnings on the packaging during transport. The device must be transported or stored so that it is well protected, dry and upright in a package.

In case of longer storage, the device should be stored in a dry place in the original packaging. Care must be taken to comply with the storage requirements as stated in the specification.

**NOTE!**

Transport the scanner only in the original packaging to avoid damage to the device.

2.1 Unpacking and setting up the device

First, open the top of the box and remove the top foam cover with the optional accessories. Then you can hold the scanner by the two recesses and pull it out of the box. Now the fixation for the arm can be removed, the standard accessories are inside the fixation.

Save the packaging for future transport (relocation, service, ...).

Dispose of waste materials in an environmentally friendly manner.

2.2 Select a setup area

Since the environment of the scanner has a significant influence on the product, you should be careful when selecting its location and observe the following:

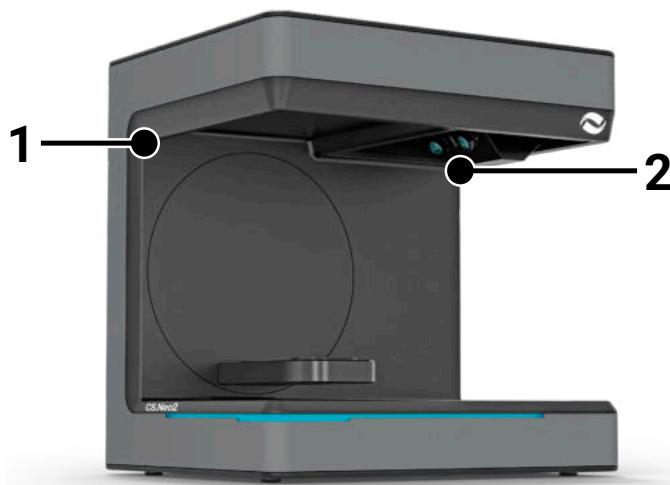
- Avoid rapidly changing lighting conditions such as moving shadows, office lighting that flickers (fluorescent lamps, LED...).
- Never expose the scanner to direct sunlight.
- Avoid direct proximity to heat sources (e.g. radiators, other electrical appliances, fireplaces, etc.).
- Ensure that the installation site is level and vibration-free.
- It must be ensured that there is no extraordinary exposure to dust, respectively poisons or corrosive gases at the installation site.
- Select a mounting or installation location where liquids or objects cannot get into the scanner under any circumstances (e.g. condensation, roof leaks, watering water, etc.).
- Ensure that the mains plug of the power supply unit is always accessible.
- The scanner may only be operated indoors.

Info: Strong external light or sunlight may affect the scan result.

This can lead to poor scan quality or to undesired deviations of the measurement result.

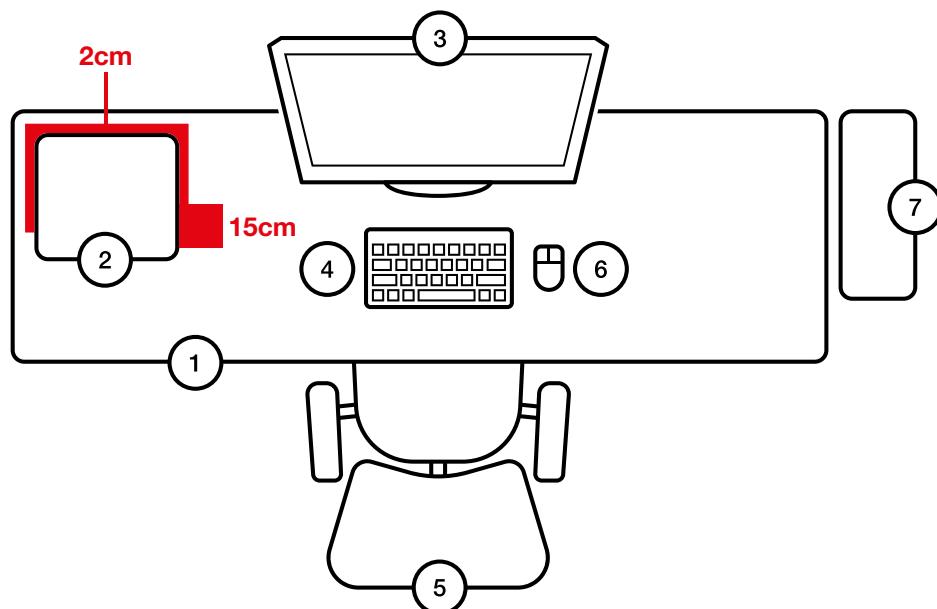
2.3 Carrying points

The two carrying points (1) and (2) are provided to carry the scanner. Hold the scanner at these two points with your left and right hands. The scanner weighs 11-13 kg and can be carried by one person. The open side of the scanner should face away from the body.

2.4 Installation example

The following figure is merely a schematic representation intended to provide an illustration of the overall system. Pay attention to an ergonomic arrangement as well as to the minimum distances for the scanner as shown (red).

No.	Description
1	Desk
2	Optical 3D scanner
3	Computer screen
4	Keyboard
5	Desk chair
6	Mouse
7	Computer

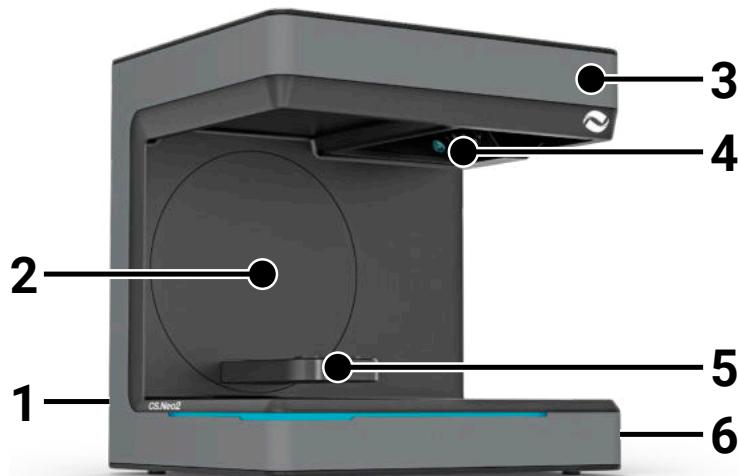
	The power supply unit may only be connected to a socket with protective contact
	The cables should be laid along walls

3. Device overview and connections

3.1 Components of the scanner

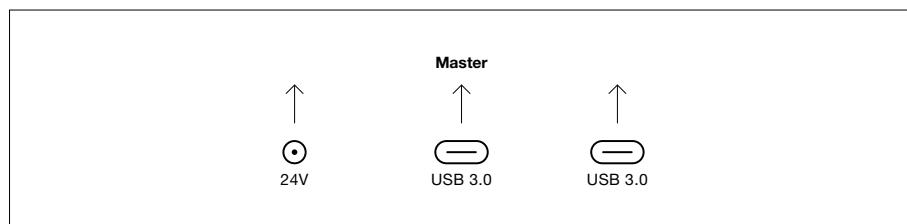
These are the main components in the scanner

No.	Description
1	Connector strip (rear side)
2	Swivel bracket
3	Housing
4	Optical scan unit
5	Model support (CS.Neo2 Max = two support)
6	On/Off button

3.2 Scanner connector strip

The connector strip is located on the back of the scanner and is designed as follows:

No.	Description
1	24V DC power supply socket
2	USB C 3.0 Port Master Unit
3	USB C 3.0 Port Slave Unit (CS.Neo2 Max only)

3.3 Power supply unit

Technical data of the power supply unit

	<table><thead><tr><th>Property</th><th>Value</th></tr></thead><tbody><tr><td>Manufacturer</td><td>MEAN WELL</td></tr><tr><td>Type</td><td>GST90A 24 – P1M</td></tr><tr><td>Input</td><td>100 - 240 V AC +/-10%</td></tr><tr><td></td><td>50 ~ 60Hz</td></tr><tr><td>Power-Out</td><td>90W</td></tr><tr><td>Output</td><td>24 V, 3.75 A</td></tr><tr><td>Max. Sea-Level</td><td>2000m</td></tr></tbody></table>	Property	Value	Manufacturer	MEAN WELL	Type	GST90A 24 – P1M	Input	100 - 240 V AC +/-10%		50 ~ 60Hz	Power-Out	90W	Output	24 V, 3.75 A	Max. Sea-Level	2000m
Property	Value																
Manufacturer	MEAN WELL																
Type	GST90A 24 – P1M																
Input	100 - 240 V AC +/-10%																
	50 ~ 60Hz																
Power-Out	90W																
Output	24 V, 3.75 A																
Max. Sea-Level	2000m																

Use only the supplied power supply and power cord. The power supply unit may only be connected to a socket with a protective contact.

3.4 USB cables

If possible, use the supplied cable. If you use your own USB cable, pay attention to the following properties. Full 5Gbit USB C cable with shielding maximum 1.4m. For extensions beyond 1.4m, an active USB extension must be used.

	For scanners with 2 USB cables, make sure that the full bandwidth is available for each of the two USB cables. (Often USB connectors share the speed)
---	---

3.5 Accessories

Each scanner is delivered with the standard accessories. Further accessories can be purchased for certain functions.

	NOTE! Only use original accessories. Otherwise, warranty provisions and product liability risks could be affected.
	Take care of the calibration plate so that the surface is not damaged.

4. Function and installation

This optical 3D scanner is designed for the dental field to measure and digitize jaw models in three dimensions. Based on the principle of "fringe light scanning", a fringe light pattern is projected by a projector onto an object to be measured. The distortion of the light pattern caused by the structure of the model is registered by four high-resolution cameras installed in the scanner unit, processed and thus digitized into a virtual and three-dimensional model.

4.1 Installing the scanner

After setting up the scanner, connect it with one or two USB cables, then connect the 24V supply to the external power grid and plug in the power supply.

If the USB cables are too short, they can be extended with special active extensions. Contact CADstar for appropriate products. Important note for devices with two USB connections, it must be ensured that the two USB ports on the PC do not belong to the same USB controller / HUB, since the scanner requires the full bandwidth of 2x 5Gbit/s. This means that a total of 10 GBit/s must be available.

4.2 Installing the computer

The scanner software requires at least Windows 10 or newer. Make sure that all drivers are up to date (graphics card, USB host, motherboard, etc...). Observe the recommended minimum PC workstation requirements to ensure smooth scanner operation.

Refer to the user manual of the PC.

4.3 Installing the software

The scan software is already pre-installed on the workstations supplied. If this is not the case, the software can be obtained from your dealer or installed remotely by our customer service. Contact CADstar Technology customer service to do so. See the chapter "Customer service."

Note that an internet connection must be available to license the software. This must be done for each new installation.

Administrator rights are required for the installation.

Our support works with "TeamViewer". Please download this software (www.teamviewer.com). After installation, please have your ID and password ready so that our support can connect to your computer.

5. Working process

After you have started the software, you must execute the scanner calibration under the Options menu. To do this, place the calibration plate in the scanner and start the calibration.

Homepage scan software

Options > Scanner calibration

OR Menu Options > Scanner calibration

After successful calibration, you can create a new case. Following fields are mandatory and need to be selected or filled out:

- Patient
- Doctor
- Model type
- Scan mode type

For further details, please refer to the software operation manual.

Once you are in the scan menu, you can scan a model that is in the scanner.

5.1 ID recognition

Each accessory has its own ID recognition by which the scanner can determine what is currently in the scanner and thus perform appropriate functions.

	Watch out for the exposed gold pins (24V). Do not short-circuit them!
--	---

5.2 LED indication bar

The LED bar is used to indicate states in the software.

- White pulsating > No software connection
- Red > Error
- Rainbow > Calibration
- Blue progress bar > Scan progress indicator

5.3 Switching off the scanner

	To switch off the scanner, press the On/Off button for at least 1 second.
--	---

6. FAQ Troubleshooting

Scanner does not start:

- If the scanner does not light up at startup, check the power supply and check whether the AC adapter is connected to the scanner.
- Contact our support

Scanner is not recognized on the PC:

- Be sure to use good USB cables with a maximum length of 1.5m (except if active USB extensions from CADstar are used). Exchange the cable to complete a test
- Check to ensure the USB port meets the USB3.0 specification
- Check the driver status in the device manager to ensure it is up to date
- Check if the LEDs on the device are on
- Restart the PC and the scanner
- Contact our support

Stripes are visible on the scan result:

- Watch out for changes in the light during a scan, as this can cause streaking. This is often caused by blinds that move during the scan or sunlight that changes due to trees or reflections on windows.
- Flickering light that can be emitted, for example, neon tubes or LED lighting. You can record a slow motion video with your smartphone to check this. If flickering light is visible, look for and eliminate the cause.

Holes are visible in the scan result:

- Use the "Integrate" tool for this area to close the hole.
- If there are dark spots on the object to be scanned, spray the object with our scan spray.

7. Cleaning and maintenance work for the operator

To ensure long product life and scanning accuracy, it is necessary to regularly clean and calibrate the device.

7.1 Cleaning

Before cleaning, the optical 3D scanner must be put out of operation. To do so, unplug the scanner. Use only dry cloths for cleaning. The scanner may be vacuumed or blown out. However, pay attention to the optical elements so that they do not become misaligned (camera focus). To prevent damage to the optical sensors located in the upper part of the scanner, they may only be dusted gently using a microfiber cloth.

NOTE!

The optical sensor system in the top part of the scanner is extremely sensitive.

DANGER!

Accidentally turning on the scanner during cleaning.

Do not use water, steam or cleaning agents for cleaning.

7.2 Recalibration

To guarantee a consistent scan result, it is recommended to use the supplied calibration plate to perform a calibration once weekly. The scanner should also be calibrated after transport or if it is moved to a different location. Instructions for recalibrating your scanner can be found in our software manual. Start the scanning process using software and proceed step by step according to the software instructions. If anything is unclear, contact our customer service department (see chapter "Customer service").

Make sure that the circles on the glass plate are not damaged, otherwise calibration is no longer possible.

Do not insert the calibration plate, as touching the circles may affect the accuracy of the calibration.

7.3 Repair / Troubleshooting

The optical 3D scanner contains sensitive components and sensor technology. Repair work may therefore only be carried out by CADstar Technology service personnel.

In case of malfunctions that cannot be solved by restarting the scanner, contact customer service (see the chapter "Customer service").

8. Disposal

The scanner can be returned to the dealer or manufacturer for disposal. Therefore, if you have any questions regarding proper disposal, please contact our customer service department.

Please note that the scanner is an electronic device intended for use in commercial/industrial environments only.

Disposal via public waste management authorities is therefore not permitted.

9. Technical data

General information

	CS.Neo2	CS.Neo2 Pro	CS.Neo2 Max
Manufacturer	CADstar Technology GmbH	CADstar Technology GmbH	CADstar Technology GmbH
Acceptance	2023	2023	2023
Weight	11.40 kg	11.50 kg	12.60 kg
Dimensions (W x H x D)	396 x 405 x 337 mm	396 x 405 x 337 mm	396 x 405 x 337 mm

Environment

	CS.Neo2	CS.Neo2 Pro	CS.Neo2 Max
Working temperature range	18 °C to 35 °C	18 °C to 35 °C	18 °C to 35 °C
Storage temperature range	-15 °C to 50 °C	-15 °C to 50 °C	-15 °C to 50 °C
Altitude	0 to 2000 m	0 to 2000 m	0 to 2000 m
Pollution level	2	2	2

Electrical systems

	CS.Neo2	CS.Neo2 Pro	CS.Neo2 Max
Mains voltage	100 - 240 V AC +10%	100 - 240 V AC +10%	100 - 240 V AC +10%
Mains frequency	50 - 60Hz	50 - 60Hz	50 - 60Hz
Overtoltage category	2	2	2
Power supply unit output	90 Wh	90 Wh	90 Wh
Scanner voltage	24V DC +10%	24V DC +10%	24V DC +10%
Power consumption operation	35 Wh	35 Wh	45 Wh
Power consumption stand by	1 Wh	1 Wh	1 Wh
Camera	2x 6Mpix	4x 6Mpix	2x 4x 6Mpix
Type of USB connection	USB 3.0 TYPE C (5Gbit/s)	USB 3.0 TYPE C (5Gbit/s)	USB 3.0 TYPE C (5Gbit/s)
Number of USB connections	1	1	2

Scanner

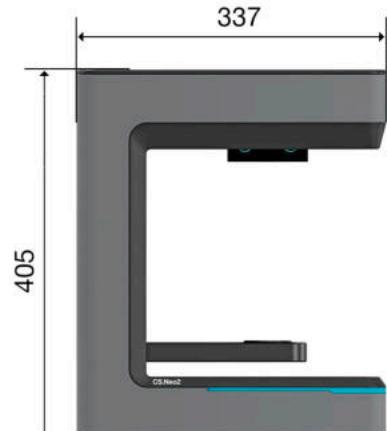
	CS.Neo2	CS.Neo2 Pro	CS.Neo2 Max
Measuring range (W x H x D)	100 x 100 x 100 mm	100 x 100 x 100 mm	2x 100x100x100mm
Maximum object size	200 x 200 x 150 mm	200 x 200 x 150 mm	300x200x150mm
Maximum object weight	2 kg	2 kg	2 kg
Absolute accuracy	<5 µ	<5 µ	<5 µ
Repeat accuracy	<2 µ	<2 µ	<2 µ
Output format	STL, PLY	STL, PLY	STL, PLY

9.1 Product diagrams & clearances

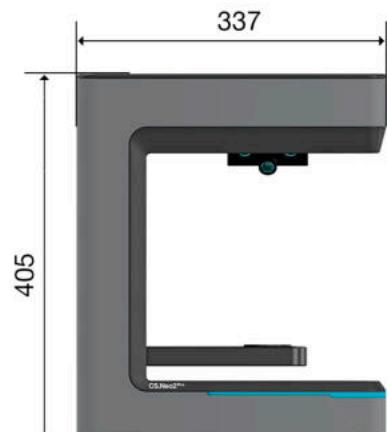
CS.Neo2

CS.Neo2

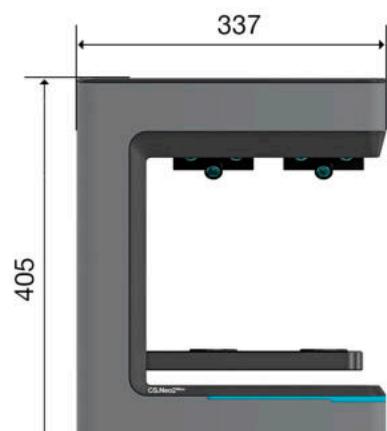
CS.Neo2 Pro

CS.Neo2 Pro

CS.Neo2 Max

CS.Neo2 Max

10. Declaration of Conformity

(Original EC Declaration of Conformity)

Manufacturer: CADstar Technology GmbH Grasslau 30 5500 Bischofshofen	Authorized signatory: Georg Dick (Managing Director)
Product: Optical 3D Scanner CS.Neo2, CS.Neo2 Pro, CS.Neo2 Max	Explanation: CADstar Technology GmbH hereby confirms that the product shown here complies with the basic requirements of EMC 2014/30/EU, Low Voltage 2014/35/EU and the regulations on the restriction of the use of certain hazardous substances in electrical and electronic equipment according to RoHS electronics 2011/65/EU.
Applicable standards:	
Audio/video, information and communication technology equipment	DIN EN IEC 62368-1:2021-05
Safety requirements for electrical equipment for measurement, control, and laboratory use.	IEC 61010-1:2010/AMD1:2016
Safety of machinery - General principles for design - Risk assessment and risk reduction.	DIN EN ISO 12100:2011-03
Electromagnetic compatibility. (EMC)	CISPR 11:2015 +AMD1:2016 +AMD2:2019 CISPR 11:2015 + AMD1:2016 + AMD2:2019 DIN EN IEC 61000-4-2: 2009-12 DIN EN IEC 61000 4-3:2021-11 DIN EN IEC 61000 4-4:2013-04 DIN EN IEC 61000-4-5:2019-03 DIN EN IEC 61000-4-6:2014-08 DIN EN IEC 61000-4-8:2010-11 DIN EN IEC 61000-4-11:2021-10
Registration, Evaluation, Authorisation and Restriction of Chemicals. (REACH)	RoHS 2011/65/EU RoHS 2015/863/EU REACH1907/2006/EC
Bischofshofen, January 2023	Gerog Dick
(Place and date of declaration)	(Name and signature of authorised signatory)

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Manufacturer: CADstar Technology GmbH Grasslau 30 5500 Bischofshofen	Authorized signatory: Georg Dick (Managing Director)
Product: Optical 3D Scanner CS.Neo2, CS.Neo2 Pro, CS.Neo2 Max	Explanation: CADstar Technology GmbH hereby confirms that the product shown here complies with the basic requirements of EMC 2014/30/EU, Low Voltage 2014/35/EU and the regulations on the restriction of the use of certain hazardous substances in electrical and electronic equipment according to RoHS electronics 2011/65/EU.
Applicable standards:	
Audio/video, information and communication technology equipment	DIN EN IEC 62368-1:2021-05
Safety requirements for electrical equipment for measurement, control, and laboratory use.	IEC 61010-1:2010/AMD1:2016
Safety of machinery - General principles for design - Risk assessment and risk reduction.	DIN EN ISO 12100:2011-03
Electromagnetic compatibility. (EMC)	CISPR 11:2015 +AMD1:2016 +AMD2:2019 CISPR 11:2015 + AMD1:2016 + AMD2:2019 DIN EN IEC 61000-4-2: 2009-12 DIN EN IEC 61000 4-3:2021-11 DIN EN IEC 61000 4-4:2013-04 DIN EN IEC 61000-4-5:2019-03 DIN EN IEC 61000-4-6:2014-08 DIN EN IEC 61000-4-8:2010-11 DIN EN IEC 61000-4-11:2021-10
Registration, Evaluation, Authorisation and Restriction of Chemicals. (REACH)	RoHS 2011/65/EU RoHS 2015/863/EU REACH1907/2006/EC
Bischofshofen, January 2023 (Place and date of declaration)	 Georg Dick (Name and signature of authorised signatory)

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