

Order information

Harvard SuperSnap®

Regular Snap

2 automix cartridges ea. 50 ml, 12 mixing tips 1:1, 4 IntraTips yellow
Light orange 7083801
Medium green 7083811

2 automix cartridges ea. 50 ml, 12 mixing tips 1:1
Mono blue 7083821
Heavy black-grey 7083831

2 jars ea. 600 g, 2 spoons
Putty black 7083841
Putty Soft black 7083851

380 ml maximix cartridge, 10 dynamic mixers, 1 bayonet ring
Putty Soft black 7083843
Heavy black 7083833

Quick Snap

2 automix cartridges ea. 50 ml, 12 mixing tips 1:1, 4 IntraTips yellow
Light orange 7083802
Medium green 7083812

2 automix cartridges ea. 50 ml, 12 mixing tips 1:1
Mono blue 7083822
Heavy black-grey 7083832

2 jars ea. 600 g, 2 spoons
Putty black 7083842
Putty Soft black 7083852

Mixing Tips und Zubehör

Harvard Auto-T 1:1 O-Yellow 7098000
 Refill bag with 50 Mischkanülen for
 Light, Medium – Regular / Quick

Harvard Auto-T 1:1 O-Green 7098010
 Refill bag with 50 Mischkanülen for
 Heavy – Regular / Quick

Harvard Auto-T 1:1 O-Pink 7098020
 Refill bag with 50 Mischkanülen for
 Mono – Regular / Quick

Harvard IntraTips yellow 7083619
 Refill bag with 50 intra tips for
 Light, Medium – Regular / Quick

Harvard Maxi 5:1 Dynamic 7091400
 Refill bag with 50 dynamic mixers and
 2 bayonet rings for Putty Soft MaxiMix
 Heavy – Regular MaxiMix

Harvard Dispenser Automix 1:1 7095100

Harvard TraySive 7083753
 10 ml bottle



1892
 seit
HARVARD®

 **HARVARD®**

Marke und Qualität seit 1892

Harvard Dental International GmbH
 Margaretenstr. 2 - 4, 15366 Hoppegarten, Germany
 Phone: + 49 (0) 30/99 28 978-0
 Fax: + 49 (0) 30/99 28 978-19
 info@harvard-dental-international.de

Liability is excluded for all printing errors and omissions. Before using our Harvard products, the respective directions for use should be noticed in every case. All measurements are internal measurements of Harvard Dental International.

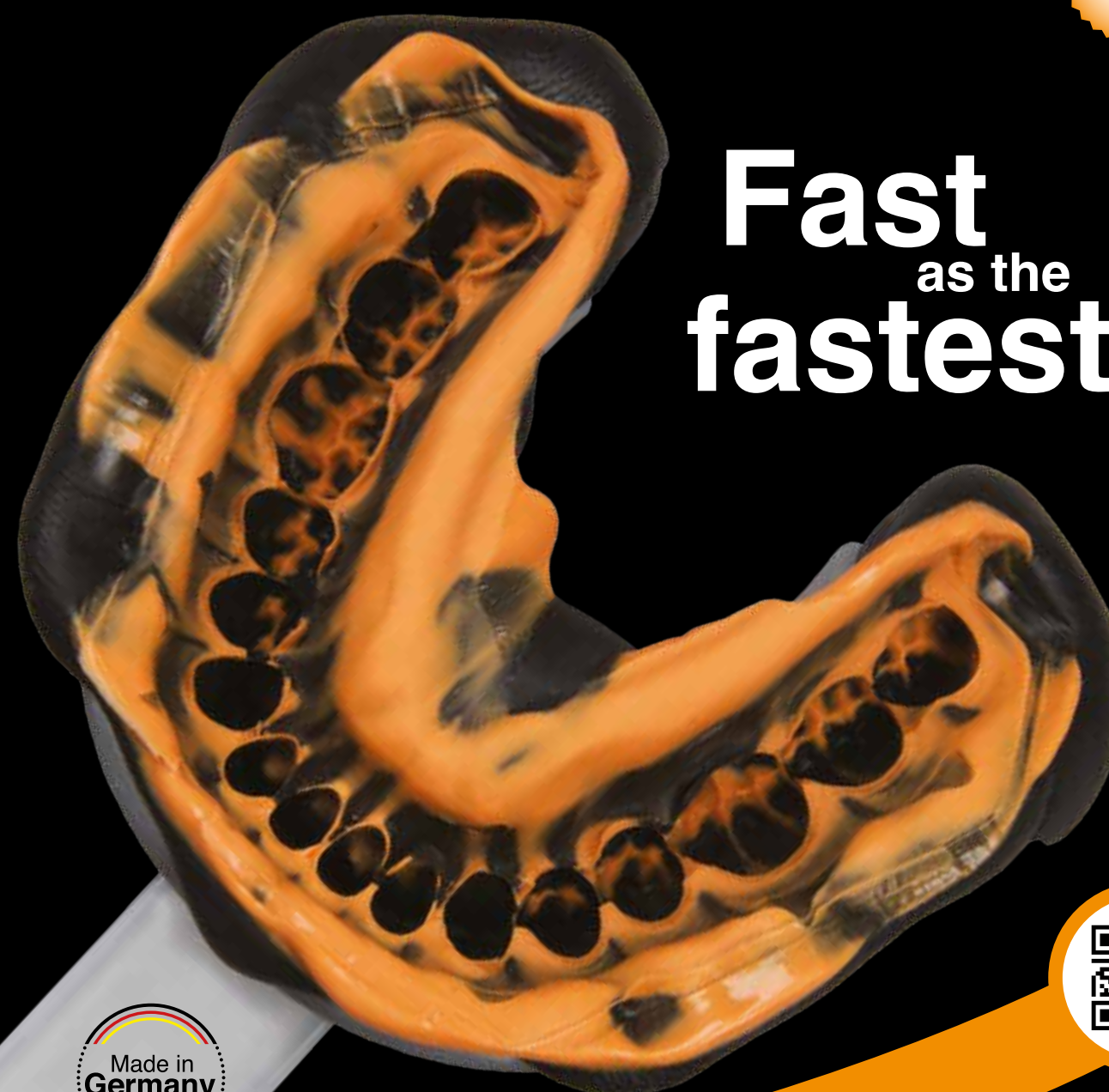
Harvard Distribution Partner.

Harvard SuperSnap®

Vinylpolysiloxane impression material

NEW

Fast
 as the
fastest!



Made in
 Germany



EN

www.harvard-dental.de

Impression material of a new Generation...

...that is Harvard SuperSnap®. This product combines all the advantages of our previous Premium-Sil products with new setting properties.

You are looking for a combination of a long working time and a short oral setting-time

- This allows a stress-free workflow, time savings and increases patient comfort.

You want to use two different types of setting times depending on the clinical situation.

- Available in 2 snap-set variants: **Regular Snap** and **Quick Snap**

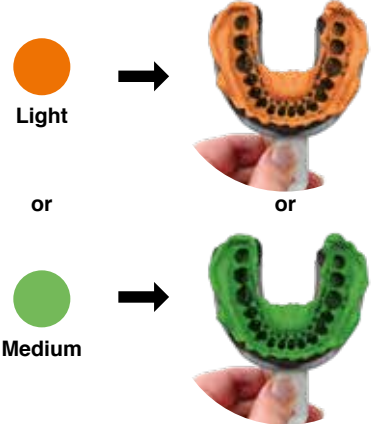
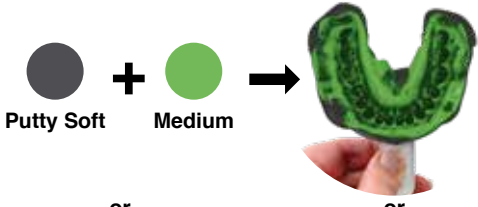
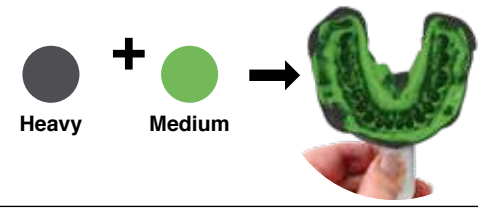
You want an exact readability.

- Harvard SuperSnap® offers new high-contrast colors for exact readability.

Harvard SuperSnap® offers as usual:

- Excellent flow properties
- High elastic recovery and dimension stability
- Excellent legibility
- Super hydrophilicity for perfect results and excellent reproduction of details
- As usual: very berry!
- Suitable for the sandwich technique and the putty-wash technique
- Different viscosities: Putty, Putty Soft, Heavy, Mono, Medium, Light
- Optimal kneadable consistency for the putty products

The recommended combinations of Harvard SuperSnap®:

Technique	Phase 1	Phase 2
2 steps / 2 phases putty-wash technique (Material application successively)		
1 step / 2 phases sandwich technique (Material application simultaneously)	 or 	
1 step / 1 phase Monophasetechnique		

Numbers that speak for themselves!

Product	Working time 23°C / 73°F (min: sec.)		Oral setting time (min: sec.)	
	Quick Snap	Regular Snap	Quick Snap	Regular Snap
Harvard SuperSnap® Light	1:10	1:45	1:15	1:45
Harvard SuperSnap® Medium	1:10	1:45	1:15	1:45
Harvard SuperSnap® Mono	1:10	1:45	1:15	1:45
Harvard SuperSnap® Heavy	1:10	1:45	1:15	1:45
Harvard SuperSnap® Heavy MaxiMix	1:10	1:45	1:15	1:45
Harvard SuperSnap® Putty Soft MaxiMix	-	1:45	-	1:45
Harvard SuperSnap® Putty Handmix	1:10	1:45	1:15	1:45
Harvard SuperSnap® Putty Soft Handmix	1:10	1:45	1:15	1:45
	Fast Set	Normal Set	Fast Set	Normal Set
Harvard PremiumSil (Light, Medium, Mono, Heavy)	1:10	1:45	1:30	2:30
Harvard PremiumSil (Putty, Putty Soft)	1:00	1:20	1:45	2:30
Harvard PremiumSil (Putty Soft MaxiMix)	-	1:20	-	2:30
	Super Quick	Regular Set	Super Quick	Regular Set
3M Imprint 4 Ultra Light	1:15	-	1:15	-
3M Imprint 4 Light	1:15	2:00	1:15	2:00
3M Imprint 4 Regular	1:15	2:00	1:15	2:00
3M Imprint 4 Penta Heavy	1:15	2:00	1:15	2:00
3M Imprint 4 Penta Putty	-	1:30	-	2:30

