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CEKA
PRECİ-LINE

Copyright© ALPHADENT NV 03/2021

Crystalline powder to improve light reflection and aesthetics avoiding yellow-green discolouration during the production of artificial teeth.

Field of application: All non-precious base metals with an aesthetic application area.

INSTRUCTIONS FOR USE

Metal preparation

Sandblast with aluminium oxide. Clean in an ultrasonic bath or by boiling for 10 minutes in distilled water, or steam clean.

Degas according to the instructions of the used alloy, if applicable.

The metal frame must be completely dry and should only be touched with clean tweezers.

Mix 3-CP with distilled water until a smooth paste is obtained.

Apply in a thin layer. Do not cover marginal areas. Leave a space of minimum 2 mm.

Never re-use already mixed 3-CP.
Firing instructions: see table below.

SAFETY INSTRUCTIONS

Skin contact (in case of allergy)
Wash with water and soap.

STORAGE

3-CP must be stored in a well-sealed package in a dry place.

The product should not be used if the package is damaged.

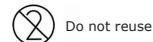
For the purpose of traceability we advise you to record the lot number of the applied products.

FIRING TABLE


Always fire 3-CP under vacuum.

Preheating temperature	650 °C (1200 °F)
Temperature rise	55 °C (131 °F) per minute
Final temperature	980 °C (1800 °F)

After firing, immediately remove the metal frame from the furnace and bench cool.


Do not reuse

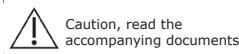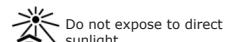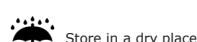
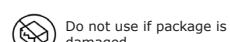
Caution, read the accompanying documents

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