making work easy

- de
- en
- fr
- it
- es
- pt
- tr
- ru
- pl
- zh
- ja
- ko
- ar

# BASIC prebonder

Made in Germany

20-5202 22012026

Renfert

making work easy

# BASIC prebonder

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Made in Germany

# Inhalt

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making work easy

# BASIC prebender

TRANSLATION OF THE ORIGINAL INSTRUCTIONS FOR USE

Made in Germany

20-5202 22012026

# Contents

en

1	Online help and information	3
2	Introduction	3
2.1	Employed Symbols	3
3	Safety	4
3.1	Intended use	4
3.2	Meaning of these instructions	4
3.3	Requirements that the operator must meet	4
3.4	Safe condition of the unit	4
3.5	Risks as a result of modifications and repairs	4
3.6	Risk as a result of electric shock and fire	4
3.7	Risk of injury to the eyes or skin	4
3.8	Risk to health as a result of dust	4
4	Product description	5
4.1	General description	5
4.2	Components and functional elements	5
4.3	Standard Delivery	6
4.4	Accessories	6
5	Commissioning	7
5.1	Installation	7
5.2	Foot switch connection	7
5.3	Compressed air connection	7
5.4	Electrical connection	7
5.5	Connection to suction device	7
5.6	Filling the Sandblasting Tanks	8
5.6.1	Authorised sandblasting material	8
5.7	Adjusting the Operating Pressure	8
5.8	Air nozzle	9
6	Operation	9
6.1	Prebonder blasting concept	9
6.1.1	Control tip	9
6.1.2	Jet nozzle	9
6.1.3	Function check with the control pad	9
6.2	Blasting Notes	9
6.3	Cold silanization in the sandblasting unit	10
7	Cleaning / Maintenance	10
7.1	Sandblasting tank	10
7.2	Cleaning the Interior	10
7.3	Condensation	10
7.4	Blasting Nozzle Replacement	11
7.5	Cleaning the dosing nozzle	11
7.6	Cleaning / replacing the filter cartridge	11
7.7	Lamp cover	11
7.8	Spare Parts	11
7.9	Shipping Notes	11
8	Trouble shooting	12
9	Technical Specifications	13
10	Warranty	14
10.1	Liability Exclusion	14
11	Obligations regarding information	14
12	Information on disposal for countries within the EC	14

# 1 Online help and information

<https://www.renfert.com/basic-prebonder>

en

Here you will find the following documents and information for download:

- Operating Instructions
- Quick Start Guide
- Spare parts lists and spare parts drawings
- Maintenance manual
- FAQs, cause and rectification of faults
- How to - Videos
- Declarations of conformity

Please note: The documents and information listed are not available for all products.

## 2 Introduction

### 2.1 Employed Symbols

In these instructions or on the unit itself you will find symbols which have the following meaning:

#### **Danger**

This indicates an imminent risk of injury. In these instructions or on the unit itself you will find symbols which have the following meaning.

#### **Electrical hazard**

There is a risk of electrical shock.

#### **Attention**

Failure to observe the associated information can result in damage to the unit.

#### **Note**

*The operating instructions contain useful information to make handling easier.*

#### **Attention!**

Wear eye protection.

The unit complies with the relevant EC guidelines.

This product complies with the relevant UK legislation.

See UKCA Declaration of Conformity in the Internet under [www.renfert.com](http://www.renfert.com).

The unit is subject to the EC guidelines 2002/96/EG (WEEE Directive).

► List, particular attention should be paid

- List
- subordinate list

⇒ Instructions / required action / input / operational sequence:

You are asked to carry out the specified action in the order indicated.

◆ Result of an action / response by the equipment / response by the program:

The equipment or program responds as a result of your actions or when a specific incident occurs.

Other symbols are explained as they occur.

# 3 Safety

en

---

## 3.1 **Intended use**

---

The Basic sandblaster is intended for commercial use in dental applications in dental offices and in-office labs in accordance with the Instructions For Use (IFU). The unit is intended for use in the following tasks:

- Prebonder tank:  
Surface conditioning prior to inserting fixed dental restorations in the mouth – suitable for zirconium dioxide / metals / hybrid materials (ceramic / resin) and polymers.
- Classic tank (25–70 µm / 70–250 µm):  
All dental activities that have been carried out in the dental office so far:
  - Removal of dental cement residues
  - Cleaning dentures
  - Satin finishing
  - All classic dental activities

---

## 3.2 **Meaning of these instructions**

---

These instructions for use act as a guide for safe usage of the unit. If the instructions for use are not observed, accidents may lead to life-threatening electric shock, injury, or damage.

- ⇒ Only use the unit in accordance with these instructions for use.
- ⇒ Keep the instructions for use ready with the unit.
- ⇒ Provide the instructions for use to all subsequent users of the unit.

---

## 3.3 **Requirements that the operator must meet**

---

- ⇒ This product is intended for use by persons 14 years of age or older who are familiar with the procedures and regulations used in an in-office lab or any other dental lab.

---

## 3.4 **Safe condition of the unit**

---

**Faulty or broken components, or those that are not sealed as required, may cause injuries.**

- ⇒ Check the power cord, housing, and other components such as connection cables, tubes, and the key pad for damage, for example such as kinks, cracks, porosity, and signs of aging.
- ⇒ Defective equipment must be taken out of service immediately. Disconnect the power plug. Send the unit in for repair or dispose of it safely. Until this happens, secure the unit to prevent it from being switched on again so that it is not used inadvertently.
- ⇒ Do not expose the unit to mechanical impact. Do not drop it.

---

## 3.5 **Risks as a result of modifications and repairs**

---

**Incorrect modifications and repairs to the unit may cause accidents, fire, and electric shock.**

- ⇒ Only have the unit repaired by qualified specialist dealers or return it.

---

## 3.6 **Risk as a result of electric shock and fire**

---

**Risk of electric shock and of fire as a result of excessive operating voltage or if there is no protective conductor.**

- ⇒ Ensure that the power plug is easily accessible.
- ⇒ Only operate the unit using a power cord equipped with a plug appropriate to the local power supply.  
Only allow qualified electricians to modify power cords.
- ⇒ Only start the unit if the information on the nameplate matches the power supply.
- ⇒ Only connect the unit to electrical sockets that are connected to the protective conductor system.

---

## 3.7 **Risk of injury to the eyes or skin**

---

**Injury may occur to the eyes or skin as a result of high pressure or of components fracturing that are subject to high pressure.**

- ⇒ Always wear appropriate protective goggles when carrying out work on the unit.
- ⇒ Never direct the blasting material towards the eyes or unprotected areas of skin.
- ⇒ Never perform sandblasting with the viewing panel open.

---

## 3.8 **Risk to health as a result of dust**

---

**Dust emissions may result in damage to health.**

- ⇒ Operate the unit with a suitable extraction unit. The extraction unit must be adapted to the dust that occurs.

# 4 Product description

## 4.1 General description

The BASIC prebonder sandblaster from Renfert features a specially developed concept for blasting before bonding. The blasting parameters and the surface pro abrasive have been precisely tailored for surface conditioning of fixed dental restorations based on scientific studies conducted by Ludwig Maximilian University (LMU) of Munich in Germany. The unit guarantees maximum and reliable adhesive strength for restoration materials, such as zirconium oxide, metals, hybrid materials (ceramic/resin), and modern polymers. It is therefore ideal for preparing materials prior to final insertion into the mouth.

The BASIC prebonder is supplied as a 2-Tank version. With additional tanks (see accessories) it can be upgraded to a 3- or 4-Tank version.

The BASIC prebonder has an automatic quick stop function and is therefore especially suitable for exact and selective use of the sandblasting material.

## 4.2 Components and functional elements

- |                                    |                               |                               |
|------------------------------------|-------------------------------|-------------------------------|
| 1 Pressure regulator               | 9 Selector switch             | 19 Foot switch connection (B) |
| 2 Appliance socket                 | 10 Protection screen          | 20 Mixing chamber             |
| 3 ON / OFF switch                  | 11 Hand piece                 | 21 Suction tube               |
| 4 Tank lid                         | 12 Hand sleeves               | 22 Filter cartridge           |
| 5 Ventilation valve                | 13 Air nozzle                 | 23 Sandblasting tank          |
| 6 Pressure gauge (Flow pressure)   | 14 Base grid                  | 24 Releasing tool             |
| 7 Pressure regulator (fixed)       | 15 Air connection             | 25 Foot switch                |
| 8 Pressure gauge (Outlet pressure) | 16 Water trap                 | 26 Compressed air tube        |
|                                    | 17 Water trap valve           | 27 Power cord                 |
|                                    | 18 Foot switch connection (A) |                               |

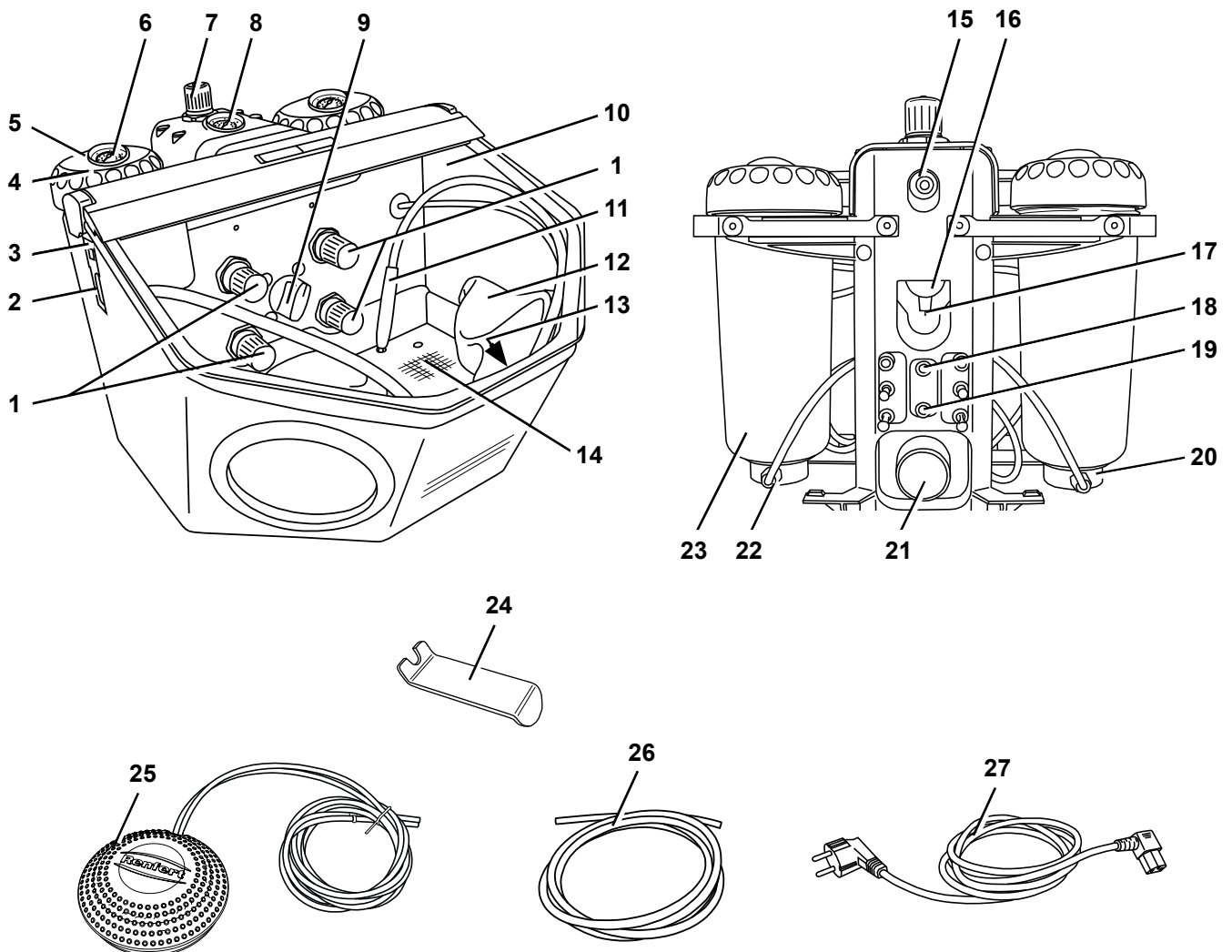
Fig. 1

## 4.3 Standard Delivery

- 1 BASIC prebonder
- 1 Foot switch
- 1 Connection set
- 1 Prebonder control pad 50 µm
- 5 Prebonder control tip (1 + 4)
- 1 Prebonder surface pro 50 µm
- 1 Set of Operating Instructions, with attachment
- 1 Power cord

## 4.4 Accessories

### Retrofit tanks

- 2959-0050 Basic quattro IS Retrofit tank 25 - 70 µm
- 2959-0250 Basic quattro IS Retrofit tank 70 - 250 µm

### Abrasives

#### Prebonder surface pro

- 1597-0050 50 µm, white 2 kg, 4x 500 g bags

#### Cobra aluminium oxide

- 1594-1105 25 µm, white 5 kg canister
- 1594-1205 50 µm, white 5 kg canister
- 1594-2220 50 µm, white 20 kg bucket
- 1584-1005 90 µm, white 5 kg canister
- 1583-1005 110 µm, white 5 kg canister
- 1583-1020 110 µm, white 20 kg bucket
- 1587-1005 125 µm, pink 5 kg canister
- 1587-1020 125 µm, pink 20 kg bucket
- 1585-1005 250 µm, white 5 kg canister
- 1585-1020 250 µm, white 20 kg bucket

#### Rolloblast glass beads

- 1594-1305 50 µm 5 kg canister
- 1594-2312 50 µm 12.5 kg canister
- 1589-1005 100 µm 5 kg canister

### Nozzles

#### Round nozzles

- 90003-8070 Prebonder jet-nozzle
- 90002-1203 0.4 mm [0.0158 inch]
- 90003-3213 0.65 mm [0.0256 inch]
- 90002-1204 0.8 mm [0.0315 inch]
- 90003-3211 1.0 mm [0.0394 inch]
- 90003-3214 1.2 mm [0.0473 inch]
- 90002-1205 1.4 mm [0.0552 inch]
- 90002-1206 2.0 mm [0.0788 inch]

#### Slotted nozzles

- 90003-1739 1.5 x 3,5 mm [0.0591 x 0.1379 inch]
- 90003-0375 Prebonder control pad 50 µm
- 90003-9010 Prebonder control tip

# 5 Commissioning

**Risk of injury!**

**Loose tubes may hit objects around them.**

**Tubes must be connected first to the unit.**

**Only press the foot switch after all tubes have been connected.**

## 5.1 Installation

Choose a stable and non-slip surface to stand the unit on.

## 5.2 Foot switch connection

⇒ Remove the foot switch from its shipping location in the blasting chamber.

⇒ Insert the end of the tube marked with an „A“ (25) as far as it will go into the tube connection point also marked with an „A“ (18) on the blasting unit.

⇒ Insert the end of the tube marked with an „B“ (25) as far as it will go into the tube connection point also marked with an „B“ (19) on the blasting unit.

When inserting the tubes you will notice a slight resistance which must be overcome.

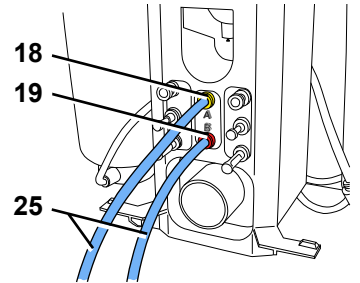
Fig. 2

## 5.3 Compressed air connection

⇒ Push the compressed air tube (26) onto the tube connection point (15) as far as it will go. In doing so there will be a slight resistance to overcome.

⇒ To connect to the compressed air system, select the relevant tube coupling from the supplied set and attach this to the other end of the tube.

⇒ Connect to the compressed air network.

⇒ Check whether the pressure is at the required level (see tech. data). Display will be shown on the manometer (8, Fig. 1).

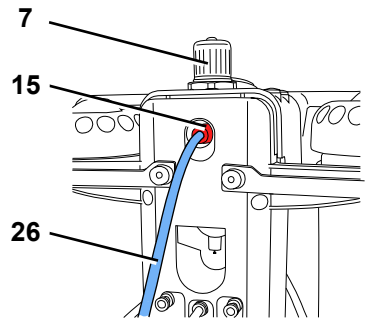
Fig. 3

**Never press the pressure regulator (7) on the maintenance unit. This has been set and fixed to max. 6 bar in order to guarantee the best possible function of the pinch valves.**

## 5.4 Electrical connection

**Note the instructions in Chap. 3.6 Risk as a result of electric shock and fire!**

⇒ Plug the mains cable (27) into the socket (2).

⇒ Plug in the mains cable.

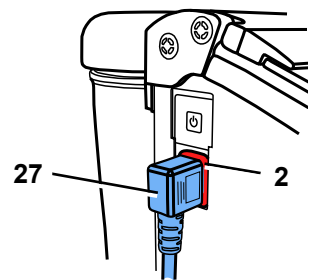
Fig. 4

## 5.5 Connection to suction device

⇒ Plug the suction tube from the extraction unit to the suction pipe (21).

▶ For suction devices with automatic activation:

⇒ Connect the mains plug from the sandblasting unit into the user socket on the suction device.

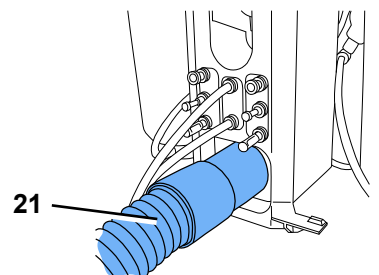
Fig. 5

## 5.6 Filling the Sandblasting Tanks

Do not write on or stick anything to the sandblasting tank and tank lid.

Do not press the foot switch while filling is in progress!

**When using Prebonder tank abrasives, only fill in and use a quantity of 500 g at a time.**

- ⇒ Ventilate the blasting tank completely by pressing the ventilation valve (5).
- ⇒ Unscrew the tank lid (4). (If it is not possible to loosen the lid, there is still pressure within the blasting tank >>> Press the ventilation valve again!)
- ⇒ Add abrasive up to the maximum fill level (Fig. 7) (the point where the tank jacket thickens just below the thread).
- ⇒ Clean the thread on the sandblasting tank and tank lid as well as the seal.
- ⇒ Screw the tank lid closed firmly by hand.

**Abrasive residue on the seal causes leakage and premature wear of the seal.**

**Warning: Do not tilt the tank lid when screwing it on.**

**Check that the tank lid is seated securely prior to use. Tank lids that are not closed securely can suddenly blow off. There is a risk of injury as a result of flying parts and sudden emission of abrasive sand.**

- ⇒ Always use only clean, dry abrasive of the appropriate grain size (refer to the accessories).

**The sandblasting unit is now operational.**

### 5.6.1 Authorised sandblasting material

- Prebonder surface pro
- Cobra (Renfert GmbH)
- Rolloblast (Renfert GmbH)
- ROCATEC (3M ESPE)

Sandblasting materials from other manufacturers may be used providing the grain size, grain shape and degree of purity comply with the above mentioned materials.

The “Prebonder surface pro” abrasive has undergone comprehensive scientific testing at LMU Munich. These tests covered grain size distribution, geometric shape, hardness, and degree of purity.

Despite having the same technical parameters, many of the commercially available abrasives exhibit significant differences in these properties. In contrast, Prebonder surface pro is a specially developed solution that stands out due to its consistently high quality.

The company Renfert GmbH assumes no liability for function and durability of the units if other sandblasting material is used.

## 5.7 Adjusting the Operating Pressure

**The Prebonder tank is equipped with a very sensitive pressure gauge to permit a more precise pressure adjustment. Ensure that the pressure gauge is within the green area (0.6 – 1 bar).**

- ⇒ Activate the foot switch and set the blasting pressure at the pressure regulator (1) of the respective blasting tank.
  - ◆ The blasting pressure will be shown on the pressure gauge (6).

During longer downtimes, the pressure within the blasting tanks will reduce. By short activation of the foot switch the set pressure within in the blasting tanks will be restored.

**Only blast restorations when the indicator is in the green area.**

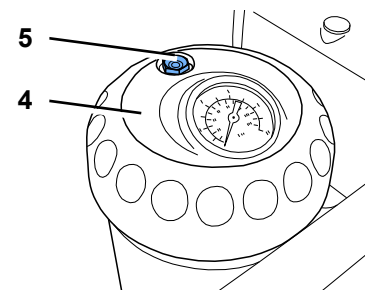
Fig. 6

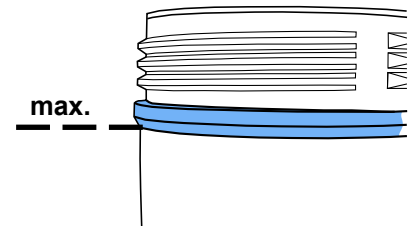
Fig. 7

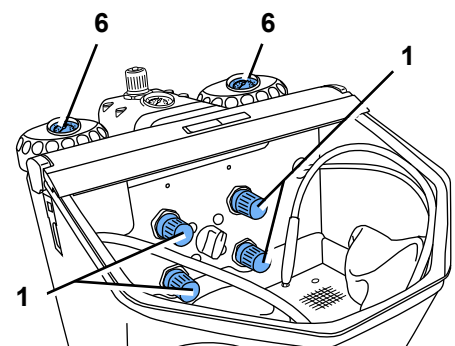
Fig. 8

## 5.8 Air nozzle

The air nozzle (13, Fig. 1) can be used to remove sandblasting residues and dust.

# 6 Operation

**If the membrane key is damaged or defect, please disconnect the device and send away for repair! If the membrane is removed there is the risk of an electrical shock!**

**Do not operate the foot switch during tank selection.**

- ⇒ Switch the illumination on (3).
- ⇒ Select the blasting tank at the selection switch (9, Fig. 1).
- ⇒ Take the respective hand piece from the selected blasting tank (see coloured marking).
- ⇒ Activate the foot switch (25, Fig. 1).
  - ◆ The sandblasting material will flow as long as the foot switch is activated.

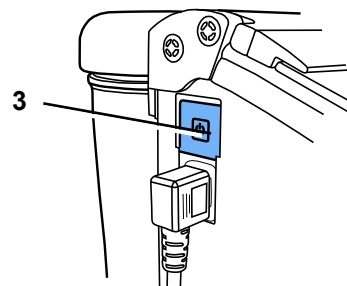
Fig. 9

## 6.1 Prebonder blasting concept

### 6.1.1 Control tip

The Prebonder handpiece features a specially designed ergonomic handle and an integrated control tip. This ensures the following advantages:

- It helps to keep the optimum distance from the surface during the blasting process.
 

The distance has a significant impact on the energy input:

  - If the energy input is too high, it may damage the material structure and/or reduce the surface's wettability.
  - If the energy input is too low, surface activation will be insufficient, which will in turn reduce the adhesive strength.

Fig. 10

Fig. 11

- The precise blasting indicator permits to see exactly where the abrasive is applied. This helps to protect sensitive areas, such as crown margins or already polished surfaces, and to avoid unintentional sandblasting.

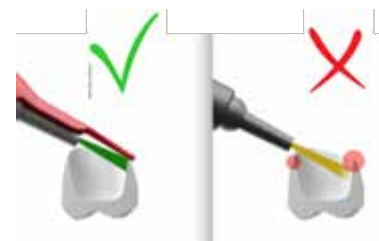
Fig. 12

### 6.1.2 Jet nozzle

Unlike conventional nozzles, which are designed primarily for maximum material removal, the prebonder jet nozzle has been optimized specifically for the requirements of adhesive surface pretreatment.

Thanks to its unique geometric design, the focus is on targeted transformation of the surface for optimal adhesion, rather than removal of material.

### 6.1.3 Function check with the control pad

We recommend using the enclosed control pad to ensure the correct setting of blasting energy before sandblasting the dental restoration.

If the color of the pad changes after 1-2 seconds of sandblasting, then the energy level is set optimally for surface conditioning.

## 6.2 Blasting Notes

Flat blasting with wide nozzles:

90003-1739 1.5 x 3.5 mm [0.059 x 0.138 inch]

When blasting glass ceramics, refer to manufacturer's notes.

## 6.3 Cold silanization in the sandblasting unit

The Basic series of Renfert sandblasting units is exclusively recommended by 3M ESPE for use with ROCATEC™. This bonding system is ideal if an adhesive bond is required between a composite and a dental material (metal, acrylic, porcelain, zirconia porcelain).

### Advantage:

The surface film is applied cold, preventing thermal loading of metal frameworks and consequently the risk of distortion. All units in the Basic series can be used for ROCATEC™ due to their regular, homogeneous jet flow.

## 7 Cleaning / Maintenance

**Before cleaning or servicing:**

- ▶ Switch off and unplug the device!
- ▶ Disconnect the appliance from the compressed air supply!

### 7.1 Sandblasting tank

**Check the sandblasting tank or tank lid regularly, at least 1 x a year, by visually inspecting for damage or changes to the material (e.g., cracks ...).**

**If in doubt, replace!**

**The sandblasting tank and tank lid are subject to a natural aging process depending on ambient conditions and must be replaced as standard after every 15 years of operation.**

**Note the year of manufacture on the round stamp engraved on the sandblasting tank and in the tank lid; Our customer service team are happy to advise you.**

Example of an engraved round stamp:  
Date of manufacture 01-2003

**To clean:**

**DO NOT use solvent-based cleaning agents or disinfectants.**

**Solvents and tensides can result in microcracking in the plastic (danger of explosion!).**

⇒ Only clean the sandblasting tank and tank lid by rubbing with a dry cloth.

Fig. 13

### 7.2 Cleaning the Interior

- ▶ DO NOT use solvent-containing cleaning agents or disinfectants. (use e.g., soapy water).
- ⇒ Remove the grate and vacuum the blasting chamber.

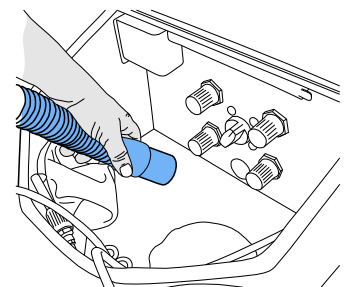
Fig. 14

### 7.3 Condensation

⇒ Inspect the water separator (16).

⇒ Drain any water off through the valve (17).

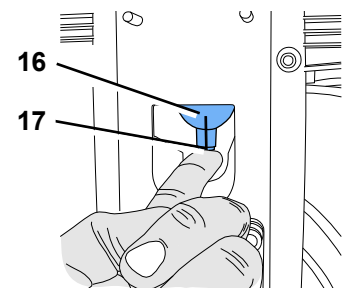
Fig. 15

### 7.4 Blasting Nozzle Replacement

For installation see accessory supplement.

### 7.5 Cleaning the dosing nozzle

The dosing nozzle from the mixing chamber can become blocked due to contamination and moist blasting sand.

For cleaning, the dosing nozzle is removed using the supplied tool and replaced after cleaning.

- ⇒ Vacuum clean the blasting tank completely.
- ⇒ Hold the dosing nozzle with the tool and pull upwards (Fig. 16).
- ⇒ Clean the dosing nozzle (e.g. blow with compressed air).
- ⇒ Replace dosing nozzle.

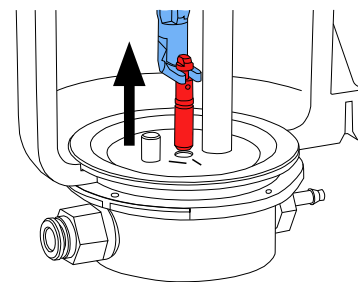
Fig. 16

**Before remounting, the dosing nozzle should be completely dry!**

The dosing nozzle must be adjusted with blasting tanks of 25 - 70 µm (Fig. 17)!

**See also supplement „Cleaning the dosing nozzle“.**

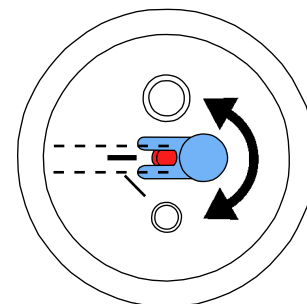
Fig. 17

### 7.6 Cleaning / replacing the filter cartridge

- ⇒ Disconnect the unit from the compressed air supply.
- ⇒ Disconnect the compressed air hose from the tank.
- ⇒ Unscrew the filter cartridge (spanner width SW14).
- ⇒ Blow out or replace the filter cartridge.
- ⇒ Screw in the filter cartridge and reconnect the compressed air hose.

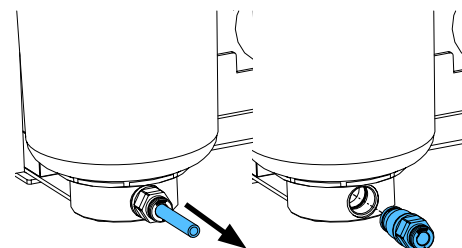
Fig. 18

### 7.7 Lamp cover

In the course of time, the lamp cover can become matt. This cover can be ordered as a spare part.

### 7.8 Spare Parts

**Only use accessories and spare parts that have been supplied or approved by Renfert GmbH. Using other accessories or other spare parts can lead to inadvertent accidents and damage.**

You can find components subject to wear and the spare parts on the spare part list in the internet at [www.renfert.com/p918](http://www.renfert.com/p918).

The components excluded from the warranty (such as consumables or parts subject to wear and tear) are marked on the spare part list.

Serial number and date of manufacturing are shown on the type plate of the unit.

### 7.9 Shipping Notes

- Empty all blasting tanks.
- Order Renfert transport packaging (Tel. +49 7731 8208-383).

The return form can be downloaded from the „Advice“ section on the Renfert website at [www.renfert.com](http://www.renfert.com).

# 8 Trouble shooting

Problem	Possible cause	Corrective action
<b>Prebonder – function check:</b> <b>No color change after 4 seconds of blasting on the control pad.</b>	<ul style="list-style-type: none"> <li>Operating pressure too low.</li> </ul>	<ul style="list-style-type: none"> <li>Check the pressure gauge indicator and adjust the setting, if necessary.</li> </ul>
	<ul style="list-style-type: none"> <li>Clogged funnel in the abrasive tank after prolonged periods of non-use.</li> </ul>	<ul style="list-style-type: none"> <li>Carefully tap the abrasive tank to loosen any blockages caused by a clogged funnel.</li> </ul>
	<ul style="list-style-type: none"> <li>Clogged dosing nozzle.</li> </ul>	<ul style="list-style-type: none"> <li>Clean the dosing nozzle in the abrasive tank (see chapter „7.5 Cleaning the dosing nozzle“)</li> <li>Short-term measure:               <ul style="list-style-type: none"> <li>Open the tank lid.</li> <li>Grasp the blue tube.</li> <li>Press the foot switch.</li> </ul>               This will loosen any contamination from the dosing nozzle, allowing the sandblasting process to continue.             </li> <li>CAUTION: Since the tank is still contaminated, we recommend vacuuming out the abrasive material and replacing it with fresh material, as well as cleaning the dosing nozzle.</li> </ul>
<b>No air and no abrasive.</b>	<ul style="list-style-type: none"> <li>Blasting nozzle blocked at the hand-piece.</li> </ul>	<ul style="list-style-type: none"> <li>Unscrew the nozzle from the hand-piece and blow it out.</li> <li>With the nozzle removed, blow out the blasting hose by activating the foot pedal.</li> <li>Inspect / correct the connection lines.</li> <li>Turn the switch until you feel it click into place.</li> </ul>
	<ul style="list-style-type: none"> <li>Foot switch connecting lines kinked.</li> <li>Tank selection switch in an intermediate position.</li> </ul>	
<b>Only air, no or very little abrasive.</b>	<ul style="list-style-type: none"> <li>The dosage nozzle in the mixing chamber is blocked due to contaminated or moist sand.</li> </ul>	<ul style="list-style-type: none"> <li>Cleaning the dosing nozzle, see chapter „7.5 Cleaning the dosing nozzle“.</li> </ul>
<b>Only air, no or very little abrasive.</b>	<ul style="list-style-type: none"> <li>Mixing chamber filter combination blocked.</li> <li>Abrasive contains impurities or is moist.</li> </ul>	<ul style="list-style-type: none"> <li>Change filter cartridge (22 / Fig. 1) (wrench SW14).</li> <li>Replace the abrasive.</li> </ul>
<b>Unit blasts continuously.</b>	<ul style="list-style-type: none"> <li>Foot switch connected incorrectly.</li> <li>Valve in foot switch is defective.</li> </ul>	<ul style="list-style-type: none"> <li>Inspect / correct the foot switch connection.</li> <li>Replace foot switch.</li> </ul>
<b>Lamp is not alight.</b>	<ul style="list-style-type: none"> <li>Power cord not plugged in.</li> <li>Illumination defect.</li> </ul>	<ul style="list-style-type: none"> <li>Plug the power cord into the wall outlet.</li> <li>Change light using the service or see spare part list.</li> </ul>
<b>Compressed air leaking from the foot switch.</b>	<ul style="list-style-type: none"> <li>Pneumatic hoses reversed.</li> </ul>	<ul style="list-style-type: none"> <li>Reverse the connections.</li> </ul>
<b>Foot switch fails to switch on.</b>	<ul style="list-style-type: none"> <li>No pressure at the compressed air connection.</li> <li>Pneumatic hoses are kinked.</li> </ul>	<ul style="list-style-type: none"> <li>Inspect the compressed air lines and / or the compressor.</li> <li>Reposition the hoses.</li> </ul>
<b>Foot switch hoses cannot be disconnected from the unit.</b>	<ul style="list-style-type: none"> <li>Push-in connection is not released (Fig. 2).</li> </ul>	<ul style="list-style-type: none"> <li>Press the push-in connection ring on the hose side firmly while pulling on the hoses.</li> <li>Please use the release tools supplied (24).</li> </ul>
<b>Quick-stop not functioning (the nozzle blasts continuously).</b>	<ul style="list-style-type: none"> <li>Pneumatic connections are loose.</li> <li>Sandblasting hose worn.</li> <li>The connecting pressure at the external air connection (15, Fig. 1) is too small.</li> <li>Outer pressure regulator on the maintenance unit is altered (fixing screw is loosen).</li> </ul>	<ul style="list-style-type: none"> <li>Inspect the pneumatic connections for correct seating.</li> <li>Replace the sandblasting hose.</li> <li>Increase the external connecting pressure at a min. of 5 bar. The external connecting pressure is displayed on the manometer (8, Fig. 1).</li> <li>Turn the inner pressure regulator clockwise until stop. Activate foot pedal and set the pressure regulator on the maintenance unit at 5 bar (see manometer).</li> </ul>
<b>Sound of rushing air when the unit is operating.</b>	<ul style="list-style-type: none"> <li>A small amount of leakage is always present at the control valves, thus allowing (a minimum) of air to escape.</li> <li>Continue to turn the tank selection switch whilst activating the foot switch at the same time.</li> </ul>	<ul style="list-style-type: none"> <li>This is not a fault, but instead is the result of a pressure release opening which must not be blocked.</li> <li>Only turn the tank selection switch when the foot switch is not activated.</li> </ul>

# 9 Technical Specifications

Article number:	2969 1xxx	2969 4xxx
Nominal voltage / V:	230	120
Permissible mains voltage / V:	220 - 240	100 - 120
Mains frequency / Hz:	50/60	
Power consumption lighting / W:	16	
Working pressure / hPa (bar) [psi]: Standard Tank Prebonder Tank	1000 - 6000 (1 - 6) [14.5 - 87] 600 - 1000 (0,6 - 1) [8.7 - 14.5]	
Connecting pressure, external / hPa (bar) [psi]:	6000 - 8000 (6 - 8) [87 - 116]	
Air consumption / l/min [cfm]:	98 bei 6000 hPa [3.46 bei 87 psi]	
Cable length, approx. / m [inch]:	2 [78]	
∅ Compressed air hose / mm [mil]: interior exterior	4 [157] 6 [236]	
∅ Size of pipe union for external extraction unit / mm [inch]: interior exterior	35 [1.38] 40 [1.57]	
Dimensions (W x H x D) / mm [inch]:	395 x 320 x 510 [15.6 x 12.6 x 20.1]	
Blasting chamber volume / l [US gal]:	20 [5.3]	
Tank capacity, each/ ml [fl oz]:	1000 [34]	
Weight (empty), approx. / kg [lbs]:	11.5 [25.4]	
This product contains a light source of energy efficiency class:	F	
<b>Environmental conditions for operation *)</b>		
The ambient conditions according to EN 61010-1 apply.		
<b>Environmental conditions for safe operation *)</b>		
Ambient conditions:	Indoors	
Installation height, max.:	2,000 m [6,500 ft] above sea level	
Ambient temperature:	5 - 40 °C [41 - 104 °F]	
Relative humidity:	Max. 80 % at 31 °C [87.8 °F], decreasing linearly to 50 % at 40 °C [104 °F]	
Voltage fluctuations in the mains power supply from the nominal value, max.:	10 %	
Contamination level:	2	
Over-voltage category:	II	

\*) The product performs as specified in the technical data when used under ambient conditions. If the unit is operated outside these ambient conditions, safety devices contained in the product may stop operation, the optimum performance specified in the technical data may not be achieved, and the service life of components may be reduced.

The product fulfills the safety requirements of standard EN 61010-1 when operated within ambient conditions. Operation outside of these ambient conditions is not permitted, as electrical safety may be compromised.

## 10 Warranty

Provided the unit is properly used, Renfert warrants all parts of the BASIC prebender for a **period of 3 years**. In case of any claims for warranty original dealers' invoice is required.

Parts which are subject to natural wear and tear (wear parts) and consumables are excluded from the guarantee.

The warranty is voided in case of improper use; failure to observe the operating, cleaning, maintenance, and connection instructions; in case of independent repairs or repairs by unauthorized personnel; if spare parts from other manufacturers are employed, or; in case of unusual influences or influences not in compliance with the utilization instructions. Warranty service shall not extend the original warranty.

### 10.1 Liability Exclusion

Renfert GmbH shall be absolved from all claims for damages or warranty if:

- ▶ The product is employed for any purposes other than those cited in the operating instructions.
- ▶ The product is altered in any way other than those alterations described in the operating instructions.
- ▶ The product is repaired by other than an authorized facility or if any but Renfert OEM parts are employed.
- ▶ The product continues to be employed, despite obvious safety faults.
- ▶ In the case of mechanical impact or if the unit is dropped.
- ▶ Accessories or consumer products used, which are not supplied or authorised by Renfert GmbH.

## 11 Obligations regarding information

- ▶ Information on REACH and SVHC is available on our website [www.renfert.com/service](http://www.renfert.com/service), in the Support area.

## 12 Information on disposal for countries within the EC

**The unit must be disposed of by an authorized recycling operation. The selected firm must be informed of all possibly health-hazardous residues in the unit.**

To conserve and protect the environment, prevent environmental pollution and improve the recycling of raw materials, the European Commission adopted a directive that requires the manufacturer to accept the return of electrical and electronic units for proper disposal or recycling.

**Within the European Union units with this symbol should not therefore be disposed of in unsorted domestic waste.**

Please contact your local dealer for more information on proper disposal.

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Renfert GmbH • Untere Gießwiesen 2 • 78247 Hilzingen/Germany