**1 Intended use****Intended purpose**

Fabrication of custom trays and try-in dentures

**Patient target group**

- Patients with permanent teeth
- Edentulous patients

**Intended users / Special training**

- Dentists (clinical procedure)
- Dental laboratory technicians (laboratory procedure)
- No special training required.

**Use**

For dental use only.

**Description**

ProArt CAD Try-In are PMMA discs for the CAD/CAM fabrication of functional try-in mock ups for removable and fixed denture prosthetics as well as for impression and functional trays.

**Indications**

- Partial edentulism in the anterior and posterior region
- Complete edentulism

**Areas of application:**

- Functional try-in mock-ups for removable denture prosthetics
- Functional try-in mock-ups for fixed denture prosthetics
- Impression and functional trays for denture prosthetics

**Contraindications**

The use of the product is contraindicated if the patient is known to be allergic to any of its ingredients.

**Limitations of use**

- Permanent restorations
- Temporary restorations
- Abutment mock-ups
-  Do not re-use

**Side effects**

In individual cases local allergic reactions to methyl methacrylate materials have been reported.

**Interactions**

There are no known interactions to date.

**Clinical benefit**

- Custom trays: Functional impressions
- Try-in dentures: Checking of the chewing function, speech function and esthetic appearance

**Composition**

Polymethyl methacrylate

Contains: Methyl methacrylate, ethylene glycol dimethacrylate, dibenzoyl peroxide

# ProArt CAD Try-In

**[en] Instructions for Use**

Try-in material for the CAD/CAM technique

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# ivoclar

**2 Application****2.1 Processing with CAD/CAM equipment**

When designing the restoration, the specified minimum wall thicknesses and parameters of the final restoration must be complied with. If the minimum values are not observed, this can lead to failure (e.g. fracture).

ProArt CAD Try-In discs can be processed in CAD/CAM milling systems with a standard holder of 98.5 mm. Only use milling tools recommended for the milling system being used when processing the material.

 Observe the corresponding operating instructions.

Please note that the material can be subject to overheating if milling tools, polishing brushes or steam cleaners are handled incorrectly. This may result in the material being damaged.

**2.2 Processing after the milling procedure****Finishing**

After milling, separate the completed object from the disc with a tungsten carbide bur. Use a cross-cut tungsten carbide bur for finishing or shape adjustments, and use PMMA polishing equipment if necessary.

The impression tray made of ProArt CAD Try-In can be used immediately for impression taking. High gloss polishing is not required due to the short time during which the material remains in the mouth.

**Adjustment options**

If necessary, impression trays made of ProArt CAD Try-In can be adjusted using auto-polymerizing tray materials (e.g. SR Ivolen®). To do so, roughen or sandblast the surface. Then, apply and adapt the material.

**Impression materials**

The fit of the impression tray can be checked on the patient before impression taking. All impression materials which do not require moist storage can be used for impression taking with the respective adhesive.

**2.3 Application notes**

The following points have to be observed for the successful working with ProArt CAD Try-In:

- Observe the required minimum thickness.
- Only mill the discs in a compatible CAD/CAM system.
- Failure to observe the stipulated limitations of use and processing instructions may lead to failure.

**3 Reprocessing**

The product is not intended to be reprocessed or reused.

**4 Safety information**

- In the case of serious incidents related to the product, please contact Ivoclar Vivadent AG, Bendererstrasse 2, 9494 Schaan/Liechtenstein, website: [www.ivoclar.com](http://www.ivoclar.com), and your responsible competent authority.
- The current Instructions for Use are available in the download section of the Ivoclar Vivadent AG website ([www.ivoclar.com](http://www.ivoclar.com)).

**Warnings**

- Observe the Safety Data Sheet (SDS).
- Do not inhale grinding dust during finishing – use extraction equipment and wear a mask.

**Disposal information**

Remaining stocks must be disposed of according to the corresponding national legal requirements.

**Residual risks**

Users should be aware that any dental intervention in the oral cavity involves certain risks.

**5 Shelf life and storage**

This product does not require any special storage conditions.

**6 Additional information**

Keep material out of the reach of children!

Not all products are available in all countries.

The material has been developed solely for use in dentistry. Processing should be carried out strictly according to the Instructions for Use. Liability cannot be accepted for damages resulting from failure to observe the Instructions or the stipulated area of use. The user is responsible for testing the materials for their suitability and use for any purpose not explicitly stated in the Instructions.

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